**Bivona® Mid-Range Aire-Cuf® Adult Tracheostomy Tube**

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Smiths Medical ASD, Inc.  
Gary, Indiana 46406 USA

Leic Smiths Medical International, Ltd.  
Hythe, Kent, CT21 6SL UK

Australian Representative:  
Smiths Medical Australia Pty. Ltd.

61 Broad Street,  
Brisbane, Qld 4000, Australia

Eight Mile Plains  
Queensland 4113  
Australia

[www.smiths-medical.com](http://www.smiths-medical.com)

PNU OFU-S2-REV-A0010 08/09

smiths medical

English

### Bivona® Mid-Range Aire-Cuf® Adult Tracheostomy Tube

These instructions contain important information for the safe use of the product. Read the entire content of these instructions. For use, including Warnings and Cautions, before using this product. Failure to properly follow Warnings and Cautions can result in serious patient harm or death. It is the responsibility of the healthcare practitioner to assure that the instructions for use and maintenance are understood by and provided to the care provider.

**1. DESCRIPTION:**  
The Bivona® Mid-Range Aire-Cuf® Adult Tracheostomy Tube is a radiopaque silicone tube with an integral 15 mm connection and a mid-range inflatable cuff, which has a balloon inflation port and a tube inflation port. This product is manufactured with Bivona® SuperSlick® XL-PCN formulation.

The Bivona® Mid-Range Aire-Cuf® Adult tracheostomy tube is designed with a hollow obturator, which is removed after insertion.

This device is intended for single patient use. The contents of this package are sterile unless the package is opened or damaged.

**2. INTENDED USE:**  
This tube is intended to provide direct airway access for a tracheotomized patient for up to 29 days. It may be repositioned up to 10 times for single patient reuse.

**3. CONTRAINDICATIONS:**  
Do not use with lasers or electrocautery devices where their output may contact and damage the tube.

**4. WARNING:**  
Australian Representative: Smiths Medical Australia Pty. Ltd., 61 Broad Street, Brisbane, Qld 4000, Australia

Smiths Medical Australia Pty. Ltd., 61 Broad Street, Brisbane, Qld 4000, Australia

Eight Mile Plains, Queensland 4113, Australia

[www.smiths-medical.com](http://www.smiths-medical.com)

PNU OFU-S2-REV-A0010 08/09

Caution: Use of this tube is intended for Aspirator management tracheotomized patient concept and must be used in accordance with a tracheostomy airway management plan.

Forcet grip: Gently grip the tube to prevent damage to the tube.

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Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

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