
FILTERS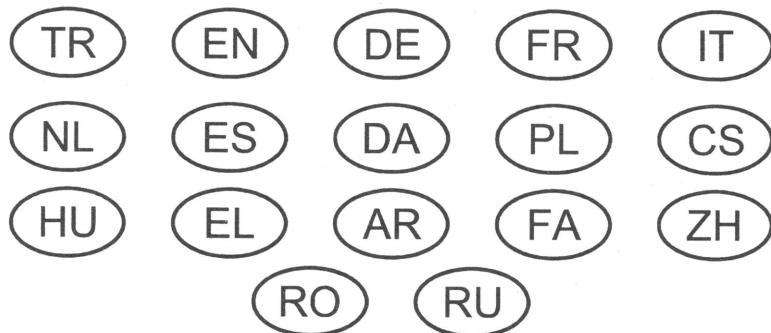

TÜRKÇE

FİLTRE KULLANIM KİLAVUZU

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ENGLISH

USER MANUAL FOR FILTER

The following instructions are provided for maximum performance and safety in use of breathing filters.

Description:

Single patient use HME and antibacterial/viral filter for breathing circuits in anesthesia, reanimation and intensive care for bacterial and viral filtration as well as humidification of patient airways.

Instruction for use:

Connect the device securely to breathing system, catheter mount, anesthesia/ventilator machine, closed suction system ensuring that the connections are tight and safe. Also check if present perfect seal of the cap for capnometry port.

Warnings:

1. The filter has been manufactured for single patient use and maximum recommended hours of use is 24 hours.
2. This filter is only used between the patient and breathing circuit.
3. The product is manufactured under clean room conditions. Do not use if the product and product package is damaged.
4. This product should be used only and exclusively under supervision of a physician.
5. Replace the filter in the event of increased resistance due to secretion from the patient.
6. The additional dead space of this device must be taken into consideration when used.

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