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Instructions for Use for

Shower guard (for tracheostomized and laryngectomized patients)

REF 102085

Read instructions for use carefully
 before each use of the product.

Product description

The Shower Guard made of medical-grade silicone enables safe showering for tracheostomized and laryngectomized patients by protecting the whole stoma and preventing the ingress of water while showering.

(Please follow the illustrated instructions, openings have to be on the bottom side of the product!)

Due to incorporated openings on the bottom of the shower guard an adequate air flow is provided while showering.

The Shower guard is also individually adjustable to different neck widths.

Incorporated braces stabilize the shape of the device and prevent the shower guard from collapsing while showering.

Made of medical-grade silicone the Shower Guard is at the same time very flexible and offers excellent wearing comfort.



Caution! Do not use for bathing and swimming.

Handling directions



- Guide the Shower Guard around the neck and fasten it according to the individual neck size of the patient by leading the provided press button through the openings of both lateral straps. (see fig. ①, ②)

The lateral straps should lie smoothly on the skin to provide optimum wearing comfort.

The curved cover has to be placed right over the stoma with the openings on the bottom side.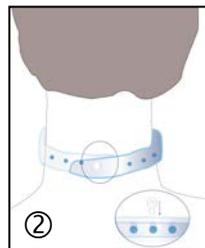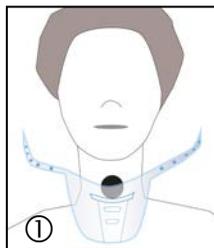

- Now showering is possible (see fig. ③)
- After showering the Shower Guard can be removed by releasing the button. (see fig. ④)

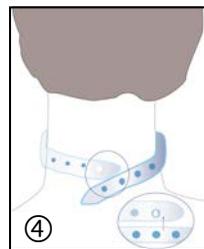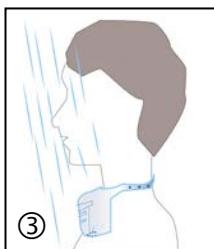Contraindications!

Do not apply if allergies are known to the material used!

Cleaning

The Shower Guard should be cleaned before each use.

Preliminary cleaning can be best done under running hand-warm water. In case a brush is needed, solely special cleaning brushes for silicone products can be used for basic cleaning. **(NOTICE: producer's instructions for use!).**

In order to clean the tubes thoroughly, a mild pH-neutral wash lotion can be used. After cleaning, once again rinse the Shower Guard thoroughly with hand-warm water.

The Shower Guard can also be rinsed using salt solution (0,9% NaCl solution).

As individually needed, it can be necessary to put the Shower Guard into boiling water (100 °C) for 5 minutes for cleaning. A period of 10 minutes should not be exceeded though. After cleaning with boiling water, let the Shower Guard chill down.

CAUTION!

The provided press button must not be cleaned with boiling water!

After wet cleaning wipe the Shower Guard dry with a clean, lint-free cloth.

After Cleaning with boiling water let the Shower Guard chill down. In case of non-compliance, there is danger of scold and burn injury.

CAUTION!

Please read instructions for use of producers of cleaning detergents.

Aggressive household detergents, concentrated alcohol or denture cleansers must not be used. The Shower Guard could be destroyed or damaged.

A dish washer, steamer or microwave must not be used for cleaning the Shower Guard!

Do not use if the packaging is damaged

Latex- free

Protect from sunlight during storage

Use by

Follow instructions for use!

Manufacturer

Reference number

Lot date