Astral Mobility Bag

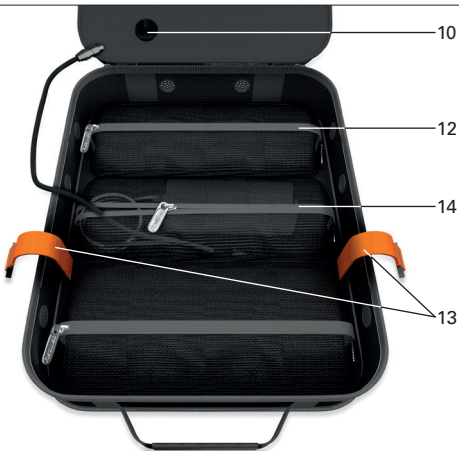
ACCESSORY

User Guide

English • Deutsch • Français • Italiano • Nederlands
• Español • Português • Svenska • Dansk • Norsk •
Suomi • Ελληνικά • Polski • Magyar • Türkçe • Český •
简体中文 • 繁體中文

Respiratory Care Solutions
Making quality of care easy

A**B****C**

RESMED

Astral Mobility Bag

User Guide

English

Thank you for choosing the Astral Mobility Bag.

Read the entire user guide before using the Mobility Bag.

Intended Use

The Mobility Bag has been designed to allow mobile use of the Astral ventilator.

WARNING

- Read the entire guide before using the Mobility Bag.
- To prevent overheating, remove the Astral power supply unit from the Mobility Bag when running Astral from an external power source.
- Ensure all alarm settings (including alarm volume) have been reviewed and adjusted accordingly for use in a mobile environment.

Note: ResMed recommends adjusting the alarm volume to the highest level before using the Astral device inside the Mobility Bag.

Mobility Bag at a glance

Refer to illustrations A, B, C & D

- | | |
|---------------------------------------|-----------------------------|
| 1. Carry handle | 9. Wheelchair strap (x4) |
| 2. Circuit flaps | 10. Cable port |
| 3. Protective cover | 11. Vents |
| 4. Access panel to Astral connections | 12. External battery pocket |
| 5. Air inlet | 13. Retention straps |
| 6. Attachment points | 14. Power supply pocket |
| 7. Backpack strap (x2) | 15. Device securing strap |
| 8. Shoulder strap | 16. Cable ports – back |

Set up

Connecting the power supply unit and battery

Refer to illustration B

1. Place bag on a flat surface and unzip the accessories compartment.
2. Unclip the retention straps.
3. Place the Astral power supply unit into the centre pocket. Position the cables towards the side so that they exit the pocket as shown.
4. If connecting to an external battery:
 - Connect the power supply DC cable to the external battery.
 - Place the external battery in the bottom pocket. Ensuring the DC cable exits the pocket.
 - Connect the external battery cable through the bottom part of the device compartment.

5. Push the DC cable through the cable port into the bottom of the device compartment.
6. Clip the retention straps and adjust the length if required.
7. Close the accessories compartment. Ensure all zippered compartments are closed.

Connecting the device

WARNING

- Ensure the air inlets and vents of the Mobility Bag and the device are kept clear. Blocking the air inlets and / or vents could lead to overheating of the device.
- Do not place Mobility Bag upright on the ground as this will block the air inlet of the Mobility Bag.
- When used with oxygen, keep sources of ignition away from the Mobility Bag.

The Mobility Bag has been designed to allow transfer of the device into the bag whilst in operation. When using oxygen, the oxygen inlet must be disconnected and reconnected.

Refer to illustration C

1. Unzip the device compartment and unfasten the device retention strap.
2. Place the device into the Mobility Bag and secure into position using the device retention strap. Ensure that the device retention strap does not compress the tubing.
3. Connect the DC plug of the power supply unit or battery to the rear of the Astral device.
4. If not already completed, attach the patient circuit and the antibacterial filter (if applicable) to the air outlet of the device.
5. Close the device compartment. Ensure all zippered compartments are closed.

Notes:

- *Always close the zippers and the flaps to protect the device from moisture.*
- *Ensure the protective cover is closed to avoid damaging the Astral screen.*
- *For more information on connecting patient circuit, see the Astral User Guide.*

Connecting oxygen and other accessories

WARNING

- When using supplemental oxygen inside the Mobility Bag, the maximum permitted flow rate is 6 L/min.
- Oxygen cannot be used with a double limb circuit.

Refer to illustration D

Oxygen and other accessories can be connected to the Astral device through the cable ports on the rear access panel of the Mobility Bag.

- Connect oxygen to the low flow oxygen inlet at the rear of the device.

For full instructions on using oxygen with the Astral device, refer to the Astral Clinical Guide.

- When using a pulse oximeter with the device, ResMed recommends placing the cables around the side of the device through the circuit flaps on the front panel.

For a full list of accessories, see the Ventilation accessories guide on www.resmed.com. If you do not have internet access, please contact your ResMed representative.

For use with a wheelchair

Refer to illustration E

1. Attach the four wheelchair straps to the attachment points on the side of the Mobility Bag.
2. Attach each of the four straps around a secure part of the wheelchair, tighten or loosen as required.

Note: *Alternatively, secure bag using the backpack or shoulder straps.*

For use as a backpack

Attach the backpack straps using the VELCRO® strips and secure to the attachment points at the top and bottom of the Mobility Bag.

For use over the shoulder

Secure the shoulder strap to the top two attachment points of the Mobility Bag.

For use with mains power

WARNING

- Ensure that the power cord does not pose a tripping or choking hazard.
 - When the Astral device is used in the Mobility Bag, the internal battery may stop charging in high ambient temperature conditions. If the internal battery is not charging, remove the Astral device and the power supply unit from the Mobility Bag to continue charging.
1. Connect the DC plug of the Astral external power supply unit to the rear of the Astral device.
 2. Before connecting the power cord to the ResMed power supply unit, ensure the end of the connector of the power cord is correctly aligned with the input socket on the power supply unit.
 3. Plug the other end of the power cord into the power socket.

Cleaning and maintenance

Wipe Astral Mobility Bag with a damp cloth and mild detergent. Allow to air dry.

Technical specifications

Nominal dimensions (L x W x H)	420 mm x 320 mm x 190 mm (16.53" x 12.59" x 7.48")
Weight	Approx. 2.3 kg (5.07 lb)
Water Ingress Protection	Astral and External battery protected against dripping water when tilted up to 15 degrees from specified orientation (IPX2).
Alarm volume range	45–75 dBA (in five steps) (when used inside the Mobility Bag)

Symbols

The following symbols may appear on your product or packaging.

 Indicates a Warning. A warning alerts you to possible injury or explains special measures for the safe and effective use of the product;
 Read instructions before use;  Manufacturer;  European authorized representative;  Batch code  Catalogue number;
IPX2 Drip proof;  No open flame.

Limited warranty

ResMed Ltd (hereafter 'ResMed') warrants that your ResMed product shall be free from defects in material and workmanship for a period of 12 months from the date of purchase by the initial consumer. This warranty is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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© 2013 ResMed Ltd.
Astral is a trademark of ResMed Paris SAS. Velcro is a trademark of Velcro Industries B.V.
Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.
Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.
EC REP ResMed (UK) Ltd 96 Jubilee Ave Milton Park Abingdon Oxfordshire OX14 4RW UK.