# AEROPart® - AB1-916

REF

HAB01-916

LOT

XXXXXX

QTY: 1

**de** Atemgasbefeuchter mit 410 mbar Sicherheitsventil  
**en** Bubble humidifier with 410 mbar safety valve  
**fr** Humidificateur-barboteur avec soupape de sécurité 410 mbar  
**it** Umidificatore a bolle con valvola di sfianto da 410 mbar  
**es** Humidificador de burbujas con válvula de seguridad de 410 mbar  
**pt** Humidificador de bolhas com válvula de segurança de 410 mbars  
**nl** Bubbelbevochtiger met 410 mbar veiligheidsklep  
**da** Boble luftfugter med 410 mbar sikkerhedsventil  
**no** Boble-luftfukter med 410 mbar sikkerhetsventil

**sv** Luftfuktare med 410 mbar säkerhetsventil  
**fi** Hengityskaasun kostutin, 410 mbar -turvaventtiili  
**ru** Увлажнитель Аеропарт, Рег. уд.: ФСЗ 2011/10317 от 08.09.2011, предохранительный клапан 410 мбар  
**pl** Nawilzacz bąbelkowy z zaworem bezpieczeństwa 410 milibarów  
**ro** Umidicator de gaz respirabil cu supapă de siguranță la 410 mbar  
**el** Υγραντήρας φυσαλιδων με βαλβίδα ασφαλείας 410 mbar  
**sl** Vlažilec mehurčkov z 410 mbarskim varnostnim ventilom  
**bg** Овлажнител за дихателна смес с 410 mbar предпазен вентил  
**cs** Zvlhčovač vdechovaného vzduchu s pojistným ventilem 410 mbar

YYY-YY-MM

4 251233 800233

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## en Instructions for use

**Intended purpose:** To humidify breathing gas in oxygen therapy.

**Caution:** Only to be used by trained and qualified personnel.

- Fill the container with water up to the

maximum line. Screw the cover tightly onto the container. The water line must not fall below the minimum line.

**Caution:** Use sterile water at room temperature.

- Use the connecting nut to connect the humidifier to the O<sub>2</sub> source.
- Warning:** Leaks can partially or completely reduce the supply of oxygen to the patient. Leak test and function test of the safety valve: Close the outlet at a flow of 3 l/min. The safety valve on the cover should respond. If the safety valve does not respond, check whether the cover is securely fixed on the container, and whether the connecting nut is screwed tightly on the O<sub>2</sub> source.
- Connect the oxygen tubing (not included in the delivery) to the outlet on the cover, and make sure that all of the connections are secure.
- Adjust the O<sub>2</sub> source to the prescribed flow (l/min). The safety valve emits an alarm when the downstream oxygen line is blocked or kinked. **Caution:** If the safety valve emits an alarm, check the downstream oxygen line (including the O<sub>2</sub> cannula or mask), and remove any blockages before reusing the product.
- Dispose of the humidifier in accordance with the hospital waste regulations. **Caution:** Disposable product for use on a single patient. Do not reuse. Reuse increases the risk of cross-contamination, poor performance and/or product malfunction.

Operating temperature:	10 °C - 40 °C
<b>Storage and transport</b>	
Temperature range:	-20 °C - 60 °C
Relative humidity:	< 85 %

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