TRACOE medical GmbH

Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany

Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

**TRACOE®**  
*technic*

## Instructions for Use

### TRACOE® cuff pressure monitor (cpm) REF 720

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### TRACOE® cuff pressure monitor (cpm) REF 720

---

2

Symbols, Symbole

---

4

(GB)

English

---

7

(D)

Deutsch

---

10

(F)

Français

---

13

(I)

Italiano

---

16

(E)

Español

---

19

(P)

Português

---

22

(DK)

Dansk

---

25

(FIN)

Suomi

---

28

(N)

Norsk

---

31

(HU)

Magyar

---

34

(S)

Svenska

---

37

(NL)

Nederlands

---

40

(GR)

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---

43

(KO)

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---

46

(CN)

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---

52

(ARA)

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**Warning. Please read instructions** • Gebrauchsanweisung beachten • Respecter les consignes d'utilisation • Osservare le istruzioni d'uso • Atención, observar instrucciones de uso • Observe as instruções de serviço • Overhold brugsanvisningen • Noudata käyttöohjetta • Følg bruksanvisningen • A használati utasítást figyelembe kell venni • Observera bruksanvisning • Gebruiksaanwijzing in acht nemen • Τηρείτε τις οδηγίες χρήσης • 경고. 사용설명서를 참고하십시오 • 注意使用说明 • تحذير. يرجى قراءة التعليمات.

**Reference number** • Bestellnummer • Référence de l'article • Codice articolo • Número de referencia • Número de referência • Bestillingsnummer • Tuotenumero • Bestillingsnummer • Megrendelési szám • Ordernummer • Bestellnummer • Αρ. παραγγελίας • 주문번호 • 订货号 • رقم الطلب •

**Lot number** • Chargenbezeichnung • Numéro de lot • Numero di lotto • Número de lote • Número de lote • Batchbetegnelse • Eränumero • Batchkode • Gyártási szám • Nummer på tillverkningssats • Lotnummer • Αριθμός παρτίδας • 카탈로그 번호 • 批号 • رقم السلسلة •

**Production date** • Herstellungsdatum • Date de fabrication • Data di produzione • Fecha da fabricación • Data de fabrico • Fremstillingsdato • Valmistuspäivä • Produktionsdato • Gyártási idő • Tillverkningsdatum • Datum van productie • Ημερομηνία κατασκευής • 제품생산일자 • 生产日期 • تاريخ الإنتاج •

**Expiration date** • Verwendbar bis • Date de péremption • Data di scadenza • Vencimiento • Data de validade • Kan anvendes indtil • Viimeinen käyttöpäivä • Skal brukes innen • Lejáratit idő • Utgångsdatum • Te gebruiken tot • Ανάλωση έως • 유 효 기 일 • 在此日期之前使用 • مدة صلاحية الاستعمال •

**STERILE EO**

**Sterile. Sterilized with Ethylene oxide** • Sterilisation mit Ethylenoxid • Stérilisé à l'oxyde d'éthylène • Sterilizzato con ossido di etilene • Esterilización con óxido de etileno • Esterilização com óxido de etileno • Sterilisation med ethylenoxid • Steriliit. Steriloitu etyleenioksidiilla • Sterilisering med etylenoksid • Sterilizáció etilénoxiddal • Sterilisering med etylenoksid • Sterilisatie met ethyleenoxide • Αποστείρωση με αιθυλενοξείδιο • 에틸렌으로 소독살균 되었습니다 • 乙撑氧 (Ethylene oxide) 灭菌 • معقم. معادة أو كسيد الأثيلين •

TRACOE medical GmbH

Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany

Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

## Instructions for Use

### TRACOE® cuff pressure monitor (cpm) REF 720

#### 1. General Information

The TRACOE® cuff pressure monitor (cpm) is a hand-held manometer intended for checking cuff pressure and inflating the cuffs of high-volume low-pressure cuffed endotracheal tubes or tracheostomy tubes.

**CAUTION:** The device is for use with endotracheal high-volume low-pressure cuffed tracheostomy tubes only.

#### 2. Description

The TRACOE® cuff pressure monitor consists of a manometer and a hand pump. Between the manometer and the pump, the device has an air outlet in the shape of a male Luer connector. Opposite the Luer connector and connected to it is a valve that can be opened and closed with a knurled screw.

A connecting tube REF 702 (length: 1m) is supplied with the TRACOE® cuff pressure monitor. The tube has a female Luer connector at one end and a male Luer connector at the other.

**STERILE** **EO**

**Sterilisation:** The connecting tube is sterilized with ethylene oxide. Contents are sterile unless the seal is broken or the package is opened or damaged.

**Storage:** The products must be stored in a cool and dry location. Temporary storage for  $\leq$  30 days at  $\leq +50^{\circ}\text{C}$  is possible. Protect the product from sunlight and extreme cold.

### **3. Indications**

The TRACOE® cuff pressure monitor is intended for use in checking cuff pressure and inflating the high-volume low-pressure cuffs of endotracheal tubes or tracheostomy tubes.

### **4. Instructions for Use**

#### **4.1 Testing the TRACOE® cuff pressure monitor**

Prior to use, the TRACOE® cuff pressure monitor must be checked for correct function. To test, close the valve by turning the knurled screw and seal the air outlet of the Luer connector with one finger. Then set up a pressure of 60 cm H<sub>2</sub>O with the hand pump and observe for three to five seconds whether the cuff pressure indicated on the display remains constant.

**CAUTION:** The unit can also generate pressure levels beyond the scale range of 100 cm H<sub>2</sub>O. Such pressure levels of more than 100 cm H<sub>2</sub>O are not shown by the display scale and may impair the measuring accuracy in the long term.

If the cuff pressure decreases, even after repeating the test, return the device to TRACOE medical for inspection. The same applies if the needle of the TRACOE® cuff pressure monitor is not within the grey area of the display at a pressure of 0 cm H<sub>2</sub>O. The latter can occur following an improper and excessive increase in pressure.

#### **4.2 Cuff Leakage Test**

Always test the cuff and the inflation system for leakage before inserting an endotracheal or a tracheostomy tube. To do this, inflate the cuff with the TRACOE® cuff pressure monitor to a pressure of 50 cm H<sub>2</sub>O and observe the cuff for possible deflation for about 1 minute. Before inserting the tube into the airway, always deflate the cuff completely, using a syringe.

#### **4.3 Applying the TRACOE® cuff pressure monitor**

Inflate the cuff of the endotracheal tube, or the tracheostomy tube, with the TRACOE® cuff pressure monitor immediately after insertion into the airway. To achieve good expansion and a close fit we recommend brief initial inflation of the cuff to e. g. 50 cm H<sub>2</sub>O. Then immediately deflate the cuff by turning the knurled screw until the indicator needle is within the green area (approx. 15 to 30 cm H<sub>2</sub>O).

**CAUTION:** The cuff pressure must not exceed 20 to 25 cm H<sub>2</sub>O during expiration.

For continuous observation of cuff pressure, the cuff pressure monitor can be connected permanently to the endotracheal or tracheostomy tube using the connecting tube supplied.

When a pressure measuring unit such as the TRACOE cpm is connected to the valve of an endotracheal tube or a tracheostomy tube, there will always be a small amount of air flowing from the cuff into the unit's measuring chamber, which was previously depressurised. As a result, there will be a slight pressure loss in the cuff. This means that the cuff pressure before the connection of the measuring unit was always slightly higher than the pressure value that is displayed at the connected measuring unit. The pressure loss due to the filling of a measuring chamber through the cuff is normal. If the measuring unit is connected frequently without the cuff being refilled, one may get the impression that the cuff is leaking.

REF 720; 1 unit = 1 TRACOE® cuff pressure monitor with 1 connecting tube (REF 702)

### **Spare Connecting Tube**

REF 702; 1 unit = 10 pieces of 1 m each, in individual sterile packages

REF 703; 1 unit = 10 pieces of 2 m each, in individual sterile packages

## **5. Returns**

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website ([www.tracoe.com](http://www.tracoe.com)).

## **6. General Terms and Conditions**

The sale, delivery and return of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website ([www.tracoe.com](http://www.tracoe.com)).

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