Directions for use

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- ENG ASSKEA Suction Canisters
- D ASSKEA Absaugbehälter
- F ASSKEA Bocal d'aspiration
- I ASSKEA Sistemi di Aspirazione
- NL ASSKEA Afzuighouders
- E ASSKEA Contenedores de Aspiración

Port identification key (Fig. 1)

- A. Vacuum Port
- B. Patient Port

Standard Setup - liner

1. In standard setup, ensure that only liners with white lid are used.
2. Remove from packaging; twist and fully extend liner (Fig. 2). Insert the liner into the reusable canister! Press down firmly on the lid to ensure proper seal (Fig. 3).
3. Connect vacuum to top end of the canister bracket.
4. Connect pre-attached liner tubing to the bottom end of the canister bracket (Fig. 4).
5. Connect patient tubing to patient port.
6. Turn on the vacuum and proceed with suction.
7. After procedure, close the liner by removing the pre-attached tubing from canister tee and fit to patient port.
8. Remove liner by lifting upward on the rim of the lid.

Note:

During the installation of the hard canister, ensure that it is undamaged. Use only hard canisters provided by ASSKEA GmbH. Do not tamper, modify or dismantle the hard canister. In case of damage, dispose the hard canister and use another undamaged one. Replace the liner according to the applying hygiene regulations, weekly at the latest, for each new patient or if it is full. Dispose of in compliance with hospital protocol. Product contents are considered potentially hazardous.

In the event of vacuum loss:

Check to ensure the lid on the canister is sealed properly. Check that all connections are airtight and that the vacuum is turned on. Note that the liner may collapse if air leaks into the canister.

Hard canister cleaning:

Rinse with water. Use a chemical disinfectant that cannot damage polycarbonate, rinse again with water, wipe with a dry, clean and non-fluffing cloth or with medical air. Steam-sterilizable up to max. 121°C.

For automated means of cleaning the hard canister, use an agent that cannot damage polycarbonate and set the temperature at a maximum of 60 °C. Replace the hard canister every 4 weeks at the latest and for each new patient!

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© Reproduction of this document, even in part, is not permitted. Contents shall not be reproduced, duplicated or given to third party, either in full or in part, without the written authorization of the ASSKEA GmbH.