## *DeVilbiss*<sup>®</sup> Oxygen Concentrator Instruction Guide

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## TABLE OF CONTENTS

Specifications.....	3
Important Safeguards .....	9
Introduction.....	9
Why Your Physician Prescribed Supplemental Oxygen.....	10
How Your Concentrator Works.....	10
Important Parts of Your Concentrator .....	11
Setting Up Your Concentrator .....	13
Before Operating Your Concentrator .....	13
Operating Your Concentrator .....	15
Reserve Oxygen System .....	17
Caring for Your Concentrator .....	17
Minor Troubleshooting .....	19
Minor OSD® Troubleshooting.....	19

**CAUTION—Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.**

**⚠ WARNING—Under certain circumstances, oxygen therapy can be hazardous. Seeking medical advice before using an oxygen concentrator is advisable.**

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**SPECIFICATIONS**

	DEVILBISS 5-LITER SERIES		DEVILBISS 3-LITER SERIES		DEVILBISS LT SERIES	
Catalog Number	505DZ/505DS	505KS/505KZ/505NS	303DZ/303DS/303DA	303IZ/303IS	501DZ	501IZ
Delivery Rate	1/2 to 5 LPM	1/2 to 5 LPM	1/2 to 3 LPM	1/2 to 3 LPM	1/2 to 5 LPM	1/2 to 5 LPM
Oxygen Percentage	1 LPM = 94% ± 3% 2-4 LPM = 95% ± 3% 5 LPM = 93% ± 3%	<b>220-230V~, 50Hz</b> 1 LPM = 93% ± 3% 2-4 LPM = 95% ± 3% 5 LPM = 85% +6% -3% <b>230V~, 60Hz</b> 1 LPM = 92% ± 3% 2-4 LPM = 95% ± 3% 5 LPM = 92% ± 3%	1/2-3 LPM = 93% ± 3%	<b>220-240V~, 50Hz</b> 1/2-3 LPM = 93% ± 3% <b>220-230V~, 60Hz</b> 1/2-3 LPM = 93% ± 3%	1-4 LPM = 93% ± 3% 5 LPM = 88% +3% -5%	<b>220-240V~, 50Hz</b> 1-3 LPM = 93% ± 3% 4 LPM = 90% ± 3% 5 LPM = 80% +3% -5% <b>230V~, 60Hz</b> 1-3 LPM = 93% ± 3% 4 LPM = 90% +5% -3 % 5 LPM = 82% ± 3%
Maximum Recommended Flow	5 LPM	4 LPM	3 LPM	3 LPM	5 LPM	5 LPM
Sound Level	49 dBA Avg.	45 dBA Avg.	51 dBA Avg.	51 dBA Avg.	51 dBA Avg.	49 dBA Avg.
Outlet Pressure	8.5 psig (58.6 kPa)	8.5 psig (58.6 kPa)	8.5 psig (58.6 kPa)	8.5 psig (58.6 kPa)	8.5 psig (58.6 kPa)	8.5 psig (58.6 kPa)
Electrical Rating	115V~, 60Hz, 4.4Amp	220-230V~, 50Hz, 1.7 Amp 230V~, 60 Hz, 2.3 Amp*	115V~, 60Hz, 2.1 Amp	220-240V~, 50Hz, 1.1 Amp 220-230V~, 60 Hz, 1.2 Amp	115V~, 60Hz, 5 Amp	220-230V~, 50Hz, 1.7 Amp 230V~, 60 Hz, 1.7 Amp
Operating Voltage Range	95-132V~, 60Hz	187-253V~, 50Hz 195.5-253V~, 60 Hz*	95-132V~, 60Hz	187-264V~, 50Hz 187-253V~, 60 Hz	95-132V~, 60Hz	187-264V~, 50Hz 193-253V~, 60 Hz
Power Consumption	400 Watts Average	330 Watts Avg @230V, 50Hz 340 Watts Avg @230V, 60Hz	225 Watts Average	240 Watts Avg @230V, 50Hz 270 Watts Avg @230V, 60Hz	415 Watts Average	315 Watts Avg @230V, 50Hz 360 Watts Avg @230V, 60Hz
Weight	51 lbs. (23 kg.)	53 lbs. (24 kg.)	37 lbs. (17 kg.)	41 lbs. (19 kg.)	47 lbs. (21 kg.)	47 lbs. (21 kg.)
Dimensions	27.75"H x 16"W x 14"D (70.5 x 40.6 x 35.6 cm)	27.75"H x 16"W x 14"D (70.5 x 40.6 x 35.6 cm)	23"H x 13.87"W x 12"D (58.4 x 35.2 x 30.5 cm)	23"H x 13.87"W x 12"D (58.4 x 35.2 x 30.5 cm)	27.75"H x 16"W x 14"D (70.5 x 40.6 x 35.6 cm)	27.75"H x 16"W x 14"D (70.5 x 40.6 x 35.6 cm)
Pressure Relief Valve	35 psig ± 3 psig (241 kPa ± 20.7 kPa)	35 psig ± 3 psig (241 kPa ± 20.7 kPa)	25 psig ± 3 psig (172 kPa ± 20.7 kPa)	25 psig ± 3 psig (172 kPa ± 20.7 kPa)	35 psig ± 3 psig (241 kPa ± 20.7 kPa)	35 psig ± 3 psig (241 kPa ± 20.7 kPa)
Operating System	Pressure Sensing/ Pressure Swing	Pressure Sensing/ Pressure Swing	Pressure Sensing/ Pressure Vacuum	Pressure Sensing/ Pressure Vacuum	Timed/ Pressure Swing	Timed/ Pressure Swing
Operating Altitude (tested at 21°C only)	Up to 1500 meters (4,921 feet) above sea level without degradation of performance across the operating voltage range. From 1500 meters (4,921 feet) to 4000 meters (13,123 feet), the end user should check with their Sunrise Medical provider for the suitability.					
Operating Environment	5°C to 40°C, humidity range of 15% to 95%, non-condensing*					
Recommended Operating Ambient Range (tested at sea level)	5°C to 40°C, humidity range of 15% to 95%, non-condensing*				5°C to 35°C, humidity range of 15% to 95%, non-condensing	
For units equipped with an OSD, the visible "low oxygen" indicator will activate at the following levels	85%±2%	82%+1%-3%	85%±2%	85%±2%	Not Applicable	Not Applicable
Storage Conditions	-40°C to 70°C, humidity range of 10% to 100%, including condensation					
Equipment Class and Type	Class II Equipment Double Insulated <input type="checkbox"/> Type B Applied Part 					
Approval Body and Safety Standard	CSA CAN/CSA-C22.2 No. 601.1-M90 IEC 601-1	TUV IEC 601-1 + A1 + A2 EN 60601 + A1 + A2 ISO8359: 1996 (505KZ excluded)	CSA CAN/CSA-C22.2 No. 601.1-M90 IEC 601-1 (303DA excluded)	TUV IEC 601-1 ISO 8359:1991 Complies with ISO 8359:1996 (303IZ excluded)	CSA CAN/CSA-C22.2 No. 601.1-M90 IEC 601-1	CSA CAN/CSA-C22.2 No. 601.1-M90 IEC 601-1
EMC Compliance To:	IEC801-2 through 5 CISPR 11/EN55011	IEC 601-1-2	IEC801-2 through 5 CISPR 11/EN55011	IEC 601-1-2	IEC801-2 through 5 CISPR 11/EN55011	IEC 601-1-2

Specifications subject to change without notice.

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**Declaration of Conformity****Manufacturer:**

Sunrise Medical  
Respiratory Products Division

**Address:**

100 DeVilbiss Drive  
Somerset, PA 15501 USA

**Product Designation:** Oxygen  
Concentrators

**Catalog Number:**

303IS, 505KS, 505NS

We herewith declare that the above-mentioned product complies with the requirements of EC directive 93/42/EEC and the following (see Specifications Matrix for specific compliance by model):

**Quality System Standards**

**Applied:**  
ISO9001/EN46001

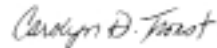
**Notified body RWTÜV****MDD Annex II Applied****Safety Standards Applied:**

IEC 601-1+A1+A2  
DIN VDE 0750 TEIL 1/12.91  
ISO8359: 1996

**EMC Compliance To:**

IEC801-2 through 5  
CISPR 11/EN55011  
IEC 601-1-2

**Place:** Somerset, PA

**Signature:**

**Name:** Carolyn D. Troast

**Title:** Director of quality and regulatory affairs

**European Contact:**

Sunrise Medical Ltd.  
Sunrise Business Park  
High Street  
Wollaston, West Midlands  
DY8 4PS  
ENGLAND  
44-138-444-6688

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**Firma:**

**Nome:** Carolyn D. Troast

**Titolo:**  
Director of quality and regulatory affairs

**Contatto in Europa:**

Sunrise Medical Ltd  
Sunrise Business Park  
High Street  
Wollaston, West Midlands  
DY8 4PS  
ENGLAND  
44-138-444-6688

## IMPORTANT SAFEGUARDS

**!** Read this entire guide before using your DeVilbiss concentrator. Important information is highlighted by these terms throughout this guide:

**DANGER**—Urgent safety information for hazards that will cause serious injury or death.

**WARNING**—Important safety information for hazards that might cause serious injury.

**CAUTION**—Information for preventing damage to the product.

**NOTE**—Information to which you should pay special attention.

Important safeguards are indicated throughout this guide; pay special attention to all safety information.

### INTRODUCTION

This instruction guide will acquaint you with the DeVilbiss oxygen concentrator. Make sure that you read and understand this guide before operating your unit. Important safeguards are indicated throughout this guide; pay special attention to all safety information. Contact your Sunrise Medical equipment provider should you have any questions.

**!** **WARNING**—For your safety, the oxygen concentrator must be used according to the prescription determined by your physician.

**!** **DANGER**—Oxygen causes rapid burning. Do not smoke while your oxygen concentrator is operating, or when you are near a person utilizing oxygen therapy. Do not use within 5 feet (1.6m) of hot, sparking objects or naked sources of flame.

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**!** **PERICOLO - NON FUMARE**

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### Why Your Physician Prescribed Supplemental Oxygen

Today, many people suffer from heart, lung, and other respiratory diseases. Many of these people can benefit from supplemental oxygen therapy. Your body requires a steady supply of oxygen to function properly. Your physician prescribed supplemental oxygen for you because you are not getting enough oxygen from room air alone. Supplemental oxygen will increase the amount of oxygen that your body receives.

Supplemental oxygen is not addictive. Your physician prescribed a specific oxygen flow to improve symptoms such as headaches, drowsiness, confusion, fatigue, or increased irritability. If these symptoms persist after you begin your supplemental oxygen program, consult your physician.

### How Your DeVilbiss Oxygen Concentrator Works

Oxygen concentrators are the most reliable, efficient, and convenient source of supplemental oxygen available today. The oxygen concentrator is electrically operated. The unit separates oxygen from room air which allows high-purity supplemental oxygen to be delivered to you through the oxygen outlet. Although the concentrator filters the oxygen in a room, it will not affect the normal amount of oxygen in your room.

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## IMPORTANT PARTS OF YOUR CONCENTRATOR

Please take time to familiarize yourself with your DeVilbiss oxygen concentrator before operating.

### Front View

- Operating instructions
- Green Power light – illuminates when your concentrator is operating.
- Power Switch  
| = ON  
0 = OFF
- Flow meter knob
- Flow meter
- Circuit breaker – (only 115 volt units) resets the unit after electrical overload shut-down.
- Oxygen outlet – oxygen is dispersed through this port.
- Normal Oxygen (green) light – ONLY on units with the OSD® (see page 15).
- Low Oxygen (yellow) light – ONLY on units with the OSD® (see page 15).
- Red Service Required light – when illuminated contact your Sunrise Medical provider.

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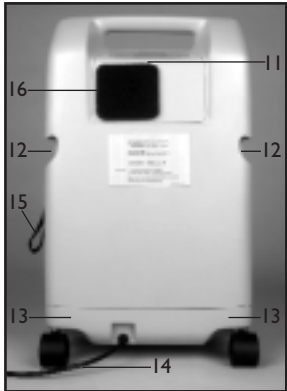
Figure 2.  
Figura 2.  
Abb 2.

### Back View

11. Important Safeguards
12. Handgrip
13. Exhaust
14. Power cord (only 115 volt units) or IEC power connector with fuse (230 volt units).
15. Line cord strap
16. Air filter – prevents dirt, dust, and lint from entering your unit.

### Accessories

The accessories below are approved for use with DeVilbiss Oxygen Concentrators:

- Bubble Humidifier (200 ml Capacity)  
- 50/case ..... HUM16
- Cannula (disposable)  
- 50/case ..... CAN00
- Cannula w/ 2.1 m (7 ft)  
Oxygen Tubing (disposable)  
- 50/case ..... CAN70
- Oxygen Outlet Connector  
- Plastic - 1/pack ..... CN100  
- Metal - 1/pack ..... CN200
- Oxygen Tubing (disposable)  
- 2.1 m (7 ft) - 50/case .... OST07  
- 4.2 m (14 ft) - 50/case ... OST15  
- 7.5 m (25 ft) - 25/case ... OST25  
- 15 m (50 ft) - 15/case ... OST50

**NOTE**—The use of certain humidifiers and accessories not specified above may impair the oxygen concentrator's performance.

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## SETTING UP YOUR OXYGEN CONCENTRATOR

1. Position your unit near an electrical outlet in the room where you spend most of your time.

**⚠ DANGER**—Keep the oxygen concentrator at least 5 feet (1.6 m) from hot, sparking objects or naked sources of flame.

**NOTE**—Do not connect to an electrical outlet controlled by a wall switch. No other appliances should be plugged into the wall outlet.

2. Position your unit at least 6 inches (16 cm) from walls, draperies, or any other objects that might prevent the proper flow of air in and out of your oxygen concentrator. The oxygen concentrator should be located so as to avoid pollutants or fumes.

## BEFORE OPERATING YOUR OXYGEN CONCENTRATOR

1. Before operating your unit, always check to be sure the air filter (located on the back of your unit) is clean. Proper cleaning of this filter is discussed in the Caring For Your Concentrator section on page 17.
2. Attach the appropriate oxygen accessories to the oxygen outlet.

### Oxygen Tubing Connection:

- a) Thread the oxygen outlet connector onto the oxygen outlet.
- b) Attach the oxygen tubing directly to the connector. **Figure 3.**

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Figure 4.  
Figura 4.  
Abb 4.

Figure 5.  
Figura 5.  
Abb 5.

### Oxygen Tubing Connection With Humidification:

If your physician has prescribed an oxygen humidifier as part of your therapy, follow these steps:

- Fill the humidifier bottle with distilled water. Do not overfill. (If using a prefill, go to letter b.)
  - Thread the wing nut located on the top of the humidifier bottle to the oxygen outlet so that it is suspended **Figure 4**. Make sure it is securely tightened.
  - Attach the oxygen tubing directly to the humidifier bottle outlet fitting **Figure 5**.
- Your physician has prescribed either a nasal cannula, catheter, or face mask. In most cases, they are already attached to the oxygen tubing. If not, follow the manufacturer's instructions for attachment.
  - Remove the power cord completely from the line cord strap. Make sure the power switch is in the "Off" position, and insert the plug into the wall outlet. The unit has a two-prong plug and is double insulated to guard against electric shock.

**NOTE**—(only 115 volt units) The plug on the DeVilbiss oxygen concentrator has one blade wider than the other. To reduce the risk of electric shock, this plug is intended to fit in a wall outlet only one way. Do not attempt to defeat this safety feature.

**⚠ WARNING—Improper use of the power cord and plugs can cause a burn, fire, or other electric shock hazards. Do not use the unit if the power cord is damaged.**

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## OPERATING YOUR DEVILBISS OXYGEN CONCENTRATOR

**⚠ DANGER**—Oxygen causes rapid burning. Do not smoke while your oxygen concentrator is operating, or when you are near a person utilizing oxygen therapy. Keep the oxygen concentrator at least 5 feet (1.6 m) from hot, sparking objects or naked sources of flame.

1. Press the power switch to the "On" position. When the unit is turned on, the Power light will illuminate, and the patient alert system – Service Required light and audible signal – will briefly alarm.

### DEVILBISS OXYGEN CONCENTRATORS WITH OSD ONLY

The OSD (Oxygen Sensing Device) is a device within your concentrator that monitors the oxygen produced by your unit. When the unit is turned "On," all four lights (Power, Service Required, Low Oxygen, and Normal Oxygen) on the front panel will illuminate briefly. After a few seconds, only the Power and Normal Oxygen lights will remain lit.

The OSD lights on the front panel are defined as follows:

- Green Normal Oxygen light—acceptable oxygen level.
- Yellow Low Oxygen light—below an acceptable oxygen level.

If the oxygen purity falls below the acceptable level, the green Normal Oxygen light will shut off and the yellow Low Oxygen light will illuminate. Switch to your reserve oxygen system. Refer to the Minor OSD Troubleshooting section in this guide on page 19 and contact your Sunrise Medical provider.

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Figure 6.  
Figura 6.  
Abb 6.

As an added safety feature should the oxygen purity continue to drop, an intermittent audible signal will sound. Contact your Sunrise Medical provider immediately. Do not attempt any other maintenance.

**NOTE**—(DeVilbiss 3 & 5 Liter Series) If the Service Required light illuminates and the audible signal alarms continuously during operation, the unit will automatically turn off. Turn the unit “Off.” Then turn the unit “On” again. If the Service Required light remains on, refer to the Minor Troubleshooting chart on page 19 and contact your Sunrise Medical provider.

**NOTE**—(DeVilbiss LT Series) If the Service Required light illuminates and the audible signal alarms but the unit is not operating, there is no power to the unit. Refer to the Minor Troubleshooting chart on page 19 and contact your Sunrise Medical provider if necessary.

2. Check the flow meter to make sure that the flow meter ball is centered on the line next to the prescribed number of your flow rate.

**CAUTION**—It is very important to follow your oxygen prescription. Do not increase or decrease the flow of oxygen—consult your physician.

**NOTE**—Your Sunrise Medical provider may have preset the flow meter so that it can not be adjusted.

**NOTE**—If the flow meter knob is turned clockwise, the flow decreases (and eventually will shut off the oxygen flow). If the knob is turned counterclockwise, the flow increases.

**NOTE**—(DeVilbiss 3 & 5 Liter Series) If the flow meter ball is on 0, it may activate the Restrict-ed Flow Alarm. The unit will continue to run, however an intermittent audible and visual alarm will be activated. Adjust the flow meter to your prescribed flow.

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3. Your DeVilbiss concentrator is now ready for use, properly position the cannula or mask **Figure 6**. Allow 20 minutes for oxygen concentrator to reach stated performance.

### RESERVE OXYGEN SYSTEM

As a precaution, your Sunrise Medical provider should supply you with a reserve oxygen system. If your unit loses electrical power or fails to operate correctly, the Patient Alert System will sound to signal you to switch to your reserve oxygen system (if provided) and contact your Sunrise Medical provider.

### CARING FOR YOUR DEVILBISS OXYGEN CONCENTRATOR

**Note**—Use no lubricants, oils or grease.

**⚠ WARNING**—Before attempting any cleaning procedures, turn the unit “Off.”

#### Oxygen Humidifier (reusable bottles)

If your physician has prescribed humidification, clean your humidifier bottle daily. Follow the instructions supplied by the manufacturer. If no cleaning instructions were supplied, follow these steps:

1. Wash the humidifier in a solution of hot water and dish-washing detergent **Figure 7**.
2. Soak the humidifier in a solution of one part white vinegar to three parts hot water for 30 minutes **Figure 8**. This solution acts as a germicidal agent.
3. Rinse thoroughly with hot water and refill with distilled water **Figure 9** for use. Do not overfill.

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Figure 9.  
Figura 9.  
Abb 9.

### Cannula, Mask, and Tubing

Clean and replace the cannula, mask, and tubing as instructed by your Sunrise Medical provider.

### Air Filter and Oxygen Outlet Connector

The air filter and connector should be cleaned at least once a week. To clean, follow these steps:

1. Remove the air filter, located on the back of the unit. Remove the oxygen outlet connector (if used).
2. Wash in a solution of warm water and dishwashing detergent **Figure 10**.
3. Rinse thoroughly with warm tap water and towel dry. The filter should be completely dry before reinstalling.

**CAUTION—To prevent product damage, do not attempt to operate the unit without the air filter or while the filter is still damp.**

### Exterior Cabinet

**⚠ WARNING—To avoid electric shock, do not remove the concentrator cabinet. The cabinet should only be removed by a qualified Sunrise Medical technician. Do not apply liquid directly to the cabinet or utilize any petroleum-based solvents or cleaning agents.**

Clean the concentrator exterior cabinet by using a damp cloth or sponge with a mild household cleaner and wipe it dry.

Figure 10.  
Figura 10.  
Abb 10.

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**MINOR TROUBLESHOOTING**

The following troubleshooting chart will help you analyze and correct minor oxygen concentrator malfunctions. If the suggested procedures do not help, switch to your reserve oxygen system and call your Sunrise Medical homecare provider. Do not attempt any other maintenance.

**⚠ WARNING**–To avoid electric shock hazard, do not remove the cabinet. The cabinet should only be removed by a qualified Sunrise Medical homecare technician.

**MINOR TROUBLESHOOTING CHART**

Symptom	Possible Cause	Remedy
A. Unit does not operate. Power light is off when the power switch is "On." Audible alarm is pulsing and Service Required light is flashing.	<ol style="list-style-type: none"> <li>1. Power cord not properly inserted into wall outlet.</li> <li>2. No power at wall outlet.</li> <li>3. Oxygen concentrator circuit breaker activated (only 115 volt units).</li> </ol>	<ol style="list-style-type: none"> <li>1. Check power cord connection at the wall outlet. On 230 volt units, also check the back of the unit.</li> <li>2. Check your home circuit breaker and reset if necessary. Use a different wall outlet if the situation occurs again.</li> <li>3. Press the concentrator circuit breaker reset button located below the power switch (only 115 volt units). Use a different wall outlet if the situation occurs again.</li> </ol> <p><b>If the above remedies do not work, contact your Sunrise Medical provider.</b></p>
B. (DeVilbiss 3 & 5 Liter Series) Unit does not operate but the Power light is on when the power switch is "On." Red Service Required light is illuminated. Audible alarm may be sounding.	<ol style="list-style-type: none"> <li>1. Air filter is blocked.</li> <li>2. Exhaust is blocked.</li> </ol>	<ol style="list-style-type: none"> <li>1. Check the air filter. If the filter is dirty, wash it following the cleaning instructions on page 18.</li> <li>2. Check the exhaust area; make sure there is nothing restricting the unit exhaust.</li> </ol> <p><b>If the above remedies do not work, contact your Sunrise Medical provider.</b></p>
C. (DeVilbiss LT Series) Unit operates, the Power light is on when the power switch is "On." Red Service Required light is illuminated. Audible alarm may be sounding.	<ol style="list-style-type: none"> <li>1. Air filter is blocked.</li> <li>2. Exhaust is blocked.</li> </ol>	<ol style="list-style-type: none"> <li>1. Check the air filter. If the filter is dirty, wash it following the cleaning instructions on page 18.</li> <li>2. Check the exhaust area; make sure there is nothing restricting the unit exhaust.</li> </ol> <p><b>If the above remedies do not work, contact your Sunrise Medical provider.</b></p>
D. (DeVilbiss 3 & 5 Liter Series) Limited flow of oxygen, audible alarm is pulsing and Service Required light is flashing.	<ol style="list-style-type: none"> <li>1. Blocked or defective cannula, catheter, face mask, or oxygen tubing.</li> <li>2. Blocked or defective humidifier bottle.</li> <li>3. Flow meter set too low.</li> </ol>	<ol style="list-style-type: none"> <li>1. Detach cannula, catheter, or face mask. If proper flow is restored, clean or replace if necessary. Disconnect the oxygen tubing at the oxygen outlet. If proper flow is restored, check oxygen tubing for obstructions or kinks. Replace if necessary.</li> <li>2. Detach the humidifier from the oxygen outlet. If proper flow is obtained, clean or replace humidifier.</li> <li>3. Set flow meter to prescribed flow rate.</li> </ol> <p><b>If the above remedies do not work, contact your Sunrise Medical provider.</b></p>
E. If any other problems occur with your oxygen concentrator:		<ol style="list-style-type: none"> <li>1. Turn your unit "Off." Switch to your reserve oxygen system, and contact your Sunrise Medical provider immediately.</li> </ol>

**MINOR TROUBLESHOOTING CHART - ONLY OSD Concentrators**

Visual Indicator Lights	Possible Cause	Remedy
A. Both the green Normal Oxygen and the yellow Low Oxygen lights are either on or off.	<ol style="list-style-type: none"> <li>1. OSD malfunction.</li> </ol>	<ol style="list-style-type: none"> <li>1. Contact your Sunrise Medical provider.</li> </ol>
B. Yellow Low Oxygen light is on or the yellow Low Oxygen light is on and the intermittent audible signal is sounding.	<ol style="list-style-type: none"> <li>1. Flow meter is not properly set.</li> <li>2. Air filter is blocked.</li> <li>3. Exhaust is blocked.</li> </ol>	<ol style="list-style-type: none"> <li>1. Ensure the flow meter is properly set to the prescribed number.</li> <li>2. Check the air filter. If the filter is dirty, wash it following the cleaning instructions on page 18.</li> <li>3. Check the exhaust area; make sure there is nothing restricting the unit exhaust. <b>If the above remedies do not work, contact your Sunrise Medical provider.</b></li> </ol>

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**Sunrise Medical Canada, Inc.**
**Sunrise Medical Pty. Limited**