
# PEP S-System

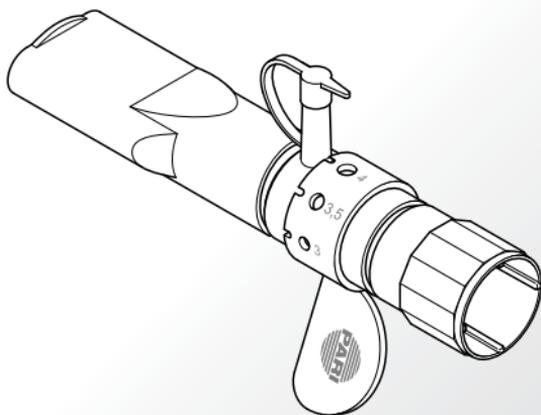CE 0123

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**en** Instructions for use ..... 25

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# Instructions for use

Information as of: 2013-06. We reserve the right to make technical changes.

<b>IMPORTANT INFORMATION</b> .....	27
General .....	27
Information about the instructions for use .....	27
Structure of safety instructions .....	27
Contraindications and side effects .....	28
Instructions for application .....	28
Hygiene .....	28
<b>PRODUCT DESCRIPTION</b> .....	29
Components .....	29
Intended use .....	30
Material information.....	30
Product combinations.....	30
Description of function.....	31
<b>APPLICATION</b> .....	32
Assembling the PARI PEP S system .....	32
Perform treatment .....	34
Frequency and duration of application .....	34
<b>CLEANING AND DISINFECTION AT HOME</b> .....	35
Preparation.....	35
Cleaning .....	35
Disinfection.....	35
Visual inspection .....	37
Drying and storage.....	37
<b>HYGIENIC RE-USE IN THE PROFESSIONAL ENVIRONMENT</b> .....	38
General .....	38
Processing limits .....	39
Preparation.....	39

Cleaning and disinfection.....	40
Sterilisation .....	43
Visual inspection.....	43
Storage .....	43
<b>MISCELLANEOUS.....</b>	<b>44</b>
Disposal .....	44
Contact .....	44
Explanation of symbols.....	44
<b>SPARE PARTS AND ACCESSORIES.....</b>	<b>45</b>

# 1 IMPORTANT INFORMATION

## 1.1 General

Please read these instructions for use carefully and completely. Do not discard, so you can consult them at a later date. Also read the instructions for use of all other products used with this accessory (compressor, nebuliser, etc.) carefully and completely.

If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

## 1.2 Information about the instructions for use

If these instructions for use are lost, you can request another copy from PARI GmbH [see also: 6.2 Contact]. The instructions for use for some products can also be retrieved on the internet in both German and English. Simply visit:  
[www.pari.de/en/products](http://www.pari.de/en/products) (on the respective product page).

## 1.3 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTICE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

## **1.4 Contraindications and side effects**

No contraindications are known.

The following side effects can occur: dizziness, shortness of breath, headaches.

If you notice any of these side effects, discontinue the treatment immediately. Contact your doctor or therapist to discuss how to proceed with treatment.

Inform your doctor or therapist if you notice any side effects that are not listed in these instructions for use.

## **1.5 Instructions for application**

The product contains small parts. Small parts can block the respiratory tract and lead to a choking hazard. Therefore, make sure that you always keep this product out of the reach of babies and infants.

Children and anyone who requires assistance must be supervised constantly by an adult when using the PARI PEP S system. This is the only way to ensure safe and effective treatment.

## **1.6 Hygiene**


Observe the following hygiene instructions:

- Only use a PARI PEP S system that has been cleaned and also disinfected and dried as necessary. Contamination and residual moisture encourage the growth of bacteria, so increasing the risk of infection.
- Each PARI PEP S system must only be used by a single patient at home for hygiene reasons.
- Before every use and cleaning wash your hands thoroughly.
- Make absolutely sure you also carry out cleaning, disinfection and drying before using the device for the first time.
- For cleaning and disinfection always use fresh tap water of drinking water quality.
- Make sure all components are dried properly whenever you have performed cleaning or disinfection.
- Do not keep the PARI PEP S system in a damp environment or together with damp objects.

## 2 PRODUCT DESCRIPTION

### 2.1 Components

Check that all components of your PARI product are contained in your package. If anything is missing, please notify the dealer from whom you purchased the PARI product immediately.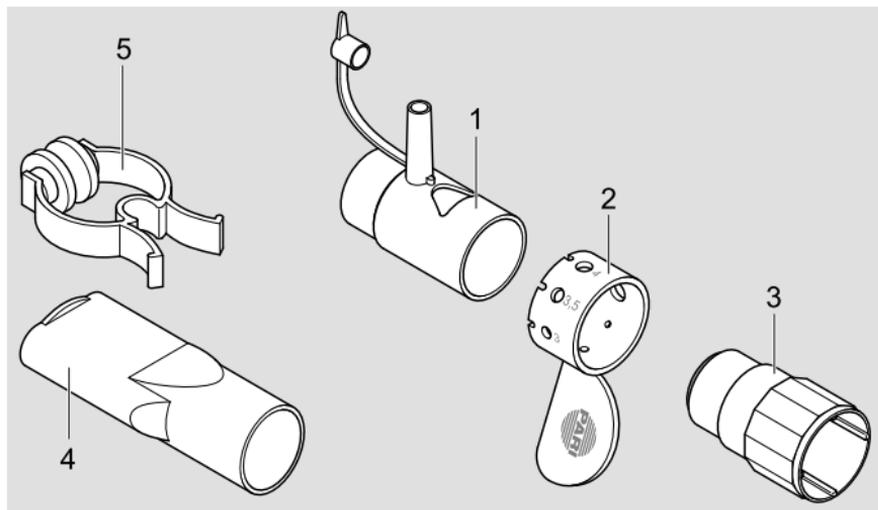

- (1) PEP S connecting element (with connector for an optional pressure gauge)
- (2) PEP S adjusting ring
- (3) PEP S inspiratory valve
- (4) Mouthpiece (without expiratory valve)
- (5) Nose clip

## 2.2 Intended use

The PARI PEP S system is used to mobilise secretion in acute and chronic diseases of the lower respiratory tract. PEP treatment can be carried out either separately or during an inhalation session with a PARI nebuliser.

The PARI PEP S system must not be used without first receiving instruction from a doctor or physiotherapist.

The PARI PEP S can be used:

- by children aged from about 5 to 8 under the supervision and direction of a competent person
- by children aged 8 and older and adults after informed instruction

## 2.3 Material information

Polypropylene	Adjusting ring, connecting element, mouthpiece, inspiratory valve
Silicone	Inspiratory valve
Thermoplastic elastomer	Nose clip
Polyacetal	Nose clip

## 2.4 Product combinations

The PARI PEP S system can be used with:

- PARI LC, PARI LC SPRINT and PARI LL type nebulisers
- an optional pressure gauge

The pressure gauge is used for adjusting the expiratory resistance to your individual needs and for monitoring this resistance during treatment.

The PARI PEP S system **cannot** be used with:

- the PARI LC SPRINT XLent nebuliser (treatment for the upper respiratory tract)
- a PARI baby mask

## 2.5 Description of function

The PARI PEP S system enables effective combination of PEP (**P**ositive **E**xpiratory **P**ressure) and inhalation treatment.

Treatment with the PARI PEP S system helps to increase lung volume and expiratory pressure through the application of adjustable expiratory resistance. The lower respiratory tract is also opened up during its application. This exercises the lungs and mobilises secretion.


### 3 APPLICATION

#### 3.1 Assembling the PARI PEP S system

- Push the adjusting ring onto the connecting element.
- Align the adjusting ring so that the aperture with the diameter recommended by your doctor or therapist is positioned over the aperture of the connecting element.


**Info:** the larger the aperture, the lower the expiratory resistance created.

- To prevent the adjusting ring from slipping out of position, engage locking notch "x" with lug "y".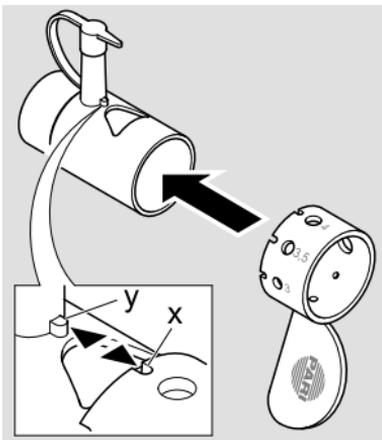


#### Treatment without a nebuliser

- Insert the inspiratory valve in the connecting element.
- Insert the mouthpiece on the other side of the connecting element.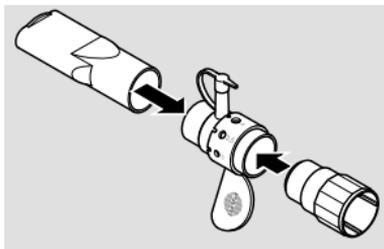


**Info:**

Make sure that the blue valve shim is seated correctly.

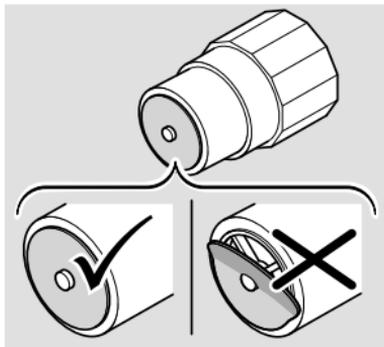**Treatment with a nebuliser**

When treatment is carried out in conjunction with a nebuliser, it is essential to follow the instructions for use of the respective nebuliser as well.

- Assemble the nebuliser you are using (without mouthpiece).
- Attach the connecting element to the nebuliser.
- Attach the mouthpiece without expiratory valve to the connecting element.

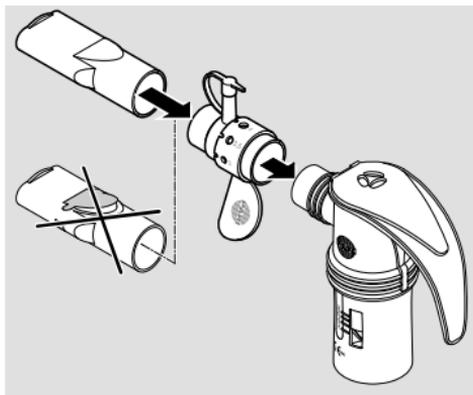- Fill the nebuliser with the medication.

## **3.2 Perform treatment**

- Before starting the treatment, ensure that all parts are firmly connected to each other.
- Close the connecting element securely with the sealing cap.
- Block your nose with the nose clip.
- Breathe in and out again through the mouthpiece slowly and deeply.
- Cough to clear any secretion that is loosened during the treatment.

For reasons of hygiene, avoid coughing into the PARI PEP S system.

## **3.3 Frequency and duration of application**

The frequency and duration of application will be determined by your doctor or therapist depending on your individual requirements.

## 4 CLEANING AND DISINFECTION AT HOME

The PARI PEP S system must be cleaned after each use and disinfected at least once a day.

### 4.1 Preparation

- When it is used with a nebuliser, detach the PARI PEP S system from the nebuliser.
- Dismantle the PARI PEP S system completely into its individual parts.

### 4.2 Cleaning

- Thoroughly clean all components in warm tap water and dish-washing liquid for about 5 minutes. If necessary, use a clean brush to remove loose dirt (the brush must be reserved exclusively for this purpose).
- Rinse all parts thoroughly under warm running water.
- You can remove excess water more quickly by shaking all parts.

### 4.3 Disinfection

Disinfect the components immediately after cleaning (only parts that have been cleaned can be disinfected effectively).

---

#### CAUTION

A damp environment may encourage the growth of bacteria. Therefore, remove all parts from the pot or disinfectant as soon as disinfection has finished. Dry the parts. The **risk of infection** is reduced when the parts are dried completely.

---

## In boiling water

- Place all the individual parts in boiling water for at least 5 minutes.

Use a clean pot and fresh drinking water.

- If your water is very hard, scale may be deposited on the plastic parts as a milky coating. This can be prevented by using softened water.

---

## NOTICE

Plastic will melt if it comes into contact with the hot base of the pot. Therefore, make sure there is plenty of water in the pot. This way you will avoid **damaging the components**.

---

## Using a standard thermal disinfectant for baby bottles (not a microwave oven)

For effective disinfection, use a disinfectant with a runtime of at least 6 minutes. Regarding disinfection, the duration of the disinfection procedure and the quantity of water required for this, follow the instructions for use of the disinfectant you are using.

---

## CAUTION

Inadequate disinfection encourages the growth of bacteria and thus increases the **risk of infection**. Thorough disinfection has not been completed until the disinfectant automatically switches itself off or the minimum disinfection time specified in the instructions for use of the disinfectant has elapsed. Therefore, do not switch the device off prematurely. Also make sure that the disinfectant is kept clean and check that it is working perfectly regularly.

---

## 4.4 Visual inspection

Inspect all product components after each cleaning and disinfection. Replace any broken, misshapen or seriously discoloured parts.

## 4.5 Drying and storage

- You can remove excess water more quickly by shaking all parts.
- Place all parts on a dry, clean and absorbent surface and let them dry completely.
- When drying, between applications, and in particular during long breaks, wrap all the individual parts in a clean, lint-free cloth (e.g., a tea-towel) and keep in a dry, dust-free environment.
- Store the product safely out of continuous direct sunlight.

## 5 HYGIENIC RE-USE IN THE PROFESSIONAL ENVIRONMENT

### 5.1 General

---

#### NOTICE

Ensure that the system is processed for re-use only with the application of adequately validated device- and product-specific procedures, and that the validated parameters are maintained during each cycle. The equipment used must be maintained regularly.

---

Follow the schedule defined below for hygienic re-use:

- Single patient use:  
Clean the PARI PEP S system **after every application**.  
Disinfect the PARI PEP S system at least once a day.
- Multiple patient use:  
Clean, disinfect and sterilise the PARI PEP S system **before every change in patient**.

The system must be processed for re-use with the application of suitable, validated methods in such manner that the success of these methods can be monitored reliably and the safety and health of patients is not threatened.

---

#### CAUTION

Ensure that the components are allowed to dry sufficiently after each processing step. Condensation or residual wetness can present an increased **risk of bacterial growth**.

---

The following text describes the methods validated by PARI, with which plastic PARI parts can be processed effectively.

Ensure that the desired results are achieved consistently by the processing methods employed by your staff with the equipment and chemicals used. Take particular care to ensure that the processing procedure you select is effective if it must differ from our validated procedures (e.g., because the equipment and chemicals described are not available). When selecting the chemicals for use, make sure that they are suitable for cleaning or disinfecting medical products made from the materials indicated. Bear in mind the manufacturer's dosing information and instructions for safe handling of chemicals.

To assist you in processing the system for hygienic re-use, a checklist may be obtained from PARI GmbH (on the internet at [www.pari.de/en/products](http://www.pari.de/en/products) or from the PARI Service Center [see also: 6.2 Contact]).

## 5.2 Processing limits

The PARI PEP S system can be sterilised up to 300 times and used for up to two years.

Do not expose the PARI PEP S system to temperatures above 137 °C.

## 5.3 Preparation

- When it is used with a nebuliser, detach the PARI PEP S system from the nebuliser.
- Dismantle the PARI PEP S system completely into its individual parts.

## 5.4 Cleaning and disinfection

Clean and disinfect immediately after use. Ideally, a mechanical procedure (instrument dishwasher) is to be used.

---

### NOTICE

When selecting the cleaning agent and/or disinfectant, ensure that it is compatible with the materials used, **otherwise damage to the product components** cannot be ruled out.

---

## Mechanical cleaning and disinfection

### EQUIPMENT

- Cleaning and disinfection device in conformance with DIN EN ISO 15883
- Cleaning agent
- Neutralising agent if necessary

### PROCEDURE

- Arrange the components so that they can be cleaned most effectively.
- Select a suitable cleaning programme and agent.

### VALIDATION

The method has been validated using:

- G7836 CD cleaner-disinfector produced by Miele, programme Vario TD.
- Standard baskets manufactured by Miele
- Neodisher Mediclean forte 0.5% and Neodisher Z for neutralisation (from Dr. Weigert)

### DRYING

Even if the cleaner-disinfector is equipped with a drying function, make sure that no residual moisture is left in the components. If necessary, remove any residual moisture by shaking the parts and allow them to dry completely.

## Manual cleaning

As an alternative to mechanical cleaning and disinfection, the following manual method can be used:

### EQUIPMENT

- Cleaning agent
- Brush
- Warm water

### PROCEDURE

- Place all the parts in the cleaning solution.
- Clean the parts thoroughly with a brush.
- Thoroughly rinse the parts under hygienically tested water and leave them to dry completely on a clean, dry and absorbent surface.

### VALIDATION

The method has been validated using:

- Korsolex® Endo Cleaner 0.5% (Bode)
- Approx. 50 °C warm water
- Duration of cleaning procedure: approx. 5 minutes

## Manual disinfection

For manual disinfection, select a suitable instrument disinfectant that has been proven to be effective and compatible with the cleaning agent used.

### EQUIPMENT

- Disinfectant
- Water

---

### CAUTION

PARI product components are not suitable for treatment with instrument disinfectants based on **quaternary ammonium compounds**. Quaternary ammonium compounds tend to become concentrated in plastic materials, which can cause **incompatibility reactions** in patients.

---

### PROCEDURE

- Place all the parts in the disinfection solution.
- Allow the solution to take effect as indicated in the instructions for use of the disinfectant.
- Thoroughly rinse the parts under hygienically tested water and leave them to dry completely on a clean, dry and absorbent surface.

### VALIDATION

The method has been validated using:

- Korsolex® Basic 4% (Bode)
- Treatment time: 15 minutes

## 5.5 Sterilisation

---

### NOTICE

A validated sterilisation process can only be carried out on products that have been cleaned and disinfected.

---

### EQUIPMENT

Steam steriliser – preferably with fractionated pre-vacuum – in accordance with DIN EN 285 or DIN EN 13060 (type B)

### PROCEDURE

- Pack all of the disassembled parts in a sterile barrier system in accordance with DIN EN 11607 (e.g., foil-paper packaging).

Maximum sterilisation temperature and holding time:

121 °C, at least 20 min. or

132 °C / 134 °C, at least 3 min.

### VALIDATION

The method has been validated according to DIN EN ISO 17665-1 using:

- Euro-Selectomat 666 from MMM (fractionated pre-vacuum method)
- Varioklav 400 E from H+P Labortechnik (gravitation method)

## 5.6 Visual inspection

Inspect all product components after each cleaning, disinfection and sterilisation. Replace any broken, misshapen or seriously discoloured parts.

## 5.7 Storage

Store the PARI PEP S system in a dry, dust-free place away from sources of contamination. It is recommended to use sterile packaging.

## 6 MISCELLANEOUS

### 6.1 Disposal

All product components can be disposed of with domestic waste unless this is prohibited by the disposal regulations prevailing in the respective member countries.

### 6.2 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)

+49 (0)8151-279 220 (international)

### 6.3 Explanation of symbols

The following symbols are displayed on the PARI PEP S system packaging:

	Manufacturer
	Please follow the instructions for use
	The product satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.
	Order no.
	Lot no.

## 7 SPARE PARTS AND ACCESSORIES

Description	Item no.
PEP S inspiratory valve	018B4020
Mouthpiece (without expiratory valve)	012E1720
Nose clip	041E3500
Pressure gauge (0 – 100 mbar); incl. PEP tubing adapter, tubing adapter and pressure tubing	018B1600

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