RESMED

Quattro™ FX NV

NON-VENTED FULL FACE MASK
FULL FACE MASKE OHNE AUSATEMÖFFNUNGEN
MASQUE FACIAL NON VENTILÉ (SANS FUITE INTENTIONNELLE)
MASCHERA ORO-NASALE NON-VENTED
MASCARILLA FACIAL SIN VENTILACIÓN
MÁSCARA FACIAL NÃO VENTILADA
NIET-GEVENTILEERD VOLGELAATSMASKER
OVENTILERAD HELMASK
IKKE-VENTILERET HEL ANSIGTSMASKE
IKKE-VENTILERT HELMASKE
ILMA-AUKOTON KOKOKASVOMASKI
ΜΑΣΚΑ ΓΙΑ ΟΛΟΚΛΗΡΟ ΤΟ ΠΡΟΣΩΠΟ ΧΩΡΙΣ ΑΝΟΙΓΜΑΤΑ ΕΞΑΕΡΙΣΜΟΥ
MASKA TWARZOWA PEŁNA BEZ WYLOTÓW WYDYCHANEGO POWIETRZA
NEM SZELLŐZŐ TELJES ARCMASZK
НЕВЕНТИЛИРУЕМАЯ ПОЛНОЛИЦЕВАЯ ΜΑΣΚΑ
CELOTVÁROVÁ MASKA BEZ VETRACÍCH OTVOROV
VENTÍLSÍZ TAM YÜZ MASKESI
NEVENTILOVANÁ CELOOBLIČEJOVÁ MASKA

User Guide

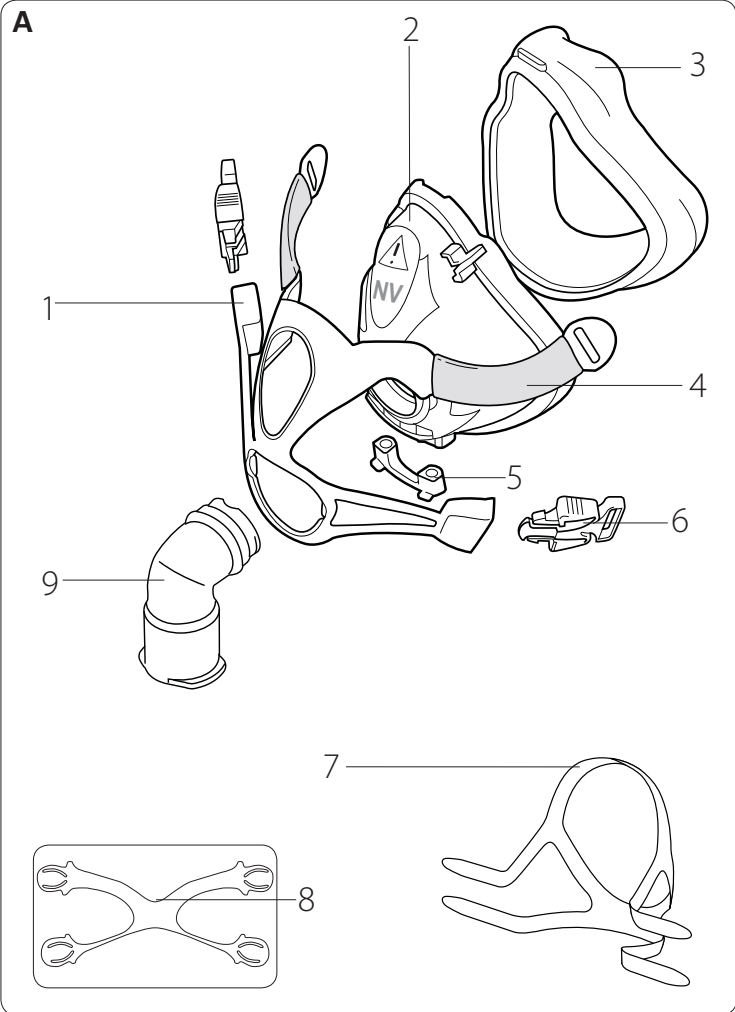
English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • Ελληνικά • Polski • Magyar • Русский • Slovenčina • Türkçe • Český

Contents

Quattro FX NV User Guide 1







Quattro™ FX NV

NON-VENTED FULL FACE MASK




Quattro FX NV

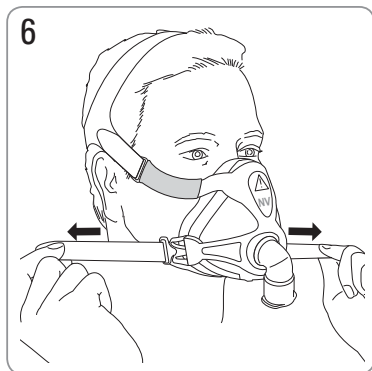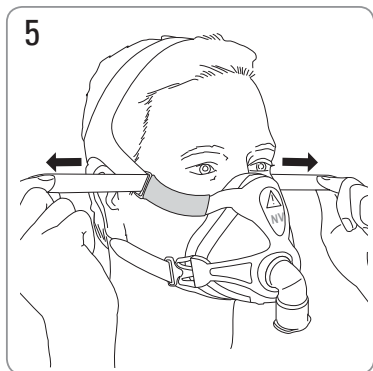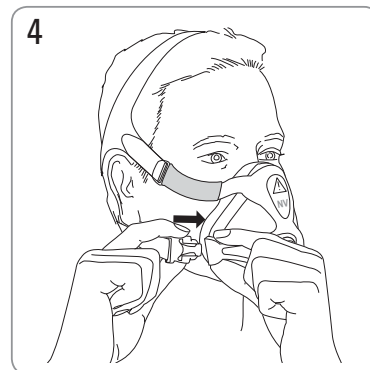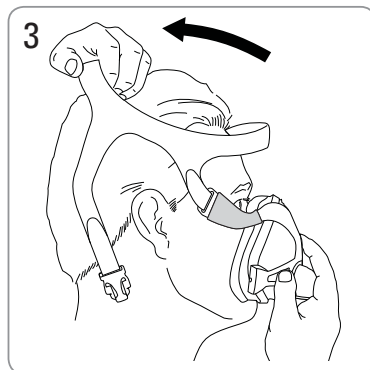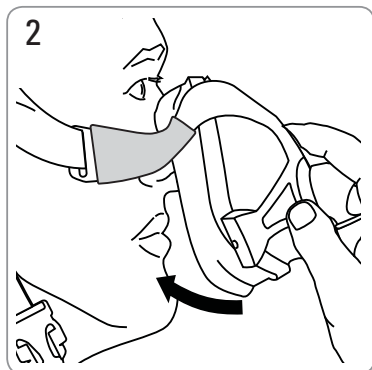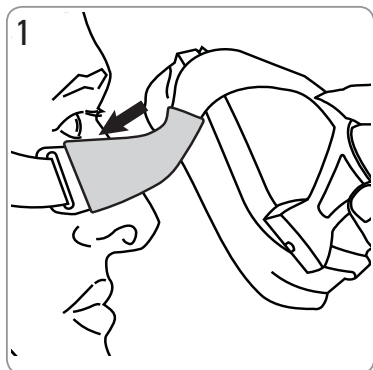Fitting / Anlegen / Mise en place / Applicazione / Colocación / Colocação / Opzetten / Tillpassning / Tilpassning / Tilpassning / Sovittaminen / Τοποθέτηση της μάσκας / Zakładanie maski / A maszk felhelyezése / Подгонка маски / Nasadzovanie / Maskenin Takılması / Nasazení masky




Quattro FX NV

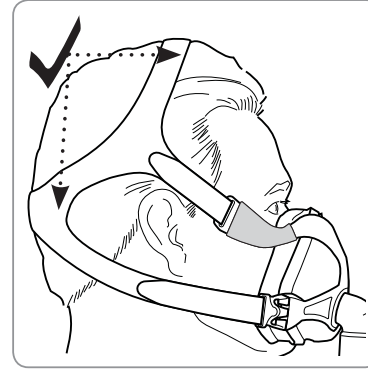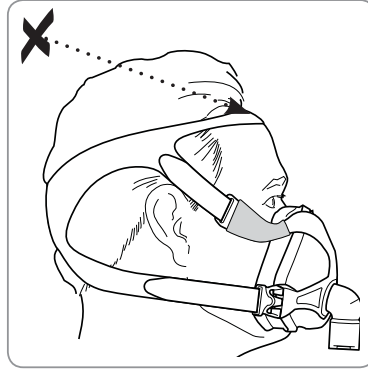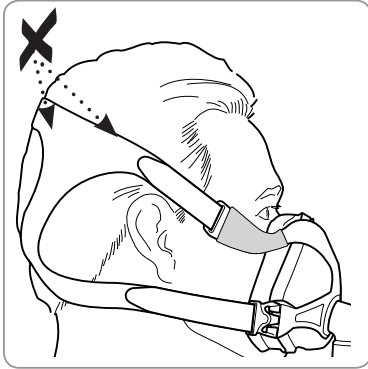Headgear check / Kontrolle des Kopfbandes / Vérification du harnais / Verifica del copricapo / Verificación del arnés / Verificação do arnês / Hoofdband controleren / Huvudbandskontroll / Kontrol af hovedbånd / Kontroll av hodestropper / Pääremmien oikea paikka / Έλεγχος μάντων κεφαλής / Kontrola części nagłownej / A fejránt ellenőrzése / Проверка головного фиксатора / Správne nasadenie náhlavnej súpravy / Başlık kontrolü / Kontrola náhlavní soupravy





Quattro FX NV

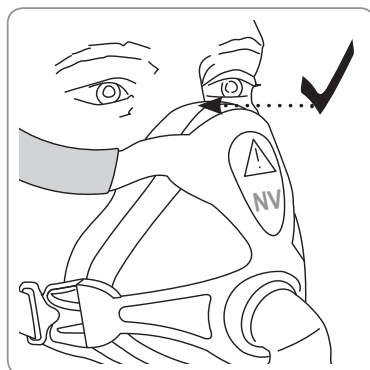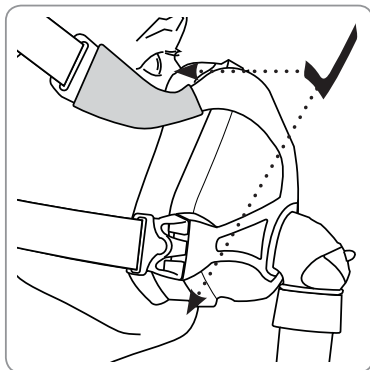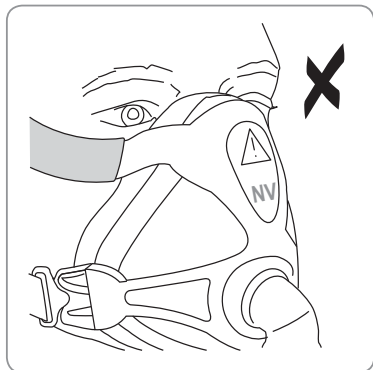Cushion check / Überprüfen des Maskenkissens / Vérification de la bulle / Verifica del cuscinetto / Verificación de la almohadilla / Verificação da almofada / Kussentje controleren / Mjuki delskontroll / Pudekontrol / Kontroll av pute / Pehmikkeen oikea paikka / Έλεγχος μαξιλαριού / Kontrola uszczelki / A párna ellenőrzése / Проверка уплотнителя / Skontrolovanie podušky / Yastık kontrolü / Kontrola polštářku



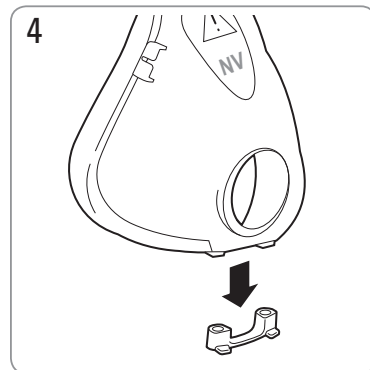
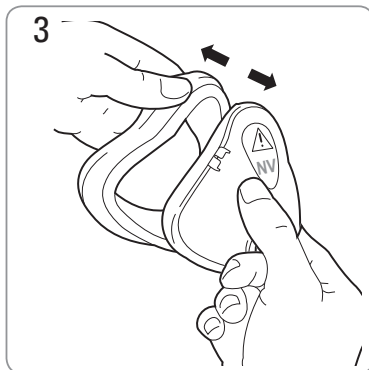
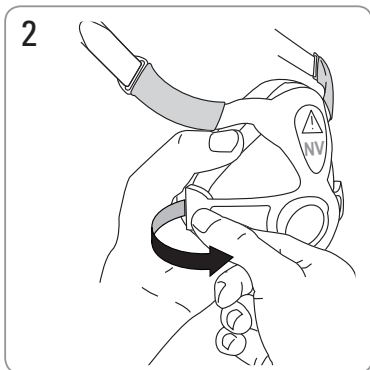
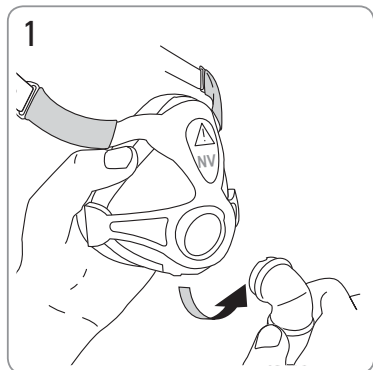
Disassembly / Auseinanderbau / Démontage / Smontaggio / Desmontaje / Desmontagem / Demonteren / Demontering / Adskillelse / Demontering / Purkaminen / Αποσυναρμολόγηση / Demontaž / Szétszerelés / Разборка / Rozobratie / Sökme / Demontáž masky

Quattro FX NV

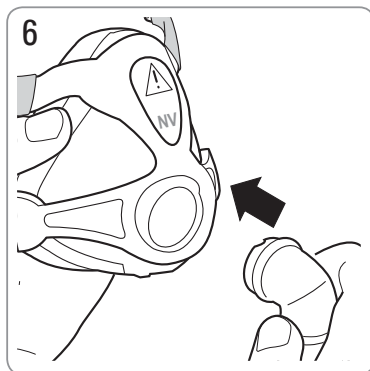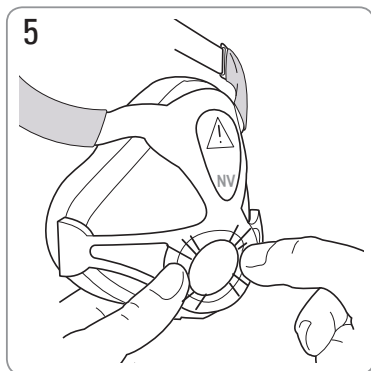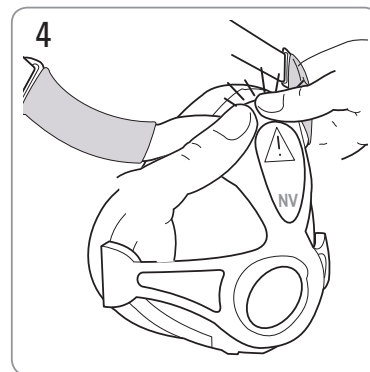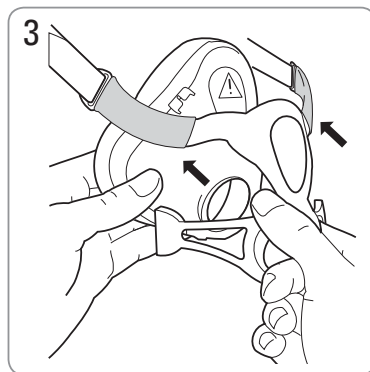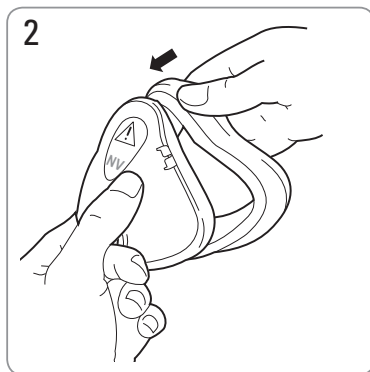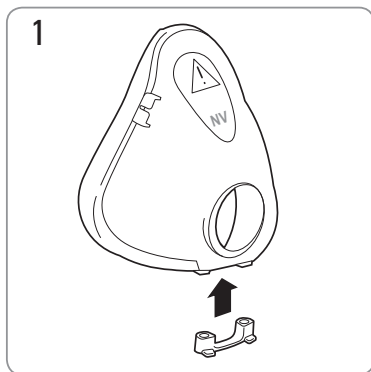Reassembly / Zusammenbau / Remontage / Riassemblaggio / Montaje / Nova montagem / Opnieuw monteren / Montering / Samling / Montering / Kokoaminen / Επανασυναρμολόγηση / Ponowny montaż / Ismételt összeszerelés / Сборка / Opätovné zostavenie / Yeniden montaj / Opätovné sestavení masky

English

Intended use

The mask directs airflow noninvasively to a user with or without supplemental oxygen.

The mask is intended to be used with active-exhaust-valve ventilator systems, to provide ventilatory assistance to patients with respiratory insufficiency and respiratory failure.

The mask is intended for:

- adult users (>30 kg) requiring non-life-support ventilatory assistance.
- single user re-use in the home environment and multipatient re-use in the hospital/institutional environment.

Using the mask

- ResMed has tinted the elbow blue to identify it as a non-vented mask.
- The mask has a standard female 22mm tapered fitting for external connection to the ventilator.
- Follow the instructions provided by your physician or sleep therapist.
- Prior to fitting your mask, remove the spring frame support from the rear of your mask and retain for future use.
- To fit your mask, follow the instructions in the Fitting section.
- Do not overtighten the headgear straps. If you feel air-leaks you may need to:
 - refit or reposition your mask
 - adjust your headgear
 - check that your mask is assembled correctly
 - contact your physician or sleep therapist to have your mask refitted.
- Mask-specific user guides are available on www.resmed.com on the **Products** page under **Service & Support**. They are also available from ResMed upon request.

Disassembly

To disassemble your mask, follow the instructions in the Disassembly section.

Reassembly

To reassemble your mask, follow the instructions in the Reassembly section.

Cleaning the mask at home

- The mask and headgear can only be handwashed. The headgear may be washed without removing the headgear clips.
- Take care when cleaning to avoid damage or loss of components.

Daily/After each use

- To optimise mask seal, remove facial oils from the cushion before use.
- Do not soak the mask.
- Handwash the mask by rubbing gently in warm (30°C) water using mild soap.
- Rinse all components well with drinking quality water and allow them to air dry out of direct sunlight.

Weekly

Handwash the headgear in warm (30°C) water using mild soap or diluted dishwashing detergent. Rinse well and allow it to air dry out of direct sunlight.

Reprocessing the mask between patients

The mask should be reprocessed when used between patients. Cleaning, disinfection and sterilisation instructions are available from the ResMed website, www.resmed.com/masks/sterilization. If you do not have internet access, please contact your ResMed representative.

Technical Specifications

Deadspace: Empty volume of mask to the elbow. The deadspace of the mask varies according to cushion size.

Quattro FX NV

243mL (Large)

Therapy pressure (cm H₂O)

Quattro FX NV

4 – 30

Gross Dimensions – Mask fully assembled – no headgear. Largest variant only shown.

Quattro FX NV

Height 140 mm

Width 176 mm

Depth 144 mm

Environmental conditions – all masks:

Operating temperature: +5°C to +40°C

Operating humidity: 15% to 95% non-condensing

Storage and transport temperature: -20°C to +60°C

Storage and transport humidity: up to 95% non-condensing

Notes:

- The mask system does not contain latex, PVC or DEHP materials
- The manufacturer reserves the right to change these specifications without notice.

Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

To maintain optimum care of your mask, ensure the spring frame support is fitted when travelling.

Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

Symbols

The following symbols may appear on the mask or packaging.

 Caution, consult accompanying documents;  Lot number;  Part number;  Temperature limitation;  Humidity limitation;  Does not contain latex;  Manufacturer;  Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.  Keep away from rain;  This way up;  Fragile, handle with care.

Warnings

- The mask must be used under qualified supervision for users who are unable to remove the mask by themselves.
- The mask is not for use on patients with impaired laryngeal reflexes or other conditions predisposing to aspiration in the even of regurgitation or vomiting.
- The mask should not be worn unless the ventilator system is turned on and operating properly.
- At a fixed flow rate of supplemental oxygen flow (if used), the inhaled oxygen concentration varies depending on the pressure settings, user breathing pattern, mask size selection and leak rate.
- If oxygen is used with the ventilator, the oxygen flow must be turned off when the ventilator is not operating so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Discontinue using the mask and consult your physician or sleep therapist if you have ANY adverse reaction.
- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.
- Do not use aromatic-based solutions or scented oils (eg, eucalyptus or essential oils), bleach, alcohol or products that smell strongly (eg, citrus) to clean any of the mask components. Residual vapours from these solutions can be inhaled and may also damage the mask, causing cracks.

- Do not combine components from different mask systems.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.
- Remove all packaging before using the mask.
- Ensure the soft sleeves are fitted at all times during use.

Consumer Warranty

ResMed acknowledges all consumer rights granted under the EU Directive 1999/44/EC and the respective national laws within the EU for products sold within the European Union.

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