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GB sanabelle cleaning brush straight

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GB sanabelle cleaning brushes straight

sanabelle cleaning brushes straight are specially designed for cleaning flexible tracheostomy tubes.

Because of the various tube dimensions, they are available in three sizes that differ in bristle diameter and wire strength. The wire used has no sharp edges and is made of stainless steel. The bristles are made of hygienic nylon. Insert the brush into the tube until all the bristles protrude from the other end. Clean the bristles under running water before pulling them back through the tube. This is a clear advantage to conventional short brushes. To be able to push the brush past the curve stretch flexible tubes during cleaning. In the case of rigid tubes, prebend the brush in accordance with the shape of the tube. After use, clean the brush thoroughly and hang it up to dry.

Caution: Always remove the tube from the tracheostoma before cleaning. Never use the brush in the tracheostoma. It is best to insert the brush from the tip of the tube. Ensure the tube is intact before reinsertion. For reasons of hygiene, we recommend that you change the brush once a week. For hygienic reasons, we recommend to use one separate brush for every patient and change it once a week.

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