Respiro Soft Baby - non vented ENGLISH

NASAL MASK 3-12 kg

Instructions for use

1 - APPLICATION

Respiro Soft Baby non vented is a non invasive nasal mask used for treating respiratory failure in babies weighing between 3 and 12 kg.
Respiro Soft Baby non vented is designed for use by babies with controlled non-invasive positive pressure ventilation (NPPV).
Respiro Soft Baby non vented is designed for use by a single patient at home or more than one in the hospital environment.

2 - IMPORTANT NOTES

- The mask must be used under the supervision of an adult who has read and understands the instructions.
- The mask must only be used with a ventilator equipped with an active exhalation valve or a respiratory circuit with an exhalation valve.
- The operating pressure range is 4-24 cmH2O.
- If you experience discomfort or irritation, or suffer an allergic reaction to any of the components of the mask, please report this to your doctor or respiratory therapist.
- If oxygen is used with the device, the oxygen supply must be turned off when it is not in use to prevent oxygen accumulating inside, which is a fire risk.
- Never smoke or light naked flames when using oxygen.
- Do not use the mask in the presence of nausea or vomiting.
- The mask must be cleaned and disinfected. For further details refer to section CLEANING THE MASK.
- Check the mask for damage before use. If there is any visible damage, report it to the dealer.
- The mask must only be used in accordance with these instructions only. No other instructions are applicable.
- Some of the components of the mask are small enough to be swallowed by children and must never be left unattended. An adult must always be present.

3 - COMPONENTS OF THE MASK (Fig.A)

- (a) Main body: frame (1), cushion (2), forehead rest (3);
- (b) Tube assembly: elbow (4), tube (5), swivel fitting (6), cap (7);
- (c) Headgear.

4 - HOW TO ASSEMBLE THE MASK (Fig. B-C)

Connect the tube assembly (b) by inserting the sphere on the elbow fitting (4) in the slot in the main body (a), as shown in Fig. B1. The instructions on how to use the mask depend on the chosen configuration (WITH or WITHOUT the forehead rest). Refer to the relevant subsection:

4.1 - Using the mask WITH the forehead rest (3)

4.2 - Using the mask WITHOUT the forehead rest (3)

4.1.1 USING THE MASK WITH THE FOREHEAD REST (FIG. B2)

- Insert the three straps at the top of the headgear (c) in the three slots in the forehead rest (3), as shown in Fig. B2. Then fasten the straps.
- Move on to Section HOW TO PUT THE MASK ON THE CHILD.

4.2 USING THE MASK WITHOUT THE FOREHEAD REST (FIG. C) (OPTION)

- To remove the forehead rest (3), merely cut it off the main body (a) using a pair of scissors, as shown in Fig. C1.
- Remove the forehead rest (3), merely cut it off the main body (a) using a pair of scissors, as shown in Fig. C2.

5 - HOW TO PUT THE MASK ON THE CHILD (FIG.D)

- Place the mask gently on the child's face, making sure the cushion (2) fits snugly round the nose. Pull the headgear over the child's head, as shown in Fig. D1.
- Insert the loose lower straps of the headgear (c) in the side slots in the main body (a), as shown in Fig. D2. Then tighten the straps.
- Position the headgear (c) comfortably on the child's head and adjust all the straps, starting from the lower ones, as shown in Fig. D3.
- Connect the mask to the ventilator circuit (Fig. D4) Then turn on the flow generator.
- Readjust the lower straps if necessary to eliminate any leaks, but make sure the mask is not too tight.

N.B. Do not throw away the forehead rest (3). It will be required later. Once the forehead rest (3) has been cut off from the main body (a), it cannot be reattached.

4.2.2 Take the cut-off forehead rest (3) and insert the central strap at the top of the headgear (c) in the top slot, as shown in Fig. C2.

4.2.3 Insert the side straps at the top of the headgear (c) in the side slots, as shown in Fig. C3.

4.2.4 Insert the central strap at the top of the headgear (c) in the slot at the top of the main body (a), as shown in Fig. C4. Then fasten the strap.

5 - HOW TO PUT THE MASK ON THE CHILD (FIG.D)

- Place the mask gently on the child's face, making sure the cushion (2) fits snugly round the nose. Pull the headgear over the child's head, as shown in Fig. D1.
- Insert the loose lower straps of the headgear (c) in the side slots in the main body (a), as shown in Fig. D2. Then tighten the straps.
- Position the headgear (c) comfortably on the child's head and adjust all the straps, starting from the lower ones, as shown in Fig. D3.
- Connect the mask to the ventilator circuit (Fig. D4) Then turn on the flow generator.
- Readjust the lower straps if necessary to eliminate any leaks, but make sure the mask is not too tight.

N.B. The lower straps only serve to hold the mask in position, so they must be kept fairly loose.

6 - HOW TO REMOVE THE MASK

- Unfasten one of the lower straps on the headgear (c). Take off the mask off the face and lift the headgear off. It is not necessary to unfasten the other straps.

7 - HOW TO DISASSEMBLE THE MASK (FIG. E)

- Unfasten and pull out the straps, as shown in Fig. E1, then detach the headgear (c) from the main body (a).
- Detach the tube assembly (b) by pulling the elbow fitting (4) out of the main body (a), as shown in Fig. E2.

8 - HOW TO CLEAN THE MASK

8.1 - DAILY CLEANING FOR REUSE BY THE SAME PATIENT

First wash your hands thoroughly.
Disassemble all the components (see HOW TO DISASSEMBLE THE MASK) and wash carefully in warm water at about 30°C, using mild soap. Rinse thoroughly under the tap and leave in the air to dry, but not in direct sunlight.

N.B. The headgear (c) should be cleaned once a week.

8.2 - REPROCESSING THE MASK FOR RE-USE WITH OTHER PATIENTS

N.B. The mask must always be regenerated before being used by another patient.
N.B. The headgear (c) must be replaced each time.
N.B. In case of thermal disinfection and autoclave sterilization, the tube assembly (b) must be replaced.

The mask can be regenerated up to 12 times, using one of the following methods:

- high-level thermal disinfection;
- high-level chemical disinfection;
- sterilisation in an autoclave at 121°C for no more than 20 minutes;
- Sterrad gas-plasma sterilisation.

Full details of the mask regeneration process can be obtained by logging on to www.airliquidemedicalsystems.com or by contacting your dealer.

9 - PRECAUTIONS

Do not clean the mask with solutions containing alcohol, aromatic compounds, humidifiers, antibacterial agents or aromatic oils as they would damage the product and reduce its life.

If any damage is noticed (cracks, tears, etc.), the damaged part must be replaced.

Check carefully to make sure there are no soap deposits or foreign bodies inside the mask.

Fit on the cap (g).

Pass the two lower straps of the headgear (l) through the slots in the clips (m).

Attach the two upper straps of the headgear (l) to the main body (b).

Clip the two clips (m) back onto the main body (b).

Turn the adjusting knob (e) to the reference position you noted previously.

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