
S9™ Oximeter Adapter

DATA TRANSFER ACCESSORY


User Guide

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • 简体中文 • 繁體中文

A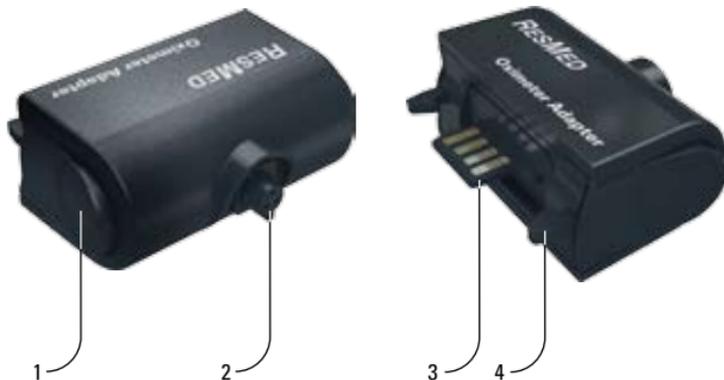

B

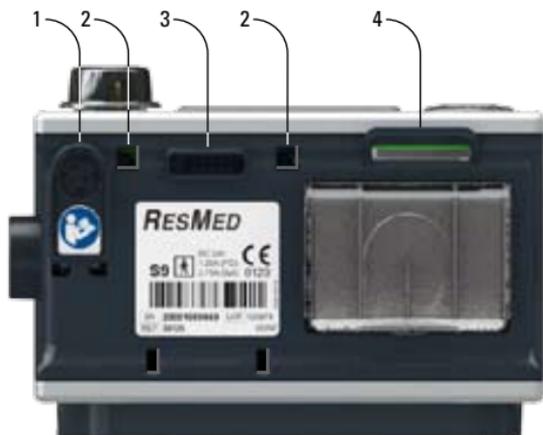Thank you for choosing the ResMed S9 Oximeter adapter. The S9 Oximeter adapter connects a NONIN Xpod™ oximeter to your S9 device. Oximetry data (oxygen saturation and pulse rate) is recorded on the SD card inserted into the S9 device.

Please read the entire guide before using this device.

S9 Oximeter adapter

Refer to illustration A.

1. Release button
2. Oximeter port
3. Adapter connector
4. Adapter clip

Rear panel of S9 device

Refer to illustration B.

1. DC power inlet
2. Adapter clip socket
3. Module/Adapter port
4. SD card slot

Associated accessories

- Xpod oximeter
- Xpod fixation clip
- Finger pulse sensor
- S9 Oximetry accessories pack (including Xpod oximeter, Xpod fixation clip and 3m oximeter flex sensor with 25 sensor wraps).

Setup

Before setting up, disconnect the S9 device from the power outlet.

Refer to illustrations C and D.

1. Ensure an SD card is inserted into the SD card slot on the S9 device.
2. Align the S9 Oximeter adapter clips with their corresponding sockets and push the adapter onto the S9 device. Ensure the adapter clips click into place.
3. Connect the Xpod oximeter to the oximeter port on the S9 Oximeter adapter.

4. Connect the the Xpod oximeter to the finger pulse sensor.
5. Reconnect the S9 device to the power outlet. The S9 device briefly displays Oximeter enabled.

You are now ready to start therapy. For further information, please refer to your S9 Welcome Guide.

Removing and returning the S9 Oximeter adapter and SD card

1. Disconnect the S9 device from the power outlet.
2. Press and hold in the release buttons located on each side of the S9 Oximeter adapter while removing it from the S9 device.
3. Remove the Xpod oximeter from the oximeter port on your S9 Oximeter adapter.
4. Remove the finger sensor cable from the Xpod oximeter.
5. Remove the SD card from the S9 device. For instructions on removing the SD card, please refer to the S9 SD card protective folder provided with the S9 device.
6. When instructed to do so, return this User Guide, the S9 Oximeter adapter and the SD card to your healthcare professional in the original packaging.

Note: Ensure that you remove the S9 Oximeter adapter from the S9 device before removing the Xpod oximeter. Failure to do so may damage the S9 Oximeter adapter.

Cleaning and maintenance

If necessary, wipe the outside of the S9 Oximeter adapter with a clean, dry cloth. For instructions on how to clean the oximeter and its sensor, please refer to the user guide supplied with these accessories.

Save the packaging

Please keep this user guide and all the original packaging for the S9 Oximeter adapter.

Troubleshooting

If there is a problem, try the following suggestions. If the problem cannot be solved, contact your healthcare professional.

Problem/possible cause	Solution
S9 device displays: Insert SD Card or press dial to accept no oximetry data logging	
The S9 device cannot detect an SD card and no oximetry data can be logged or stored.	Ensure that an SD card is correctly inserted in the SD card slot of your S9 device.
S9 device displays: Check oxi sensor or press dial to accept no oximetry data logging	
The finger pulse sensor is not attached to the finger or the sensor cable is not connected to the Xpod oximeter at the commencement of therapy.	Disconnect and reconnect the Xpod oximeter connector and sensor cable according to the instructions supplied with these accessories.
S9 device displays: Connect oxi sensor or press dial to accept no oximetry data logging	
The S9 Oximeter adapter is not connected properly or data is not transmitting correctly.	Remove your S9 Oximeter adapter from the S9 device according to the instructions. Reattach your S9 Oximeter adapter and reconnect your S9 device to the power outlet.

Technical specifications

Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity	10 to 95% non-condensing
Storage and transport temperature	-4°F to +140°F (-20°C to +60°C)
Storage and transport humidity	10 to 95% non-condensing
Gross dimensions (L x W x H)	1.74" x 2.12" x 0.91" (44.35 mm x 54.10 mm x 23.30 mm)
Weight	0.77 oz (22.1 g)
Electromagnetic compatibility	Please refer to the S9 Information Guide for details

Note: The manufacturer reserves the right to change these specifications without notice.

Symbols

The following symbols may appear on the S9 Oximeter adapter or packaging:

 Caution;  Read instructions before use;  Drip proof;  Manufacturer;
 LOT Lot number;  RoHS European RoHS;  REF Part number;  SN Serial number;
 Canadian Standards Association;  UL Listed.

Environmental information

WEEE 2002/96/EC is a European Directive that requires the proper disposal of electrical and electronic equipment. These devices should be disposed of separately, not as unsorted municipal waste. To dispose of the device, you should use appropriate collection, reuse and recycling systems available in your region.

The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device, please contact your ResMed office, local distributor or go to www.resmed.com/environment.

WARNING

- The S9 device should be disconnected from the power outlet when attaching or detaching the S9 Oximeter adapter.
- Beware of electrocution. Do not immerse the device or power cord in water.
- Always disconnect the device from the power supply before cleaning and make sure it is dry before plugging it back in.

CAUTION

- In the US, Federal law restricts this device to sale by or on the order of a physician.
- Only use the S9 Oximeter adapter as specified in this guide.
- Avoid exposing your S9 Oximeter adapter to moisture or extreme temperatures.
- Do not use liquids to clean any part of the S9 Oximeter adapter.
- Do not open your S9 Oximeter adapter. There are no user serviceable parts inside. Repairs and internal servicing should only be performed by an authorized service agent.

Limited warranty

Please refer to the ResMed S9 Information Guide.

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© 2010 ResMed Ltd. 368214/1 2010-06
S9 is a trademark of ResMed Ltd and is registered in U.S. Patent and Trademark Office.
NONIN and Xpod are trademarks of Nonin Medical Inc.
Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.