EN Heated disposable breathing tube system with W-adapter

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V 890.100.2

Application

This product is to be used for respiration in connection with the humidifier PMH5000 by WILAmEd.

Instruction for use

(1) Remove closing cap from Y-piece or exhalation valve (if existent). (2) Connect ventilation device and input of humidifier chamber with a short tube. **Note:** Connecting respectively loosening of the tube is accomplished by turning, pressing and/or pulling at the cuffs – and not at the tube itself! (3) Connect inspiration tube to output of the humidifier chamber. **Note:** Inspiratory side of a dual tube system is the side which has an opening for insertion the temperature sensor close to patient (e.g. at Y-piece etc.). (4) Connect corollary lines (if existent) (e.g. pressure line etc.) to the ventilator device. (5) Please connect the heating wire of the inspiratory tube with the dispo adapter to outlet INSP at the humidifier. **Note:** The dispo adapter is an electrical adapter for disposable heating wires. Don't apply any force! Contacts are not arranged centered; the wire plug fits in one position only! (6) Using a breathing tube system with two tubes, please, connect expiratory tube to the ventilator device. (7) Please connect the heating wire of the expiratory tube (if existent) with the dispo adapter to outlet EXP at the humidifier. (8) Arrange any water trap (if existent) in the way that the reservoir faces downwards. (9) Insert both sensors of temperature probe into the provided ports at the connectors by turning slightly; the sensor with two wire outputs is plugged in upstream of the humidifier chamber, the sensor at the end of temperature probe is plugged near the patient. **Note:** Don't apply any force! (10) Arrange temperature sensor near the patient in the way that sensor head is not pointed towards the patient's chest. Thereby temperature measurement is unadulterated and water drops can drain off easier. (11) Before starting operation, check system for blockages, leakages and malfunctions. (12) At first please start ventilator device, then the humidifier. **Note:** Slight condensation inside tube and humidifier chamber during operation is quite normal. (13) After starting operation, check if heating wires are getting warm. (14) Check any water trap/s (if existent) in a regular basis, empty out water if necessary. Then check, if water trap is closed correctly, again.

Warnings

(1) Store breathing tube system in a dry cool place, do not expose it to direct sunlight, and do not keep or operate it near heat sources. (2) Please, make sure that this breathing tube system is appropriate for the ventilation device provided before use. (3) Please, observe operating instructions for humidifier PMH5000 and its accessories. (4) Breathing tube systems must be connected by respectively instructed persons only. (5) This breathing tube system is not suitable for the administration of inflammable gases. (6) Danger of burns: a) Do not cover breathing tubes with insulating materials (e.g. cloths, blankets etc.)! b) Breathing tubes must never touch the patient. c) To avoid overheating, a minimum continuous flow of respiratory gas inside the breathing tube is necessary. d) The Y-piece or expiration valve (if existing) must neither be closed nor blocked. (7) After use, please dispose according to regulations. (8) This product is destined for one single patient and is not intended for reprocessing with a term of usage of max. 7 days (do not utilize again after use, do not clean or sterilize it).

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