EN

Heated disposable breathing tube system with A-adapter

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Stand: 2016-06-01
P/N 890.100.7

Application

This product is to be used for ventilation and respiratory therapy in connection with the humidifier by WILAméd, in particular AIRcon.

Instruction for use

(1) Remove closing cap from Y-piece or exhalation valve (if existent). (2) Connect ventilation device and input of humidifier chamber with a short tube. **Note:** Connecting respectively loosening of the tube is accomplished by turning, pressing and/or pulling at the cuffs – and not at the tube itself! (3) Connect inspiration tube to output of the humidifier chamber. **Note:** Inspiratory side of a dual tube system is the side which has an opening for insertion the temperature sensor close to patient (e.g. at Y-piece etc.). (4) Connect corollary lines (if existent) (e.g. pressure line etc.) to the ventilator device. (5) When connecting the electrical connection(s) of the heating wire distributor cable to the breathing tube or breathing tubes (such as in a double tube system with expiratory heating), please make sure that the colour coding is: green to green, white to white. (6) Arrange any water trap (if existent) in the way that the reservoir faces downwards. (7) Insert both sensors of temperature probe into the provided ports at the connectors by turning slightly: the sensor with two wire outputs is plugged in upstream of the humidifier chamber, the sensor at the end of temperature probe is plugged near the patient. **Note:** Don't apply any force! (8) Arrange the temperature sensor nearest the patient so that drops of water from the sensor head can easily run off. (9) Before starting operation, check system for blockages, leakages and malfunctions. (10) At first please start ventilator device, then the humidifier. **Note:** Slight condensation inside tube and humidifier chamber during operation is quite normal. (11) After starting operation, check if heating wires are getting warm. (12) Check any water trap/s (if existent) on a regular basis, empty out water if necessary. Then check that the water trap is correctly closed again.

Warnings

(1) Store breathing tube system in a dry cool place, do not expose it to direct sunlight, and do not keep or operate it near heat sources. (2) Please, make sure that this breathing tube system is appropriate for the ventilation device provided before use. (3) Please, observe operating instructions for humidifier and its accessories. (4) Breathing tube systems must be connected by respectively instructed persons only. (5) This breathing tube system is not suitable for the administration of inflammable gases. (6) Danger of burns: a) Do not cover heated breathing tubes with insulating materials (e.g. cloths, blankets etc.)! b) Heated breathing tubes must never touch the patient. c) To avoid overheating, a minimum continuous flow of respiratory gas inside the breathing tube is necessary. d) The Y-piece or expiration valve (if existing) must neither be closed nor blocked. (7) After use, dispose in accordance to regulations for infectious waste. (8) This product is destined for one single patient and is not intended for reprocessing with a term of usage of max. 7 days (do not utilize again after use, do not clean or sterilize it).

Key to Symbols Zeichenerklärung Signification des symboles Spiegazione dei simboli Significado de los símbolos

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