
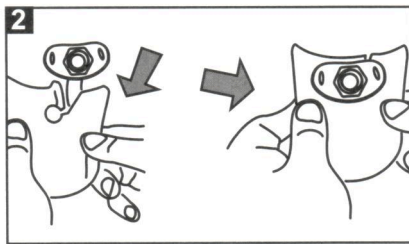
# SENSOTRACH® UNO PED SLIT

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## INSTRUCTIONS FOR USE

### BILDER / PICTURES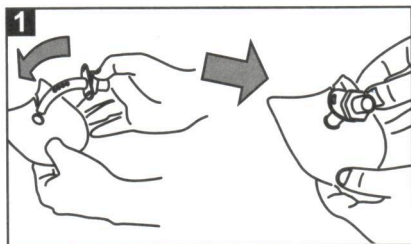

### PIKTOGRAMM-LEGENDE / PICTROGRAM LEGEND

Bestellnummer /  
Catalogue number

Gebrauchsanweisung beachten /  
Consult instructions for use

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## I. FOREWORD

These instructions for use are valid for tracheal compresses. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

## II. INTENDED USE

Tracheal compresses serve to absorb tracheal secretions and to cushion the tracheal neck flange against the skin.

## III. WARNINGS

Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

## IV. PRODUCT DESCRIPTION

Different versions of tracheal compresses are available. The compresses have a circular hole through which the cannula tube can be passed and are also available with an additional slit opening, for fast compress changing.

## V. INSTRUCTIONS

Before replacing the compress, ensure that you have all required material available. The tracheal cannula should be cleaned and if required, disinfected before application. The tracheal cannula is inserted through the existing, circular opening of the compress ❶. An additional slit next to the circular opening allows fast compress replacement without prior removal of the tracheal cannula from the tracheostoma ❷.

## CAUTION!

When inserting the tracheal cannula, please observe the application instructions of its manufacturer.

## VI. SERVICE LIFE

The compress is unsterile and intended for single use only. For hygienic reasons, the compress must be replaced at least every 12 hours or several times daily during severe secretion.

## VII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby. Product specifications described herein are subject to change without notice. SENSOTRACH® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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