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### 4.1 Fristen

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## 4 Hygienic preparation

### 4.1 Intervals

A dirty exhalation system can make exhalation difficult and prevent gas exchange. Therefore, it is important to clean Silentflow 2 to assure trouble-free functioning of the mask.

Clean Silentflow 2 after each use. To do this, dismantle the exhalation system as described above.

### 4.2 Procedures and Disinfecting Procedures:

The following procedures are allowed:

#### Cleaning by hand

Remove all residue with warm water and mild detergent, just as you would wash your dishes. Rinse all parts carefully with clean water.

#### Dishwasher

Wash cycle at maximum 65° C. Afterwards, rinse all parts carefully with clean water.

#### Disinfection\*

Clean all parts as described in "Cleaning by hand" and then dip the parts in a diluted disinfectant solution. After disinfection, rinse all parts thoroughly with **distilled** water.

Dry all parts thoroughly after cleaning. Check the components for any remaining dirt or damage. Repeat cleaning if necessary. If damaged, Silentflow 2 must be replaced. Assemble Silentflow 2 as described above.

\* We recommend **Gigasept FF**. Please be sure to observe the manufacturer's instructions. Tolerance to this disinfection agent has been proven. We recommend that you wear suitable gloves when disinfecting the equipment (e.g. household or disposable gloves).

### 4.3 Notes on disinfection

It may be necessary to disinfect the parts after each use by patients with a weakened immune system or a certain clinical background. Please consult your physician in this case.

### 4.4 Disinfection with change of patient

If the exhalation system is to be used for another patient, it must be hygienically treated beforehand. For this purpose, dismantle and disinfect all parts as described above or use a new exhalation system.

## 5 Maintenance

Silentflow 2 is maintenance-free.

When Silentflow 2 can no longer be used, dispose of it with your regular hospital waste disposal or with normal household waste removal.

## 6 Supply Schedule

Silentflow 2	WM 23600
consists of:	
- Silentflow 2	
- Instructions for use DE, GB, FR	WM 16656
<b>Other available products:</b>	
Instructions for use IT, NL, ES	WM 16776
Instructions for use DK, NO, SE	WM 16815

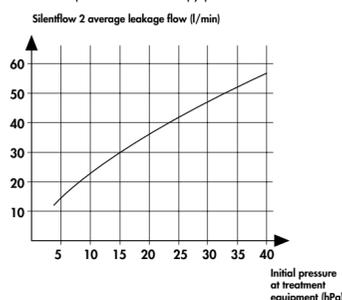
All parts listed are available as replacement parts.

## 7 Technical data

Device class according to 93/42/EEC:	Ila
Connection to mask:	Cone 22 mm diam. (EN ISO 5356-1)
Connection to hose: suitable for hoses made of	Ø 21.5 mm, 1° taper, elastomer with an interior connection diameter of 20 to 21 mm
Therapy pressure range:	4 hPa to 30 hPa
Weight:	1.5 grams
Temperature range:	
- Operating:	+5°C to + 40°C
- Storage:	-20°C to + 70°C
Flow resistance at 50 l/min	0,04 hPa
Flow resistance at 100 l/min	0,18 hPa
Noise level at 10 hPa:	28 hPa
Usage life:	6 months, with careful maintenance up to 12 months, depending on the frequency of use and cleaning
Material:	PC (polycarbonate)

Pressure-flow characteristic curve  
In the pressure-flow characteristic curve, the outlet flow is shown in dependence with therapy pressure.

(We reserve the right to make structural changes.)

## 8 Warranty

Weinmann guarantees that the product, when used in accordance with requirements, will remain free from defects for a period of 6 months from date of purchase.

Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.

We offer no warranty in the case of:

- Disregard of usage instructions
- Operating errors
- Improper use or handling
- Third-party intervention by non-authorized persons for the purpose of device repair
- Acts of God
- Transport damage as a result of improper packaging of returned items
- Operational and normal wear and tear
- Failure to use original spare parts.

Weinmann is not liable for consequential harm caused by a defect if it is not based on intention or gross negligence. Weinmann is also not liable for minor physical injury to life or limb resulting from negligence.

Weinmann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.

If Weinmann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer. Implied rights to claim under warranty remain unaffected by these changes.

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GmbH+Co. KG