DAR Shiley Phonate™

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ENGLISH

STERILE AND READY FOR USE: unless package has been opened or damaged. Reusable device, single patient. Discard after use. Exposure to elevated temperatures and ultraviolet light should be avoided during storage. This package contains two Shiley Phonate™ Speaking Valves. This instruction insert applies to the following Mallinckrodt products:

Shiley Phonate™ Speaking Valve **[REF]**SSV

Shiley Phonate™ Speaking Valve with Oxygen Port **[REF]**SSVO

PLEASE POST FOR USE BY ALL TRACHEOSTOMY CARE PERSONNEL.

WARNING:

Paediatric patients using the Phonate™ must be constantly observed or must be capable of removing the Phonate™ themselves in the event of an obstruction.

Use the Phonate™ only with a cuffless tube or a fenestrated cuffed tube. If a fenestrated cuffed tube is used, the cuff must always be deflated and the inner cannula, if used, must be fenestrated.

DESCRIPTION:

The Shiley SSV Phonate™ Speaking Valve and Shiley SSVO Phonate™ Speaking Valve with Oxygen Port are limited re-use devices with a one-way flutter valve and a 15mm female connector for use with a cuffless or an appropriately fenestrated tracheostomy tube. During spontaneous inspiration, the flutter valve opens and air is drawn through the valve and

tracheostomy tube into the trachea and lungs. The valve closes upon expiration, forcing the air to pass through the upper airway, and passing by the vocal cords thus allowing phonation. A port cap is provided on the oxygen port as a sanitary cover and to stop air backflow when the oxygen is not connected. The Phonate™ is a hinged single piece assembly. The valve cap and the main body can be opened to facilitate cleaning and inspection.

INDICATIONS:

The Phonate™ is intended for use only with a tracheostomy tube having a 15mm connector to provide a means of phonation for alert, awake tracheostomised patients who can breathe spontaneously without assisted mechanical ventilation. During expiration, air passage must be sufficient around the tracheostomy tube and through the fenestrations in the tracheostomy tube. The Phonate™ is intended for use by a single patient. The optional oxygen port is intended for administration of supplemental oxygen and/or humidification. The Phonate™ is intended for adult and paediatric patients.

CONTRAINDICATIONS:

This device is not for use in patients who are being mechanically ventilated or who have stenosis, vocal cord paralysis, severe tracheal stenosis, airway obstruction, respiratory infection, or heavy pulmonary mucous secretions. This device is not intended for use with endotracheal tubes or other artificial airways.

WARNING:

If the flexible diaphragm valve does not open during inspiration or close during expiration, remove the Phonate™ immediately. This device should only be used with patients who are alert and co-operative. This device should not be used during sleep. Each time the Phonate™ is attached to the tracheostomy tube the patient should be closely observed by a trained caregiver for a period of lime to ensure adequate respiration. Use the Phonate™ only with a cuffless tube or a fenestrated cuffed tube. If a fenestrated cuffed tube is used, the cuff must always be deflated and the inner cannula, if used, must be fenestrated. If a cuffless fenestrated tube is used, it is also recommended that the inner cannula, if used, be fenestrated. Do not use the Shiley Phonate™ Speaking Valve in line with a ventilator.

PRECAUTIONS:

Patients in the home care environment should be carefully instructed by a health care professional in the proper use and handling of this device. Do not attempt to remove the flexible diaphragm from the valve cap. Do not pull, twist, poke or exert excessive force which might damage the device. Use the Phonate™ only after secretions have minimised following tracheostomy. Determine that inspiratory effort is adequate, when supplemental oxygen and/or humidification is connected. Paediatric patients should not use the Phonate™ with oxygen port unless the port cap is not accessible to the patient.

DIRECTIONS FOR USE:

- Carefully remove the sterile device from its protective package.
- To attach the Shiley Phonate™ Speaking Valve, gently push the device onto the tracheostomy tube 15mm connector. Verify that the flexible diaphragm opens during patient's inspiration and closes during expiration.
- Observe the patient for a period of time and remove the Phonate™ if any respiratory distress occurs.
- To provide supplemental oxygen, when the oxygen port is present, remove the oxygen port cap and connect the oxygen line. Retain the cap to prevent air bypass when the oxygen line is not connected.
- To remove the Phonate™, carefully hold the tracheostomy tube in position while gently pulling the device from the tracheostomy tube, with a slight twisting motion.
- If the flexible diaphragm should become displaced during a cough or sneeze, the Phonate™ may temporarily leak air during expiration. The flexible diaphragm can easily be repositioned. First remove the Phonate™ (per step 4 above), then gently press the diaphragm back into place with a finger or blunt instrument.

The Phonate™ should be replaced after 29 days of normal use. Replace the Phonate™ immediately should the flexible diaphragm become damaged, sticky or malfunction in any way. Precautions should be taken when discarding this device and disposal of the device shall be made in accordance with applicable hospital or national regulations for biologically hazardous waste.

NOTE: the device is latex-free and non-conductive.

INSTRUCTION FOR THE CLEANING AND DISINFECTION OF THE DEVICE:

The Phonate™ should be cleaned daily. If mucous or secretions become attached to the device clean immediately.

- Remove the Phonate™ by carefully holding the tracheostomy tube in position while gently pulling the device from the tracheostomy tube, with a slight twisting motion.
- Disengage the valve cap from the main body for cleaning by pressing the cap away from the body at a position opposite the hinge. If an oxygen port is present, do not apply excess pressure to the side of the port during this step.
- Prepare one of the following cleaning solutions:
 - cool or lukewarm saline
 - soapy water (a pure soap is recommended)
 - diluted household strength hydrogen peroxide (one part water and one part peroxide)
 - diluted household strength vinegar (one part water and one part vinegar)
- Soak the opened Phonate™ in the cleaning solution for up to 15 minutes.
- After the soaking period, agitate the Phonate™ in the cleaning solution. Drain.
- Thoroughly rinse the Phonate™ in cool or lukewarm tap water to remove any cleaning solution residue.
- Examine the opened Phonate™ for remaining debris or encrustations. Ensure that the flexible diaphragm is clean, not torn or sticky, and lies flat in the valve cap. Check that the inner surface of the main body is smooth and undamaged. If an oxygen port is present, check for clear and unobstructed passageway. If clean, go to step 9.
- If necessary, use a moistened cotton tipped applicator to gently remove any remaining debris or encrustations. Repeat rinsing and examination following steps 6 and 7 if required.
- Close the Phonate™ by firmly pressing the valve cap over the main body to engage the snap fitting.

WARNING: Do not attempt to clean the Phonate™ while attached to the tracheostomy tube. Do not autoclave.

PRECAUTIONS:

Do not use any solution other than the recommended solutions for cleaning. In particular, DO NOT use alcoholic or germicidal based solutions or bleach to clean the device. Do not scrape or poke the device with a knife nor sharp object. Do not scrub with a sharp bristle brush when cleaning.

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