WILAqua

- DE Sterilwasser zur Atemgasbefeuchtung**
- EN Sterile water for respiratory humidification**
- FR Eau stérile pour humidification respiratoire**
- IT Acqua sterile per umidificazione respiratoria**
- ES Agua esterilizada por humificación respiradora**

CE 0653
V 1.0
Stand: 2014-09-12

Key to Symbols/Zeichenerklärung
Signification des symboles/Spiegazione dei simboli
Significado de los símbolos

REF	LOT	STERILE
Re-order number Bestellnummer No. de référence Codice articolo Número de referencia	Lot number Chargennummer Numéro de lot Lotto Número de lote	Steril Steril Stéril Sterile Estéril
Do not re-use Nicht wieder verwendbar Ne pas réutiliser Non riutilizzare No reutilizar	See instructions Handbuch beachten Lire le mode d'emploi Leggere il manuale Consultar el manual	Use by date Verfallsdatum Date d'expiration Scadenza Fecha de caducidad
Manufacturer Hersteller Producteur Produttore Fabricante	Storage Lagerung Stockage Stoccaggio Estocaje	

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Intended Use

Pyrogen-free sterile water for the humidification of inspired gases during invasive and non-invasive ventilation.

Instruction for use

(1) Hook the sterile water bottle at a suitable height to the humidifying chamber (min. 50 cm). (2) Remove one of both hygienic caps from the bottle top and wipe the seal with a disinfectant agent. (3) Press the spike of the auto-fill tube through the seal of the hygienic cap. (4) If the pressure compensation line is used, please remove the second hygienic cap and act as described before for the auto-fill tube. (5) Ensure that there are no leaks in the connection.

Warnings and Precautions

(1) Not for intravenous use. (2) Not for oral use. (3) Not for the treatment of wounds. (4) Sterile water must only be used if it is clear, colourless, and without visible particles. (5) Sterile water must be used within 7 days. (6) Do not heat above 65°C (149F).

Storage

Can be stored two years if packaging is undamaged and if kept under the correct storage conditions. Store out of direct sunlight. Please note the expiry date.

Storage and Transport Temperature

Temperature during storage and transport should be kept in the range of 5°C (41F) to 25°C (77F).

Contents and Materials

1000 ml – PE bottle with sealed perforation outlet

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