# Respironics® Total™ Face Mask

Instructions for Use Reorder #302433

LATEX-FREE

The Respironics Total Face Mask is intended to provide an interface for adult patients when used with Respironics CPAP or BiPAP® therapy. The mask contains exhalation ports that provide a continuous leak path in the patient circuit. This mask does not require the use of a separate exhalation device; however, you can use it with additional Respironics exhalation devices if you remove the accessory swivel. The Total Face Mask is intended for multiple patient use and up to 10 cleaning and disinfection cycles.

**Caution:** US federal law restricts this device to sale by or on the order of a physician.

## Warnings

- The Respironics Total Face Mask is designed for use with CPAP and BiPAP systems recommended by your physician or respiratory therapist. Do not wear this mask unless the CPAP or BiPAP system is turned on and operating properly. Do not try to block or seal the exhalation ports in the mask. Most CPAP systems have the potential to allow rebreathing of exhaled air. CPAP systems are designed for use with masks (or connectors) that have vent holes or exhalation ports. When the CPAP system is turned on, new air from the system flushes the exhaled air out through the vent holes. If the CPAP system is not operating, the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation.
- A minimum pressure of 5 cm H<sub>2</sub>O must be maintained when using the Respironics Total Face Mask.
- System pressure and oxygen therapy may be different for a Respironics Total Face Mask than for a nasal or nasal/oral interface. When changing mask types, determine the appropriate pressure level and oxygen therapy. Because a small leak exists at the nose area of the mask and through the exhalation ports, you must ensure that the pressure and oxygen therapy are maintained against this higher leak level.
- When including an additional exhalation device to the patient circuit, you may need to adjust the pressure level to compensate for the additional leak of the exhalation device.
- The entrainment valve contains a magnet. Do not use this mask in or near magnetic resonance imaging (MRI) equipment.
- To minimize the risks of vomiting during sleep, the patient should avoid eating or drinking for three hours before use.
- The patient's physician should be contacted if the patient experiences the following symptoms while using the mask or after removing it: Unusual chest discomfort, shortness of breath, stomach distension, belching, or severe headache; drying of the eyes, eye pain, or eye infections; blurred vision. (Consult an ophthalmologist if symptoms persist.)

## Contraindications

This mask may not be suitable for use on patients with the following conditions: glaucoma, recent eye surgery or dry eyes, nocturnal vomiting, hiatal hernia; impaired cardiac sphincter function; or on patients unable to remove the mask.

## Verifying Entrainment Valve Function

With the unit turned off, the entrainment valve flapper (internal blue disk) should be open, allowing the user to breathe room air. Turn the unit On. The flapper should close, allowing the user to receive airflow from the unit. If the valve still does not function properly, notify your home care dealer. **Do not** block the opening on the entrainment valve. **Check the valve before each use** for dysfunction or any blockage by patient secretions. To ensure proper entrainment valve function, verify that the flapper is free moving and dry.

## Connecting the Mask and Headgear

1. Set up the CPAP or BiPAP unit and patient circuit according to the unit's instruction manual.
2. To ensure a proper fit, place the patient's chin in the mask first, then lightly press the mask against the patient's face.
3. Pull the headgear straps around each side of the head and secure them to the mask. Do not overtighten the straps. Overtightening can cause or worsen leaks.
4. Connect the circuit tubing to the mask swivel. Turn the CPAP or BiPAP unit on.
5. Verify that there is only minor lifting of the mask from the patient's face (approximately 1/4" to 1/2") during a full respiratory cycle. Have the patient assume various sleeping positions (e.g., supine, lateral, prone) with the mask in place. Readjust the straps if necessary. If the inner flap covers the mouth, readjust the mask fit.

## Removing the Mask and Headgear

To remove the mask quickly, pull on the quick release cord, away from the patient's face, and remove the mask. The mask and headgear can also be removed by detaching the headgear straps from the tabs on the mask.

## Cleaning

**For Multi-Patient Use Reusable.** See the disinfection instructions later in this document.

Clean all parts of the mask daily. Clean the headgear at least once each week or more often if needed. Remove headgear before cleaning.




### Mask

Do not use alcohol, cleaning solutions containing alcohol, or any strong household cleaners. Do not use vinegar or warm water.

1. Hand wash the mask with a soft bristle brush in cool tap water with a mild dishwashing detergent. Do not use cleaners containing conditioners or moisturizers. Do not use bleach.
2. Rinse thoroughly. Air dry. Make sure the mask is dry before use.
3. If any parts are damaged or can not be cleaned, replace the mask.

### Helpful Tips for Mask Comfort


- Clean the mask thoroughly before every use, including first-time usage.
- Clean the patient's face thoroughly before each use.
- Make sure the mask and headgear are the correct size for the patient's face. If excessive discomfort is experienced, have the mask readjusted or consider an alternative mask.
- Do not overtighten the headgear. Overtightening can irritate the patient's face, increase leaks, or cause damage to the cushion.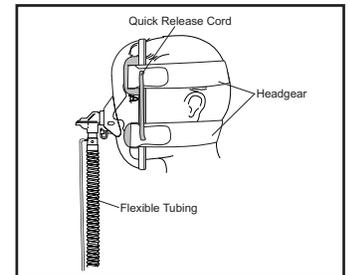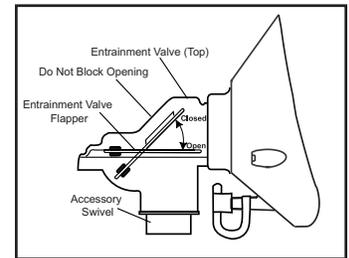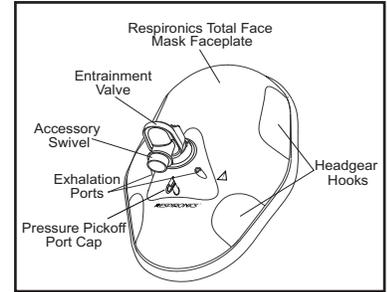

The total volume of the mask is 1500 cc. The volume of the area around the nose piece and entrainment valve where most of the gas exchange occurs is 80 cc.

### Headgear

1. Hand or machine wash with a standard laundry detergent and warm water. Rinse thoroughly. Do not use bleach.
2. Line dry or machine dry at the medium heat setting.

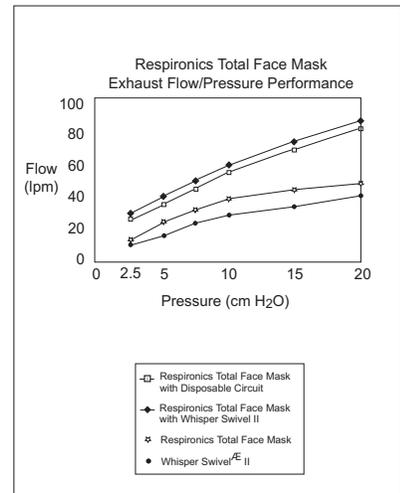  
**RESPIRONICS®**  
1001 Murry Ridge Lane  
Murrysville, Pennsylvania  
15668-8550 USA

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English  
1025904  
GK 01/26/05

## Disinfection

Clean all parts of the mask before disinfecting it as described below.

---

### PASTEURIZATION

Temperature: 70°C  
Exposure time: 30 minutes  
Result: High level disinfection  
Maximum allowable cleaning cycles: 10

- If the swivel does not move freely after disinfection, replace the sealing ring (part no. 452072).
  - Verify function of the flapper valve after processing. If the flapper does not move freely, replace the mask.
  - Inspect the mask after processing. If any componenets are damaged, replace the Total Face Mask.
- 

### CIDEX®

Exposure time: 45 minutes  
Rinse: Per manufacturer's instructions  
Result: High level disinfection  
Maximum allowable cleaning cycles: 10

- If the swivel does not move freely after disinfection, replace the sealing ring (part no. 452072).
  - Verify function of the flapper valve after processing. If the flapper does not move freely, replace the mask.
  - Inspect the mask after processing. If any componenets are damaged, replace the Total Face Mask.
  - Slight discoloration of the mask cushion after processing is normal.
- 

### CIDEX O. P. A.®

Exposure time: 12 minutes  
Rinse: Per manufacturer's instructions  
Result: High level disinfection  
Maximum allowable cleaning cycles: 10

- If the swivel does not move freely after disinfection, replace the sealing ring (part no. 452072).
  - Verify function of the flapper valve after processing. If the flapper does not move freely, replace the mask.
  - Inspect the mask after processing. If any componenets are damaged, replace the Total Face Mask.
  - Slight discoloration of the mask cushion after processing is normal.
- 

### STERRAD® System 100

Exposure time: Machine controlled  
Result: Sterilization  
Maximum allowable cleaning cycles: 10

- If the swivel does not move freely after disinfection, replace the sealing ring (part no. 452072).
  - Verify function of the flapper valve after processing. If the flapper does not move freely, replace the mask.
  - Inspect the mask after processing. If any componenets are damaged, replace the Total Face Mask.
  - A slight odor may be present after processing. This is normal.
- 

*Total is a trademark of Respirationics, Inc.  
CIDEX, CIDEX O. P. A., and STERRAD System are  
registered trademarks of Advanced Sterilization Products.*

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Total je blagovna znamka družbe Respirationics, Inc.