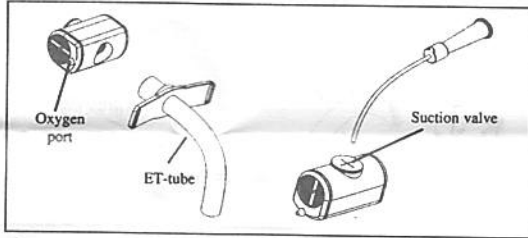
en

Instructions for use (see pictures)

- Directly on the tube during weaning.
- If supplemental oxygen is required connect an oxygen tube to the oxygen port. The oxygen will be humidified through the Trach-Vent+. Concentration of oxygen depends on the flow, see performance chart.
- Valve for forced inspiration/expiration and suctioning. Also works as a security valve.
- When using Trach-Vent+ on a silver cannulae you need an applicable ET-tube adapter.
- Trach-Vent+ not made with natural rubber latex

Caution!

- Store away unused products in clean environment.
- Replace every 24 hours or as needed to prevent accumulation of secretion.

Re-processing of medical devices intended for single use only may result in degraded performance or a loss of functionality. Re-use of single use only medical devices may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these medical devices. This product is not designed to be cleaned, disinfected, or sterilized.

- Some products contain phthalates : See product label

Contains DEHP

The results of certain animal experiments have shown phthalates to be potentially toxic to reproduction. Proceeding from the present state of scientific knowledge, risks for male premature infants cannot be excluded in the case of long-term exposure or application. Medical devices containing phthalates should be used only temporarily with pregnant women, nursing mothers, babies and infants.

- Replace unit immediately if soiled with secretion or otherwise obstructed.
- Trach-Vent+ is designed for single use only and must not be cleaned and re-used.
- Take total dead space into consideration before use.

Storage Instruction

Store in a cool and dry place, protected from light and ozone (no ultraviolet light, if applicable).

Keep away from sunlight and keep dry. Do not use if the product sterilization barrier or its packaging is compromised.

Contraindications


- Trach-Vent+ should not be used with patients producing fulminating frothy secretions within their airways, or patients with haemoptysis.
- Do not use in conjunction with conventional humidifiers and nebulizers.
- Do not add moisture to the HME.
- When an uncuffed endotracheal tube is used, leakage around the tube normally occurs, this reduces the humidification efficiency.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. (For USA and Canada only)

Explanation of important symbols and markings on the product label.

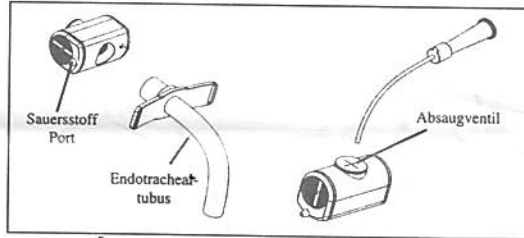Qty	Quantity, Number of items
	Not made with natural rubber latex
	Do not use if package is damaged

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