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## **GB** Instructions for use of

sanabelle TC100 tracheal compress foam for adults, REF 77700811

sanabelle TC150 tracheal compress foam for children, REF 77700815

### 1. Precautions

Please read the following instructions before using the product. The following instructions apply only to the above mentioned disposable compress and must be known to the user.

Attention, see precautions.

Gamma irradiation sterilised.

Do not use contents if pouch is opened or damaged.

Do not reuse.

### 2. Sizes / Features

The size of TC100 compresses for adults (REF 77700811) is 9 x 10 x 0.5 cm (width x length x thickness) and for TC150 compresses for children (REF 77700815) is 6.5 x 6.5 x 0.5 cm (width x length x thickness).

Each compress features a circular opening as well as a sinuous line slit. The diameter of the opening on REF 77700811 is 11 mm and on REF 77700815 is 6 mm.

Content: Each package contains 10 pieces of single packed tracheal compress foam.

### 3. Description / Properties

sanabelle tracheal compress foam is made of a highly absorbent open pore polyurethane foam. The foam has a yellowish, non-sticking surface layer on the side facing the tracheostoma which allows quick absorption of secretions into the foam's core. Even in the presence of an irritated and weeping stoma the compress does not stick to the skin, and can therefore be removed and replaced, without causing trauma or pain.

The opposite side is covered with a skin colored, vapor permeable polyurethane film. This protective layer will prevent leakage of secretions and bacterial contamination through the front of the compress.

Due to the circular opening and the sinuous line slit, the compress can easily be attached behind the flange of the tracheostomy tube in place.

#### 4. Indications

The sanabelle tracheal compress foam with central opening and sinuous line slit can be used to dress a newly established or an existing, irritated tracheostoma, as well as healed but weeping tracheostomas and any other kind of artificial orifice. This compress in particular meets modern standards of moist wound treatment.

**IMPORTANT:** The compress should be removed if allergic reactions such as itching, redness, or other signs of hypersensitivity occur.

#### 5. Instructions for Use

Procedure

- Clean and dry the tracheostoma and its surrounding as usual.
- Remove the sanabelle tracheal compress foam from the plastic tray before attaching the compress to the tracheostomy tube between neck and flange. Apply the yellowish, absorbent side of the compress toward the patient's neck.
- The slit of the tracheal compress for adults should face upwards.
- For the children's tracheal compress it is recommended positioning it with the slit facing downwards.
- If required, the compress may be cut to a specific shape and/or held in place with an adhesive tape.

#### 6. Change / Removal

During the early stages of treatment particularly following tracheotomy, sanabelle tracheal compress foam should be inspected frequently at short intervals. Swelling

and color of the front side of the compress depend on the amount of secretion from the tracheostoma and indicate, if capacity limits have been achieved which requires subsequent change.

Removal is simple due to the sinuous line slit. First of all, it is advisable to untighten the tracheostomy tube holder. Then carefully pull the compress for adults downwards away from the patient. For the children's compress we recommend turning it until the slit also faces upwards for removing downwards. With the tracheostomy tube in situ attach a new compress behind the flange in reverse action.

**IMPORTANT:** sanabelle tracheal compress foam must not be reused. Any attempt of cleaning or disinfection may damage or destroy parts of the compress.

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