Manufactured by:

ResMed Ltd | Elizabeth MacArthur Drive Bella Vista NSW 2153 Australia
Tel: +61 (2) 8884 1000 or 1 800 658 189 (toll free) Fax: +61 (2) 8883 3114 Email: reception@resmed.com.au

ResMed Offices:

ResMed Corp | 14040 Danielson Street Poway CA 92064-6857 USA
Tel: +1 (858) 746-2400 or 1-800-424-0737 (toll free) Fax: +1 (858) 746-2900 Email: reception@resmed.com

ResMed (UK) Ltd | 65 Milton Park Abingdon Oxfordshire OX14 4RX UK
Tel: +44 (1235) 862 997 Fax: +44 (1235) 831 336 Email: reception@resmed.co.uk

ResMed Asia Pacific Ltd | 97 Waterloo Road North Ryde NSW 2113 Australia
Tel: +61 (2) 9886 5000 or 1 800 991 900 (toll free) Fax: +61 (2) 9889 1471 Email: reception@resmed.com.au

ResMed Austria GmbH | Purtscherstraße 18, 9500 Villach Österreich
Tel: +43 (4242) 430 830 Fax: +43 (4242) 430 830-4 Email: reception@resmed.com.at

ResMed BV | Nieuwe Parklaan 86, 2587 BV Den Haag Nederland
Tel: +31 (70) 358 6263 Fax: +31 (70) 358 4333 Email: ReceptionDH@Resmed.nl

ResMed Finland Oy | Eteläinen Salmitie 2 Masala 02430 Suomi
Tel: +358 9 8676820 Fax: +358 9 86768222 Email: reception@resmed.fi

ResMed GmbH & Co. KG | Fraunhoferstraße 16, 82152 Martinsried Deutschland
Tel: +49 (0) 89 / 99 01 - 00 or +49 (0) 800 2 777 000 (gebührenfrei) Fax: +49 (0) 89 / 99 01 - 10 55
Email: reception@resmed.de

ResMed Hong Kong Ltd | Room 1714 Miramar Tower 132-134 Nathan Road Tsim Sha Tsui Hong Kong
Tel: +852 2366 0707 Fax: +852 2366 4546 Email: reception@resmed.com.hk

ResMed Japan | Nihonbashi Hisamatsu Bldg. 4F 2-28-1 Nihonbashi-Hamacho Chuo-Ku Tokyo 103-0007 Japan
Tel: +81 (3) 3662 5056 Fax: +81 (3) 3662 5040

ResMed Malaysia Sdn Bhd | Suite E-10-20 Plaza Mon't Kiara No. 2 Jalan 1/70C Mon't Kiara 50480 Kuala Lumpur Malaysia
Tel: +60 3 6201 7177 Fax: +60 3 6201 2177 Email: reception@resmed.com.my

ResMed NZ Ltd | PO Box 51-048 Pakuranga Auckland New Zealand
Tel: +0800 737 633 (NZ toll free) or +64 274 737 633 Fax: +0800 737 634 (NZ toll free) or +64 9 239 0193
Email: reception@resmed.co.nz

ResMed SA | Parc de la Bandonnière 2 rue Maurice Audibert 69800 Saint-Priest France
Tel: +33 (0) 4 37 251 251 Fax: +33 (0) 4 37 251 260 Email: reception@resmed.fr

ResMed Schweiz | Thannerstrasse 57 CH-4054 Basel Schweiz
Tel: +41 (061) 307 9711 Fax: +41 (061) 307 9722 Email: info@resmed.ch

ResMed Singapore Pte Ltd | 238A Thomson Road # 12-03/04 Novena Square Tower A Singapore 307684
Tel: +65 6284 7177 Fax: +65 6284 7787 Email: reception@resmed.com.sg

ResMed Spain SL | C/ Arturo Soria 245 28033 Madrid España
Tel: +34 (93) 590 8154 Fax: +34 (93) 590 8153 Email: angelo@resmed.es

ResMed Sweden AB | Industrigatan 2 S-461 37 Trollhättan Sverige
Tel: +46 520 420 110 Fax: +46 520 397 15 Email: reception@resmed.se

US DESIGNATED AGENT: ResMed Corp

EU AUTHORIZED REPRESENTATIVE: ResMed (UK) Ltd

Ultra Mirage Full Face Mask

Protected by patents: AU 710733, AU 712236, AU 724360, AU 741003, AU 744593, AU 753244, AU 766623, AU 775051, DE 29724224, CA 2261790, EP 0956069, EP 1027905, JP 3686609, NZ 513052, NZ 526165, NZ 526168, US 6112746, US 6119693, US 6357441, US 6374826, US 6463931, US 6491034, US 6513526, US 6532961, US 6561190, US 6561191, US 6581602, US 6634358, US 6701927, US 6860269. Other patents pending.

Protected by design registrations: AU 157902, CN 200330102703.8, CN 200330102704.2, EP 098348, JP 1212687, JP 1214582, US D493885. Others pending.

Ultra Mirage and Mirage are trademarks of ResMed Ltd and Mirage is Registered in U.S. Patent and Trademark Office. Velcro is a registered trademark of Velcro Industries B.V.

© 2005 ResMed Ltd.

RESMED

Ultra Mirage™ Full Face Mask

Reorder No. 60877/1 05 10

USER GUIDE

English
Deutsch
Français
Italiano
Español
Português
Svenska
Nederlands

B Mask Parts / Teile der Maske / Composants du masque / Parti della maschera / Piezas de la mascarilla / Peças da máscara / Delar i masken / Maskeronderdelen

Forehead support / Stirnbauteil / Support frontal / Supporto frontale / Apoyo para la frente / Suporte da testa / Pannstöd / Voorhoofdsteun

Forehead support pad / Stirnpolster / Tampon du support frontal / Imbottitura del supporto frontale / Almohadilla del apoyo para la frente / Almofada do suporte da testa / Pannstödsdyna / Kussentje van voorhoofdsteun

Exhaust vent / Luftausslassöffnung / Orifice de ventilation / Presa per l'esalazione / Ventilación / Orificio de ventilação / Utløpsventil / Uitlaat-ventilatieopening

Luer lock port cap/s / Anschlusskappen für Luer-Verbindungsstück / Bouchon(s) d'entrée de raccord Luer / Tappi porte con connessione Luer / Tapas de puerto Luer lock / Tampas das portas com fecho do tipo Luer / Luerportlock / Poortkap(pen) met luer-vergrendeling

Valve / Ventil / Valve / Valvola / Válvula / Válvula / Ventil / Klep

Mask frame / Maskenhalter / Entourage rigide / Telaio della maschera / Armazón de la mascarilla / Armação da máscara / Maskram / Maskerframe

Mask cushion / Maskeneinsatz / Bulle du masque / Cuscinetto della maschera / Almohadilla de la mascarilla / Almofada da máscara / Maskkudde / Maskerkussentje

Cushion clip / Clip des Maskeneinsatzes / Clip de la bulle / Fermaglio del cuscinetto / Traba de la almohadilla / Dispositivo de fixação da almofada / Kuddklämma / Klem voor kussentje

Valve membrane / Ventilmembran / Membrane de la valve / Membrana della valvola / Membrana de la válvula / Membrana da válvula / Klaffhinna / Klepmembran

Elbow retainer / Kniestückhalter / Anneau de retenue du coude / Fermaglio del gomito / Retenedor del codo / Retentor do cotovelo / Båghållare / Bochtthouder

Valve frame / Ventilrahmen / Corps de la valve / Telaio della valvola / Armazón de la válvula / Armação da válvula / Klaffram / Klepframe


Upper Velcro® straps / Die oberen Velcro-Bänder / Bandes Velcro supérieures / Cinghie laterali superiori (Velcro) / Correas laterales superiores (Velcro) / Correias Velcro laterais superiores / Øvre sidorem / Bovenste Velcro klittenbandbevestiging

Lower Velcro straps / Die unteren Velcro-Bänder / Bandes Velcro inférieures / Cinghie laterali inferiori (Velcro) / Correas laterais inferiores (Velcro) / Correias Velcro laterais inferiores / Lägre sidans kardborreremmar / Onderste Velcro klittenbandbevestiging

Headgear clip / Kopfbandclip / Clip du harnais / Fermaglio del copricapo / Broche del arnés / Dispositivo de fixação do arnés para a cabeça / Hättans klämma / Hoofdbandklem


C Fitting the Mask / Anlegen der Maske / Mise en place du masque / Come indossare la maschera / Colocación de la mascarilla / Colocação da máscara / Passa in masken / Het masker opzetten

C-1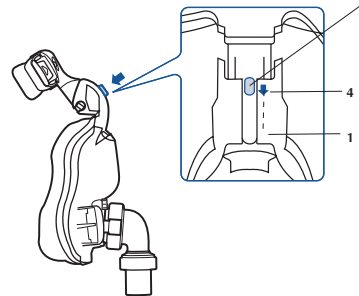


Forehead support tab / Stirnbauteillasche / languette du support frontal / Linguetta del supporto frontale / Lengüeta del apoyo para la frente / Dispositivo de fixação do suporte da testa / Pannstödsflik / Tab van de voorhoofdsteun


C-2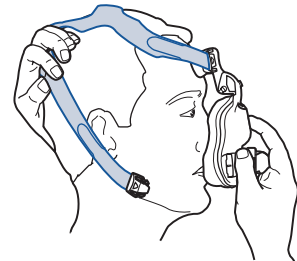


C-3

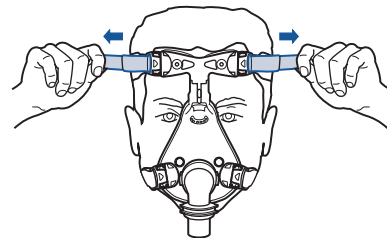
C-4

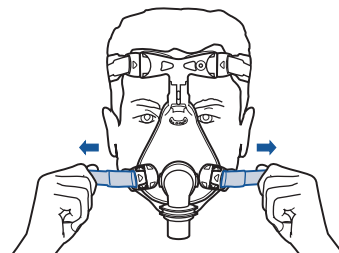C-5

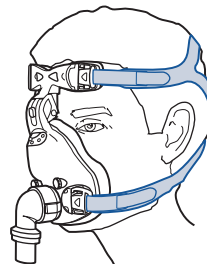C-6

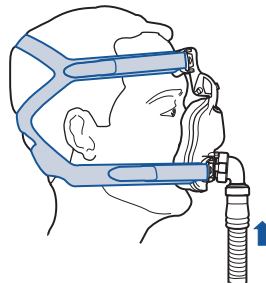
© 2005 ResMed Ltd.

D Disassembling for Cleaning / Demontage zu Reinigungszwecken / Démontage pour le nettoyage / Smontaggio per la pulizia / Desmontaje para la limpieza / Desmontagem para limpeza / Ta isär masken för rengöring / Demontage voor reiniging


D-1

L-shaped ridge / L-förmige Leiste / Bord en L / Costa sinistra a L / Borde en forma de L / Saliência em forma de L / Högra L- formade kanten / L-vormige rand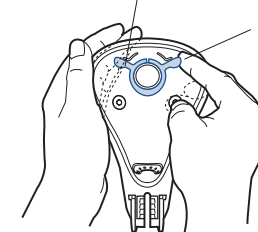


LIFT / HEBEN / SOULEVER / SOLLEVARE / LEVANTE / LEVANTE / LYFT / OMHOOGDOEN

D-2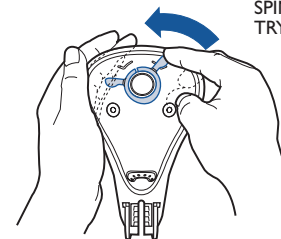


PUSH / DRÜCKEN / POUSSER / SPINGERE / EMPUJE / EMPURRE / TRYCK / DUWEN

D-3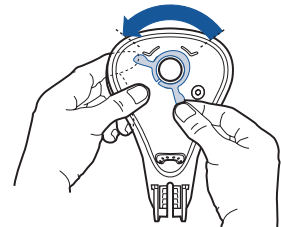


D-4

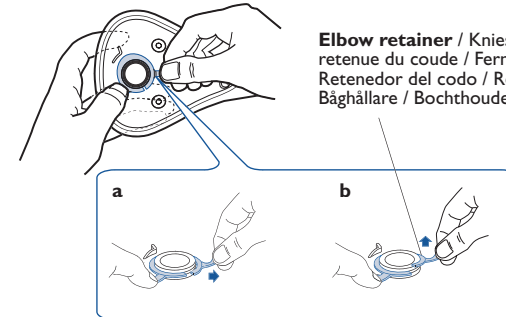Elbow retainer / Kniestückhalter / Anneau de retenue du coude / Fermaglio del gomito / Retenedor del codo / Retentor do cotovelo / Båghållare / Bochtthouder

D-5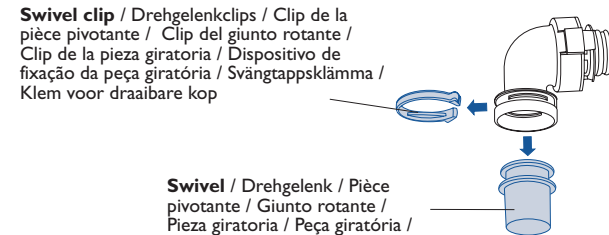


Swivel clip / Drehgelenkclips / Clip de la pièce pivotante / Clip del giunto rotante / Clip de la pieza giratoria / Dispositivo de fixação da peça giratória / Svängtappsklämma / Klem voor draaibare kop

Swivel / Drehgelenk / Pièce pivotante / Giunto rotante / Pieza giratoria / Peça giratória / Svängtapp / Draaibare kop

D-6

Valve elbow / Ventilkniestück / Coude de la valve / Gomito della valvola / Codo de la válvula / Cotovelo da válvula / Klaffbåge / Klepbocht


Valve frame / Ventilrahmen / Corps de la valve / Telaio della valvola / Armazón de la válvula / Armação da válvula / Klaffram / Klepframe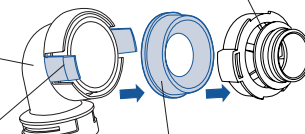


Tab / Lasche / Patte d'attache / Linguetta / Lengüeta / Lingueta / Flik / Tab

Valve membrane / Ventilmembran / Membrane de la valve / Membrana della valvola / Membrana de la válvula / Membrana da válvula / Klaffmenbran / Klepmembran

D-7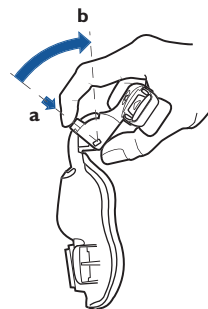

D-8

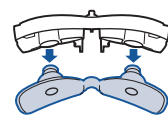
© 2005 ResMed Ltd.

Note: For complete instructions, read the text section in conjunction with the illustrations on this sheet. / **Hinweis:** Vollständige Anweisungen finden Sie im Textabschnitt und in den Illustrationen auf diesem Blatt. / **Remarque :** pour les instructions complètes, veuillez vous référer au texte et aux illustrations de cette fiche. / **Nota:** Per istruzioni complete, leggere la sezione con il testo insieme alle illustrazioni su questo foglio. / **Nota:** Para instrucciones detalladas, sírvase leer el texto en conjunto con las ilustraciones de esta hoja. / **Nota:** Para obter informações completas, leia a secção de texto juntamente com as ilustrações que se encontram neste folheto. / **Obs!** Läst textavsnittet tillsammans med illustrationerna på det här bladet för fullständiga anvisningar. / **NB:** Raadpleeg voor een volledige beschrijving het tekstgedeelte en de bijbehorende illustraties op dit blad.


Reassembling the Mask / Wiederzusammenbau der Maske / Remontage du masque / Riassemblaggio della maschera / Montaje de la mascarilla /
Montagem da máscara / Montera masken / Het masker opnieuw monteren

E Valve Reassembly / Wiederzusammenbau des Ventils / Réassemblage de la valve / Riassemblaggio della valvola / Montaje de la válvula /
Montagem da válvula / Montera klaffen / De klep opnieuw monteren


E-1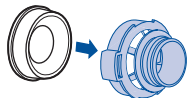


E-2

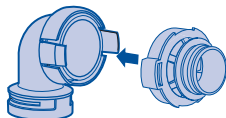E-3

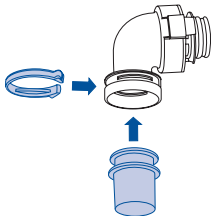E-4

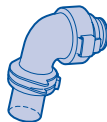© 2005 ResMed Ltd.


F Mask Reassembly / Wiederzusammenbau der Maske / Remontage du masque / Riassemblaggio della maschera / Montaje de la mascarilla /
Montagem da máscara / Montera masken / Het masker opnieuw monteren

F-1

Bump on the exhaust vent /
Erhebung an der Luftauslassöffnung /
Bosse de l'orifice de ventilation /
Protruberanza sulla presa per
l'esalazione / Protuberância de la
ventilación / Protuberância no orifício
de ventilação / Utbuktning på
Utløpsventil / Bobbel op de uitlaat-
ventilatieopening

F-2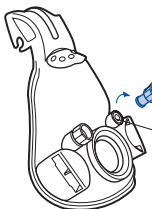


Luer lock port cap/s / Anschlusskappen
für Luer-Verbindungsstück / Bouchon(s)
d'entrée de raccord Luer / Tappi porte
con connessione Luer / Tapas de puerto
Luer lock / Tampas das portas com fecho
do tipo Luer / Luerportlock /
Poortkap(pen) met luer-vergrendeling


Mask port/s / Anschlussöffnung /
Port(s) du masque / Porta della
maschera / Puerto(s) de la mascarilla /
Orifício da máscara / Maskport /
Masterpoort(en)

F-3


Assembled valve inserted into mask frame /
Zusammengebautes Ventil wird in den Maskenhalter
eingesetzt / Valve assemblée insérée dans l'entourage rigide /
Valvola assemblata inserita nel telaio della maschera /
Válvula montada y colocada en el armazón de la mascarilla /
Válvula montada e inserida na armação da máscara /
Klaaffmonteringen sätts in på maskramen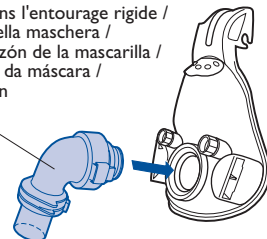

F-4

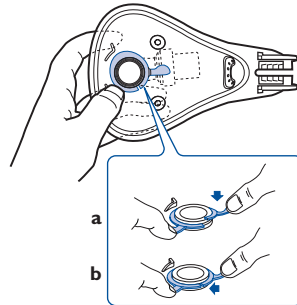
F-5

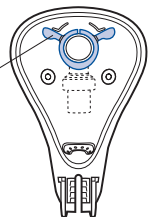
Elbow retainer / Kniestückhalter /
Anneau de retenue du coude /
Fermaglio del gomito / Retenedor
del codo / Retentor do cotovelo /
Båghållare / Bochthouder


F-6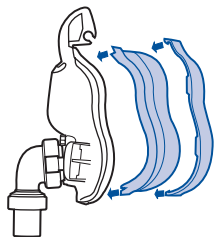

F-7

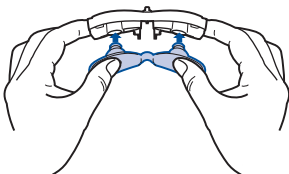F-8

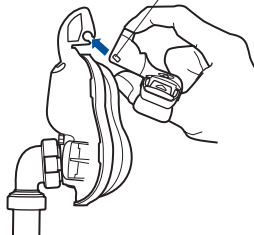
Pivot / Zapfen / Pivot /
Perno / Pivote / Pino /
Axel / Scharnier

© 2005 ResMed Ltd.

A

Pressure-Flow Curve / Druck-Durchflusskurve /
Courbe Pression-Débit / Curva di pressione e flusso /
Curva de Presión-Flujo / Curva de Pressão-Fluxo /
Tryck-Flödeskurvan / Druk-flow-curve

Vent flow rate / Durchflussrate an den
Luftauslassöffnungen / Débit à l'orifice de ventilation /
Coefficiente di flusso della presa per l'esalazione /
Caudal de flujo por la ventilación / Taxa de fluxo de
ventilação / Ventilflödestakt / Utløst-ventilationsroom
(L/min)

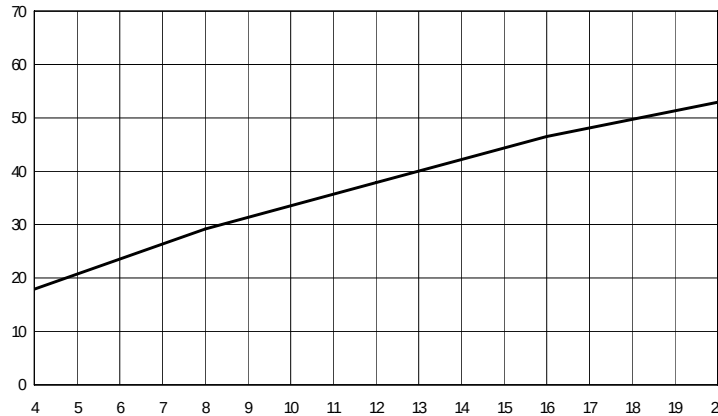Mask Pressure / Maskendruck / Pression dans le masque / Pressione della maschera /
Presión en la mascarilla / Pressão na máscara / Masktryck / Maskerdruck (cmH₂O)

© 2005 ResMed Ltd.

Ultra Mirage™ Full Face Mask

Thank you for choosing ResMed's
ULTRA MIRAGE™ FULL FACE MASK.

This is a full face mask which covers both the nose and the mouth. Thus, you can receive effective therapy even in the presence of mouth breathing and leak. This user's guide provides you with the information you need for the correct use of your mask.

INTENDED USE

The **ULTRA MIRAGE FULL FACE MASK** is intended for multipatient use for adult patients (>30 kg) prescribed continuous positive airway pressure (CPAP) or bilevel therapy in hospital, clinic and/or home environments.

MEDICAL INFORMATION

Note: *The mask system does not contain latex materials. If you have ANY reaction to any part of the mask system, discontinue use and consult your sleep therapist.*

CONTRAINDICATION

The **ULTRA MIRAGE FULL FACE MASK**, must be used under qualified supervision for patients who are unable to remove the mask by themselves.

WARNINGS AND CAUTIONS

- This mask should be used only with flow generators (CPAP or bilevel) recommended by your physician or respiratory therapist. A mask should not be used unless the flow generator is turned on and operating properly. The exhaust vent should never be blocked.

Explanation of the Warning: Flow generators are intended to be used with special masks with connectors which have vent holes or separate exhaust vents to allow continuous flow of air out of the mask. When the flow generator is turned on and functioning properly, new air from the flow generator flushes the exhaled air out through the attached mask exhaust vent. However, when the flow generator is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. In case of power failure, air is provided through the anti-asphyxia valve but there is no therapy. This warning applies to most models of flow generators.

Note: *The anti-asphyxia valve ensures that the air flow is regulated when the flow generator is turned on or off. Before using the mask, it is important to*

familiarise yourself with the functioning of the anti-asphyxia valve membrane which automatically closes when the flow generator starts operating, and opens as soon as the flow generator stops. Test its functioning by fitting the mask firmly against your face and sealing the swivel end of the valve with your hand. You should then still be able to breathe freely.

- The mask should not be worn if the anti-asphyxia valve membrane is damaged or missing. If the valve membrane is damaged, distorted or torn, it should be replaced with a new valve membrane.
- The exhaust vent flow is lower at lower mask pressures (see Pressure/Flow Curve— section **A**). If the mask pressure during expiration is too low, there may not be enough exhaust vent flow to completely flush the air delivery hose of exhaled air before the start of the next inspiration.
- The mask may not be suitable for those at increased risk of gastric regurgitation.
- The mask must be cleaned and disinfected/sterilized if it is used between patients. Refer to the section “Cleaning the Mask between Patients” for details.
- If oxygen is used with the flow generator, the oxygen flow must be turned off when the flow generator is not operating.

Explanation of the Warning: When the flow generator is not in operation and the oxygen flow is left on, oxygen delivered into the air delivery tubing may accumulate within the flow generator enclosure and create a risk of fire. This applies to most types of flow generators.

- Oxygen supports combustion. Oxygen should not be used while you are smoking or in the presence of an open flame.
- Always ensure airflow is being generated by the device before the oxygen supply is turned on.
- Always turn the oxygen supply off before stopping the airflow from the device.
- Please refrain from smoking while oxygen is in use.

Note: *At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection and the leak rate.*

PARTS OF THE MASK

See section **B** on the illustrations sheet.

FITTING THE MASK

See section **C** on the illustrations sheet.

Note: *Before fitting the mask, refer to the section on “Cleaning the Mask in the Home”.*

The **ULTRA MIRAGE FULL FACE MASK** is supplied fully assembled.

- 1 Move the forehead support tab to Position 4 (**Fig. C-1**). Position 4 is the uppermost tab position.
- 2 Disconnect one lower headgear clip. Place the mask on the face and pull the headgear over your head (**Fig. C-2**). Ensure that the upper straps pass across your forehead and the lower straps below your ears.
- 3 Reattach the headgear clip to the mask frame.
- 4 Gently tighten the upper headgear straps (**Fig. C-3**).

- 5 Gently tighten the lower headgear straps (**Fig. C-4**).

The pads of the forehead support should lightly touch your forehead when the mask is in place. If the pads do not touch your forehead, you may need to change the position of the forehead support tab as described in step 8.

- 6 The fitted mask should look like this (**Fig. C-5**).

Test the functioning of the anti-asphyxia valve by sealing the swivel end of the valve with your hand and ensuring that you can still breathe freely.

- 7 Connect the main air tubing to the swivel (**Fig. C-6**). Then connect the free end of the air tubing to your flow generator and turn the flow generator on. If you feel air leaks around the cushion, you may need to gently tighten the headgear straps.

Note: Do not overtighten the headgear straps as this may cause discomfort, cushion creases and mask leak.

- 8 If you are unable to get a proper seal through gently tightening the headgear, you will need to change the position of the forehead support tab.

Loosen the headgear straps and move the forehead support tab to the next position (eg, from Position 4 move it downwards to Position 3). Gently tighten the upper headgear straps and then the lower headgear straps.

Take time to test all four positions of the forehead support tab and to adjust the headgear straps so that you can achieve a secure and comfortable fit.

Notes:

- If you are unable to resolve any leaks you may require another mask frame and/or another cushion size.

- Select “Mir Full” as the mask option when using the ULTRA MIRAGE FULL FACE MASK with ResMed flow generators that have mask setting options.

REMOVING THE MASK

To remove the ULTRA MIRAGE FULL FACE MASK, disconnect one of the lower headgear clips and pull the mask and headgear up over your head.

Note: The headgear clips are easy to locate and disconnect. They allow you to quickly remove the mask from your face in the event of an emergency.

DISASSEMBLING FOR CLEANING

See section D on the illustrations sheet.

- 1 Detach the mask from the air tubing.
- 2 Disconnect the headgear clips to release them from the mask frame. It is not necessary to remove the headgear clips from the headgear.
- 3 Remove the exhaust vent and port caps from the mask frame.
- 4 Detach the cushion assembly from the mask frame by depressing the cushion clip at the base of the mask frame and gently pulling it off. This releases the cushion and cushion clip.
Then, separate the cushion from the cushion clip.
- 5 To detach the elbow from the frame, lift (**Fig. D-1**) and push (**Fig. D-2**) the right wing of the elbow retainer over the right L-shaped ridge on the mask frame.

- 6 Rotate the elbow retainer to the position shown in **Figure D-3**. The elbow retainer is now in the correct position for removal.

- 7 To remove the elbow retainer, hold one wing of the elbow retainer down, and firmly pull the other wing out (**Fig. D-4a**) and then up (**Fig. D-4b**) over the top of the valve. Remove the elbow retainer and valve from the mask frame.
- 8 To remove the swivel, lift the swivel clip and slide it out (**Fig. D-5**).
- 9 Squeeze the tabs of the valve elbow to remove the valve frame from the valve elbow. Then, carefully remove the valve membrane from the valve frame (**Fig. D-6**).
- 10 To remove the forehead support from the mask, press the tab (**Fig. D-7a**) and move it upwards until you can slide it off the mask frame (**Fig. D-7b**).
- 11 Pull firmly at the base of the forehead support pad plugs to release the pads from the forehead support (**Fig D-8**).

CLEANING THE MASK IN THE HOME

Notes:

- *Wash your hands before cleaning the mask.*
- *Wash the headgear before first time use, as the dye may run. It is important to always handwash the headgear.*
- *The headgear may be washed without being disassembled.*

DAILY/AFTER EACH USE

Handwash the mask components in warm (approx. 30°C), soapy water for one minute. Use pure soap. Rinse the components well with drinking quality water and allow them to air dry out of direct sunlight.

To clean the valve membrane, immerse it in soapy water and gently rub the membrane to remove any foreign matter. Rinse the membrane and allow it to air dry out of direct sunlight.

Inspect the valve membrane for wear and deterioration. If the valve membrane is damaged, distorted or torn, replace it with a new valve membrane. Ensure that the membrane is clean and dry before re-inserting it into the valve.

WEEKLY

Handwash the headgear in warm (approx. 30°C), soapy water. Use pure soap. Rinse well and allow it to air dry out of direct sunlight.

CAUTION

- Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturizing or antibacterial soaps, or scented oils to clean any of the mask components. These solutions may cause damage and reduce the life of the product.
- Do not expose any part of the mask system or tubing to direct sunlight as it may cause the product to deteriorate.
- The headgear is sensitive to heat and must not be ironed.
- If any visible deterioration of a mask component is apparent (cracking, crazing, tears, etc), the mask component should be discarded and replaced.

CLEANING THE MASK BETWEEN PATIENTS

The mask system (excluding the valve membrane) is validated for multipatient use. Clinicians should refer to the *Masks Disinfection/Sterilization Guide* for details.

To clean the headgear between patients use the method described in the previous section “Cleaning the Mask in the Home”.

CAUTION

The valve membrane must be replaced with a new valve membrane when the mask is reused on another patient. This part cannot be disinfected or sterilized.

REASSEMBLING THE MASK

ATTENTION

It is important to follow the instructions detailed in this section. Any deviation from the specified order of assembly could affect the functioning of your mask.

Reassembling the entire mask involves two stages:

1. Valve Reassembly
2. Mask Reassembly

VALVE REASSEMBLY

See section **E** on the illustrations sheet.

Note: *This stage must be completed before Mask Reassembly.*

Make sure all parts of the mask system, especially the valve membrane, are clean and dry before reassembling. If the valve membrane is damaged, distorted or torn, replace it with a new valve membrane.

- 1 Insert the valve membrane into the valve frame and gently push it into place (**Fig. E-1**). Make sure it is seated evenly and flatly around the rim of the valve

frame, as the membrane may produce a noise while you are breathing out, if it is fitted incorrectly.

- 2 Push the valve frame over the tabs of the valve elbow until they lock together (**Fig. E-2**). Make sure the valve frame and valve elbow are securely connected to each other.
- 3 Insert the swivel into the end of the valve elbow and secure it with the swivel clip (**Fig. E-3**).
- 4 Set the assembled valve (**Fig. E-4**) aside for use in the following stage.

MASK REASSEMBLY

See section **F** on the illustrations sheet.

- 1 Check that the holes in the exhaust vent are not obstructed. If they are obstructed, replace the exhaust vent. Push the exhaust vent onto the mask frame so the small round bump on the exhaust vent is on the outside of the mask frame (**Fig. F-1**).
- 2 Twist the luer lock port caps onto the mask ports (**Fig. F-2**).
- 3 Insert the assembled valve into the mask frame (**Fig. F-3**).

Note: *The valve must be assembled (as depicted in Figure E-4), before it is inserted into the mask frame.*

- 4 Hold the mask frame and the inserted valve firmly together.
- 5 To attach the elbow retainer, fit one side of the elbow retainer against the groove of the valve. While holding one wing of the elbow retainer down (**Fig. F-4a**), press the other wing down until the elbow retainer clicks into place (**Fig. F-4b**).
- 6 Rotate the elbow retainer to its original position (**Fig. F-5**).

- 7 Check that you have fitted the valve and elbow retainer correctly by firmly pulling the valve away from the frame. If the valve is fitted correctly, it should not separate from the frame.

If the valve comes away from the frame, try refitting the elbow retainer as described in Steps 4 and 5. If the valve still comes away from the frame when you pull on it, you may need to replace the elbow retainer with a new one.

- 8 Fit the grooved edge of the mask cushion to the mask frame. Fit the top section first, then work your way around the mask, pushing the cushion onto the frame, ensuring a good seal has been achieved.

Then push the cushion clip over the cushion onto the mask frame, ensuring it clicks into place in all locations (**Fig. F-6**).

- 9 Attach the forehead pads to the forehead support by inserting the pad plugs into the two holes on the back of the forehead support (**Fig F-7**). Push the pads firmly against the forehead support to ensure that the pads fit securely into the forehead support.
- 10 Connect the forehead support to the mask frame. Slide the pivots into the pivot slot on the mask frame (**Fig. F-8**). Depress the forehead support tab and move it into any of the tab positions.
- 11 Insert the headgear clips into the slots on the mask frame. The headgear label should face outwards and be near your neck when fitted.

TECHNICAL SPECIFICATIONS

PRESSURE–FLOW CURVE

See section **A** on the illustrations sheet.

DEAD SPACE INFORMATION

Dead space is the empty volume of the mask up to the swivel.

The dead space of the mask varies according to cushion sizes but is less than 258 mL.

Note: *The manufacturer reserves the right to change these specifications without notice.*

REPLACEMENT PARTS

Part No.	Item
16604	Cushion, <i>Small, Standard</i> (1 pk.)
16671	Cushion, <i>Small, Shallow</i> (1 pk.)
16605	Cushion, <i>Medium, Standard</i> (1 pk.)
16672	Cushion, <i>Medium, Shallow</i> (1 pk.)
16606	Cushion, <i>Large, Standard</i> (1 pk.)
16673	Cushion, <i>Large, Shallow</i> (1 pk.)
16674	Cushion clip, <i>Small</i> (1 pk.)
16675	Cushion clip, <i>Medium</i> (1 pk.)
16676	Cushion clip, <i>Large</i> (1 pk.)
16613	Elbow assembly (1 pk.)
60683	Elbow retainer (1 pk.)
16509	Exhaust vent (1 pk.)
16510	Exhaust vent (10 pk.)
60681	Luer lock port caps (2 pk.)
60682	Luer lock port caps (10 pk.)
16689	Swivel (1 pk.)
60680	Swivel clip (1 pk.)
16637	Swivel clip (5 pk.) and Elbow clip (5 pk.)
16617	Valve membrane (1 pk.)
16618	Valve membrane (10 pk.)

Part No.	Item
60115	Headgear clip (2 pk.)
60116	Headgear clip (10 pk.)
60123	Forehead pad (1 pk.)
60124	Forehead pad (10 pk.)
60670	Mask Frame, <i>Small</i> (1 pk.)
60671	Mask Frame, <i>Medium</i> (1 pk.)
60672	Mask Frame, <i>Large</i> (1 pk.)
60673	Forehead Support (1 pk.)
60674	Headgear, <i>Standard</i> (1 pk.)
16118	Headgear, <i>Small</i> (1 pk.)
16119	Headgear, <i>Large</i> (1 pk.)

LIMITED WARRANTY

ResMed warrants that your ResMed mask system, including its components: mask frame, cushion, headgear, tubing and other accessories ("Product") shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

For further details refer to the Warranty.

Note: *Translations are based on the English version in this guide. However, there may be minor variations based on the disinfection/sterilization procedures applicable in some countries. ResMed has reviewed these changes and all procedures included here have been validated.*

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