RESMED

**Remote alarm
User Manual**

English • Deutsch • Français • Italiano • Español

ResMed Ventilation solutions
Making quality of care easy

ResMed Ventilation solutions
Making quality of care easy

RESMED

**Remote Alarm
User Manual**

English

English

Contents

1 Safety instructions	1
1.1 General advice	1
1.2 How to use	1
1.3 Recycling	1
2 Description	2
2.1 Contents of the kit	2
2.2 Alarm box	2
2.3 Cables	3
2.4 How the alarm works	4
2.5 Setting up	5
2.6 Disconnection	8
3 Troubleshooting	8
4 Product codes	9
4.1 Contents of a typical kit	9
4.2 Kits for Elisée devices	10
4.3 Kits for VS devices	10
4.4 Kits for VS III devices	11
4.5 Kit for Eole 3 S/XLS devices	11
4.6 Kit for Helia 2 devices	12
5 Technical specifications	12
5.1 Dimensions and weight of the alarm box	12
5.2 Operating conditions	12
5.3 Applicable standards	13
5.4 Symbols	13

1 Safety instructions

1.1 General advice

You must read and understand this manual in full before handling the remote alarm.

The manufacturer accepts no liability for damage to this accessory caused by modifications which do not comply with the instructions provided and which do not have the manufacturer's prior approval.

1.2 How to use

- The remote alarm is not splash proof and is for indoor use only.
- Do not block the openings in the remote alarm box.
- Do not open the alarm box other than to replace the battery. Only qualified technicians are permitted to open the device. The guarantee does not cover the opening of the device by an unqualified person.
- The remote alarm can be used with cables of various lengths. When using with a long cable, place the cable in such a way as to prevent anyone from tripping over it or slipping on it.
- The remote alarm is not recommended for use with any other device, and such use is at your own risk.

CAUTION

The alarm box does not tell you which alarm has been triggered. Refer to the Clinical Manual for the ventilator to identify the alarm and for instructions on how to respond.

1.3 Recycling

- In accordance with Directive 2002/96/EC on waste electrical and electronic equipment, this accessory must be sorted and disposed of separately from other types of rubbish. It must not be disposed of with ordinary municipal waste. To discard this accessory, you should use an appropriate waste collection system and the waste reuse and recycling systems in your area.

2 Description

The remote alarm is an accessory for ventilators in the Elisée™ and VS™, VS III™, Eole™ 3 S/XLS, Helia™ 2 ranges. It allows an alarm to be heard at a distance from the device, with the alarm signal being relayed along a cable.

2.1 Contents of the kit

The following are typically included in a remote alarm kit:

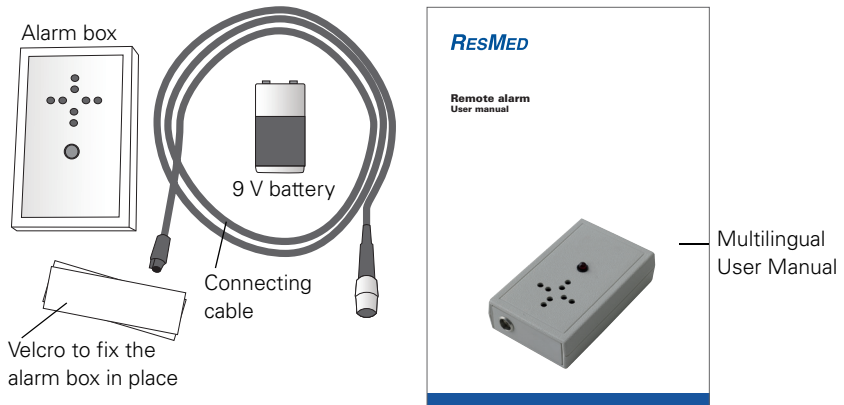
Figure 1: Contents of a typical remote alarm kit (VS ventilators)

2.2 Alarm box

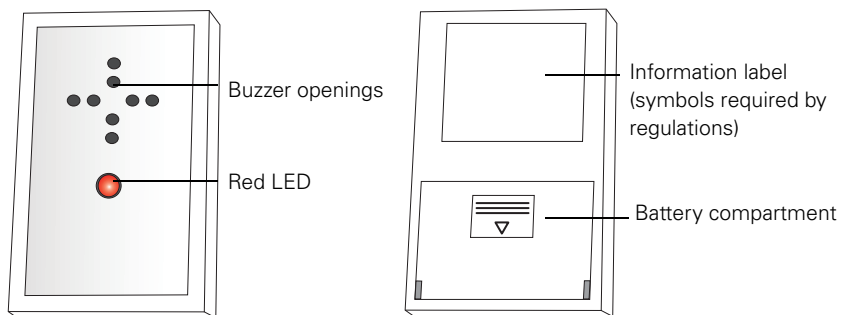
Figure 2: Top view (left) and underside view (right) of the remote alarm box

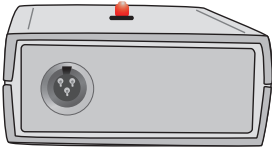
Figure 3: Front view showing the 3-pin connector (alarm box for a VS device)

Note: There are three different product codes for the alarm box, according to the ventilator being used. See „Product codes“ on page 9.

2.3 Cables

Four types of connecting cable are available, in various lengths, according to the ventilator being used (see „Product codes“ on page 9).

An adapter is additionally supplied with the Elisée ventilators.

It is used with Elisée 350 and Elisée 150 ventilators with hardware earlier than V 5.0.0 (serial number below EL2D0910700).

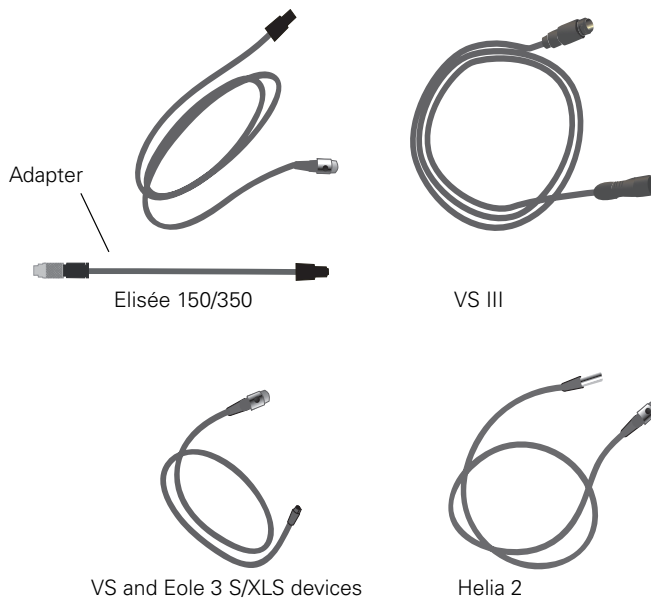
Figure 4: The various types of cable available

2.4 How the alarm works

The remote alarm is controlled by the opening and closing of a relay located inside the alarm box. The relay:

- Immediately triggers the remote alarm audible signal
- Turns on the red LED.

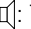
The alarm box may or may not make a sound, depending on the status of the ventilator (switched off, or delivering ventilation with or without an alarm).

The activity of the remote alarm box with various ResMed ventilators (Elisée, VS, VS III, Eole and Helia) is shown in the following tables.

Elisée and Eole XLS devices

Table 1: Activity of the remote alarm box

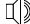
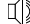
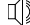
Ventilator status	Alarm box status
Ventilator off (not connected to mains power)	
Standby on mains power	
Delivering ventilation, no alarm	
Delivering ventilation, alarm triggered	

: The alarm box sounds; : The alarm box is silent.

VS and VS III devices

The alarm relay is configured via the technical menu of the ventilator. There are four preconfigurations (0, 1, 2, 3), as shown in the table below:

Table 2: Status of the remote alarm relay according to the preconfiguration

Preconfigurations Ventilator status	0	1	2	3
Ventilator off (not connected to mains power)				
Standby on mains power				
Delivering ventilation, no alarm				
Delivering ventilation, alarm triggered				

: The alarm box sounds; : The alarm box is silent.

CAUTION

Only a qualified technician should change the alarm relay configuration. Please refer to the relevant ventilator Service Manual.

2.5 Setting up

After placing the 9 V battery in its compartment:

1. Plug the ventilator into the mains power supply.
2. To prevent the alarm box from sounding as soon as it is connected, plug the cable into the ventilator first (in the socket marked ).

Note: If you are using an *Elisée* version that requires an adapter, connect the black connector of the adapter to the connecting cable, then connect the lockable connector of the adapter to the ventilator.

3. Then plug the other end of the cable into the alarm box.

The remote alarm system is ready for use (you can start ventilation).

CAUTION

- The charge level of the battery must be checked before first use, and weekly thereafter, by triggering an alarm that will have no impact on the ventilation supplied, for example by disconnecting the ventilator from the mains. The audible alarm should be triggered immediately. Do not forget to plug the ventilator back in after the test.
- All alarm boxes sound when they are disconnected from a ventilator, except for the Eole alarm boxes (reference BOI009327).

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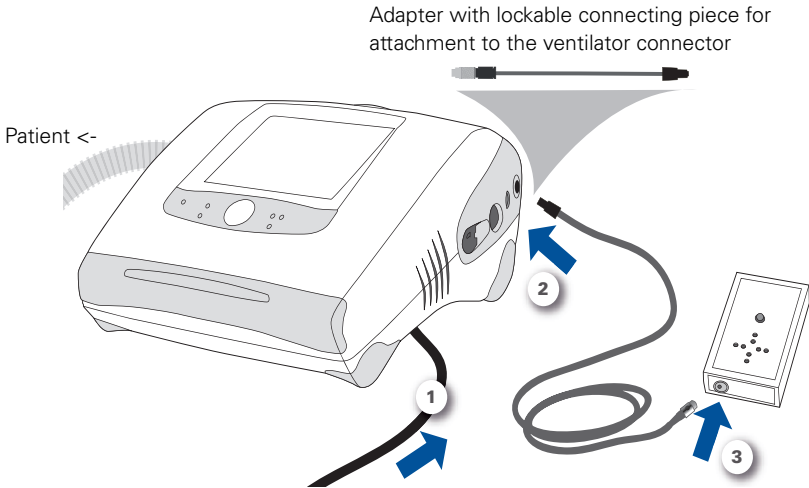
Figure 5: How to connect the alarm box to an Elisée ventilator

CAUTION

Do not use the ResMed Power Station when the remote alarm is being used with the Elisée 150.

VS devices

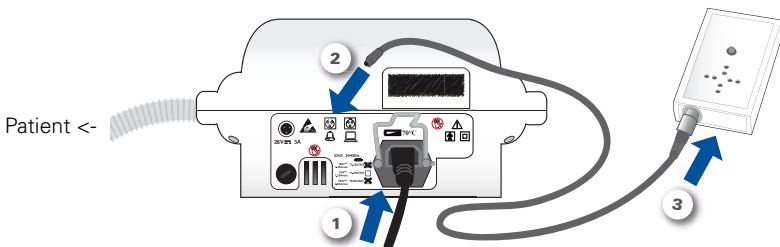
Figure 6: How to connect the alarm box to a VS ventilator

VS III

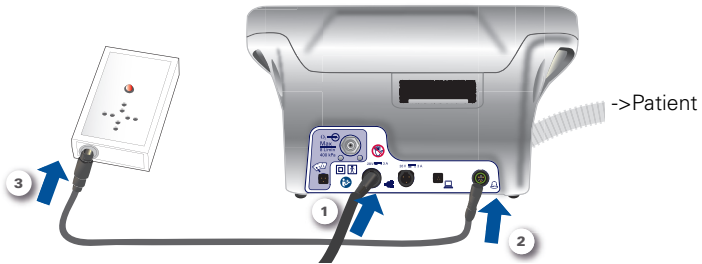
Figure 7: How to connect the alarm box to a VS III ventilator

Eole 3 S/XLS

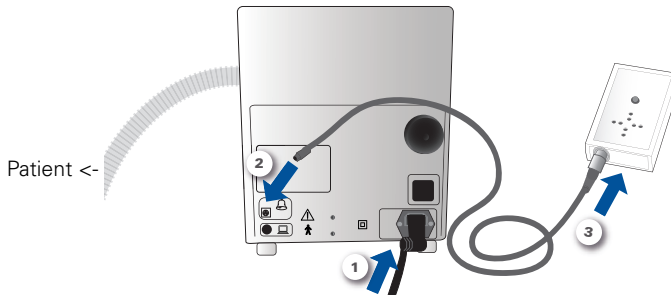
Figure 8: How to connect the alarm box to an Eole 3 S/XLS ventilator

Helia 2

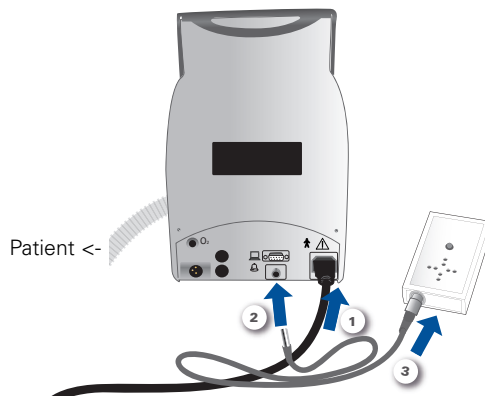
Figure 9: How to connect the alarm box to a Helia 2 ventilator

2.6 Disconnection

Follow the steps for setting up the remote alarm in the reverse order, to prevent the alarm from sounding unexpectedly.

1. Unplug the cable from the alarm box.
2. Then unplug the cable from the ventilator (the Elisée cable must be unscrewed).
3. Stop ventilation.

3 Troubleshooting

The following table lists the main causes of problems that may occur when using the remote alarm.

Table 3: Causes of problems and suggested solutions

Problem	Cause	Solution
The LED does not turn on when the remote alarm is connected to the ventilator.	Faulty LED.	Contact technical support.
	Poor connection.	Check that the connecting cable is firmly plugged into the socket on the alarm box. For Elisée ventilators requiring an adapter, make sure that it is firmly attached to the alarm box and the ventilator.
No audible signal when the alarm box is connected to the ventilator.	Power socket faulty or broken.	Contact technical support.
	Faulty buzzer.	Contact technical support.

Problem	Cause	Solution
Remote alarm not working (the ventilator has triggered an alarm but the LED is not on and there is no audible signal).	Battery faulty or flat.	Check the battery and change if necessary.
	The remote alarm is not connected properly to the ventilator.	Check that the connecting cable is firmly attached. For Elisée ventilators requiring an adapter, make sure that it is firmly attached to the alarm box and the ventilator.
	The connectors on the connecting cable or on the adapter are faulty.	Replace the connecting cable or the adapter.
	Incorrect alarm kit used.	Check that the remote alarm kit matches the ventilator being used; if it does not, contact technical support.
	The remote alarm being used is not of the recommended type.	We recommend you use only remote alarm kits sold by ResMed.
Unexpected activity of the remote alarm.	The alarm relay is incorrectly configured (VS and VS III devices).	Contact technical support to have the relay configuration checked.

4 Product codes

4.1 Contents of a typical kit

Table 4: Contents of a typical remote alarm kit

Description	Product code
Remote alarm box	The alarm box differs according to the ventilator used. See the kits below
Connecting cable	The cable differs according to the ventilator used. See the kits below

Description	Product code
Adapter (kits for Elisée devices only)	ADA015450
9 V battery	PIL007285
This User Manual	NOT015149
80 mm Velcro strip	RUB015575

4.2 Kits for Elisée devices

Table 5: Product codes for Elisée

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 3 m cable	KIT015422	Alarm box: BOI015128 Cable: CAB015260 Adapter: ADA015450
Remote alarm kit, 10 m cable	KIT015423	Alarm box: BOI015128 Cable: CAB015261 Adapter: ADA015450
Remote alarm kit, 20 m cable	KIT015424	Alarm box: BOI015128 Cable: CAB015262 Adapter: ADA015450
Remote alarm kit	KIT015803	Alarm box: BOI015128

4.3 Kits for VS devices

Table 6: Product codes for VS kits

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 20 cm cable	KIT011960	Alarm box: BOI010525 Cable: S/E011856
Remote alarm kit, 2.5 m cable	KIT011602	Alarm box: BOI010525 Cable: S/E009470
Remote alarm kit, 5 m cable	KIT013392	Alarm box: BOI010525 Cable: S/E013389
Remote alarm kit, 10 m cable	KIT013393	Alarm box: BOI010525 Cable: S/E013390

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 20 m cable	KIT013394	Alarm box: BOI010525 Cable: S/E013391
Remote alarm kit	KIT015804	Alarm box: BOI010525

4.4 Kits for VS III devices

Table 7: Product codes for VS III kits

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 20 cm cable	KIT015166	Alarm box: BOI015128 Cable: CAB015164
Remote alarm kit, 2.5 m cable	KIT015167	Alarm box: BOI015128 Cable: CAB015165
Remote alarm kit, 5 m cable	KIT015146	Alarm box: BOI015128 Cable: CAB015131
Remote alarm kit, 10 m cable	KIT015147	Alarm box: BOI015128 Cable: CAB014922
Remote alarm kit, 20 m cable	KIT015148	Alarm box: BOI015128 Cable: CAB015132
Remote alarm kit	KIT015803	Alarm box: BOI015128

4.5 Kit for Eole 3 S/XLS devices

Table 8: Product codes for the Eole 3 S/XLS kit

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 2.5 m cable	KIT011604	Alarm box: BOI009327 Cable: S/E009470
5 m cable	KIT013392	Cable: S/E013389
10 m cable	KIT013393	Cable: S/E013390
20 m cable	KIT013394	Cable: S/E013391

4.6 Kit for Helia 2 devices

Table 9: Product codes for the Helia 2 kit

Description	Kit product code	Alarm box/cable product codes
Remote alarm kit, 5 m cable	KIT011606	Alarm box: BOI010525 Cable: S/E007126
Remote alarm kit	KIT015804	Alarm box: BOI010525

5 Technical specifications

5.1 Dimensions and weight of the alarm box

Dimensions

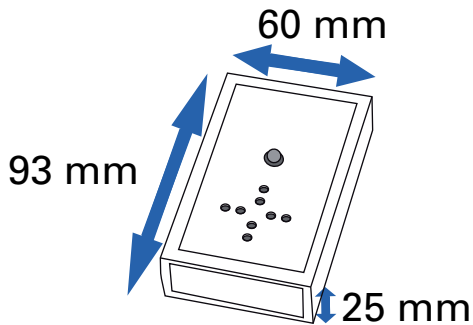
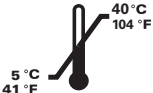
Figure 10: Dimensions of the remote alarm box

Weight

- Alarm box alone: 69 g
- Alarm box + 9 V battery: 115 g

5.2 Operating conditions

- Temperature:  5°C / 41°F to 40°C / 104°F
- Humidity:  10% to 95%

5.3 Applicable standards

- **NF EN 60601-1**: Medical Electrical Equipment – Part 1: General Requirements for Safety;
- **NF EN 60601-1-2**: Medical Electrical Equipment – Part 1–2: General Requirements for Safety – Associated Standard: Electromagnetic Compatibility – Requirements and Tests;

This device meets the following standards:

- Directive **93/42/EEC** concerning CE-marking (Notified body: TÜV Süd – CE0123);
- Directive **2002/96/EC**: Directive on waste electrical and electronic equipment (WEEE).

5.4 Symbols

These symbols appear on the label which is stuck to the underside of the alarm box.

(VS III, Elisée)

Caution, consult operating instructions

(Eole, Helia, VS)

Keep dry

CE0123

CE-marked device

Manufacturer

Device subject to separate collection

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See www.resmed.com for other ResMed locations worldwide. © 2011 ResMed Paris.