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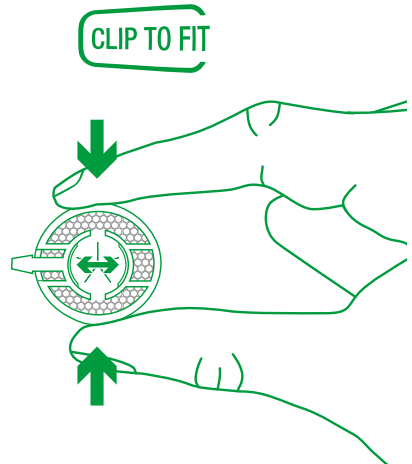
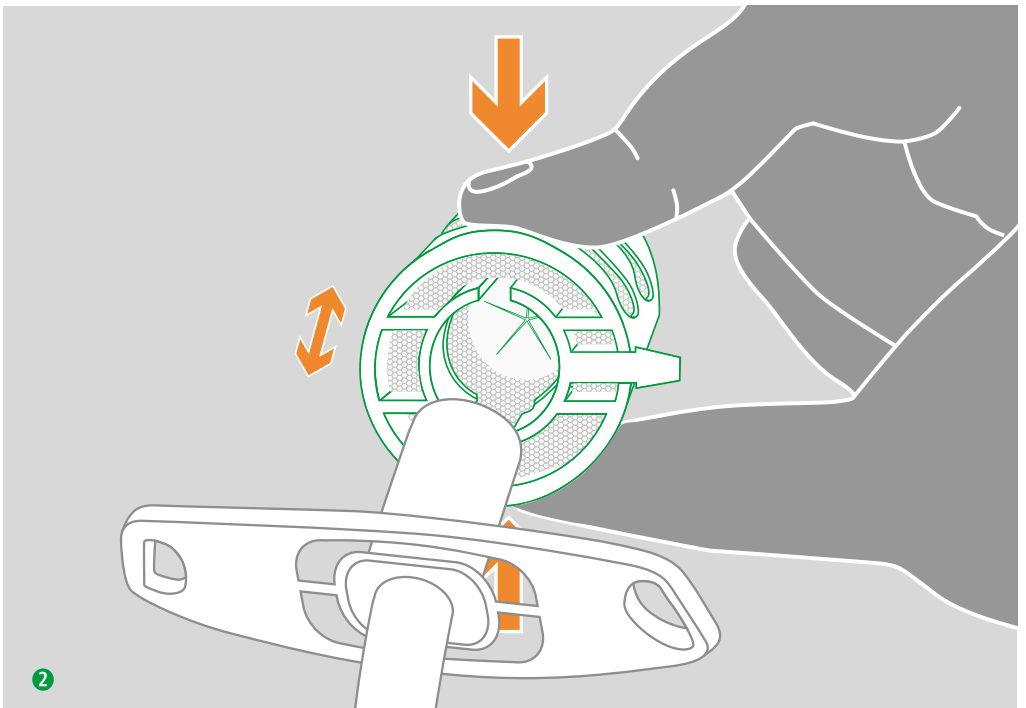
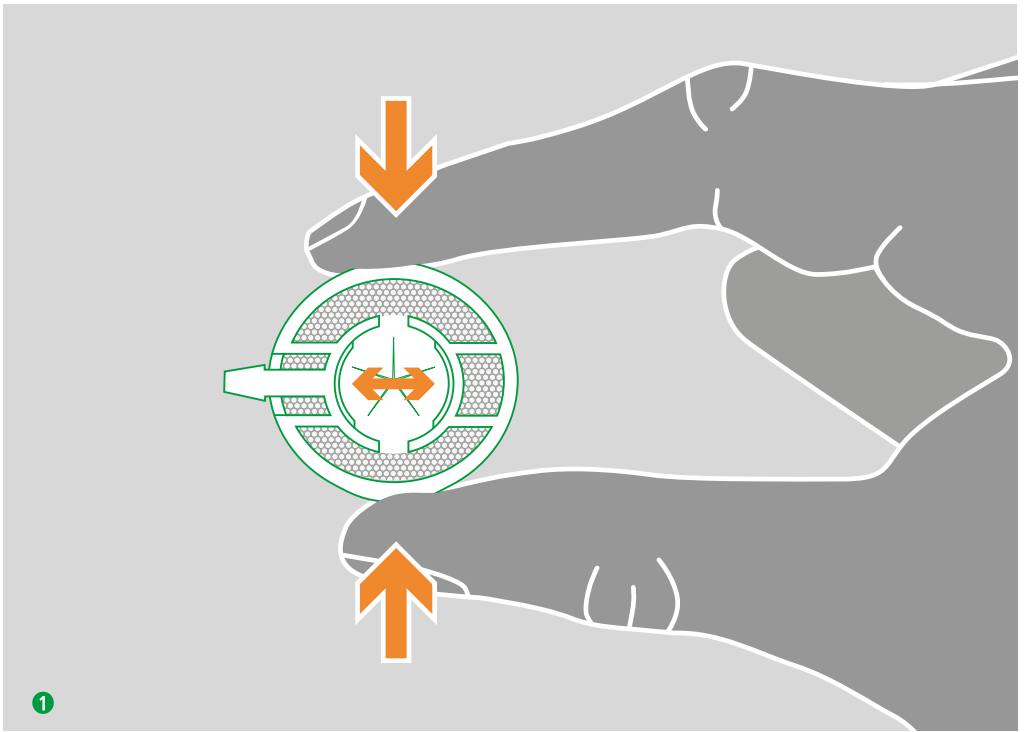
# **GB** sanabelle HME-A7

## Artificial nose with safety valve and oxygen connector

### Indication

For the efficient filtering, warming and humidifying of inhaled air for patients with spontaneous breathing through an endotracheal tube or tracheal cannula, in hospital or at home.

### Description

The sanabelle HME-A7 comprises a plastic housing with an insert of hydroscopic foam material. The HME has a 15 mm female connector, which can be fitted to the standardised 15 mm connector of the tracheal cannula. The HME is latex-free, DEHP-free and electrically non-conductive.

The safety valve minimizes the risk of occlusion of the humidifying medium, allows short-term high tidal volume and coughing up of larger quantities of secretions.

### Usage

Unpack the sanabelle HME-A7 only before immediate use carefully. Easy to insert and remove thanks to the patented **clip-to-fit** function. The casing is to be fitted to the open end of the endotracheal tube or tracheotomy cannula. For this purpose, squeeze the case at the grip moulds ① and slide the HME back to the catch ②. Ensure that the connection fits tightly! If supplemental oxygen has to be delivered, connect the oxygen connector to an oxygen supply. The normal humidifying effect sets in after the first few breaths. To easily remove the HME, squeeze the HME at the grip moulds and remove it from the 15 mm attachment ②.

The HME must not be used together with a humidifier operating with heated or nebulised water or in combination with a nebuliser.

**IMPORTANT:** The instructions for use of the sanabelle HME-A7 must be carefully read prior to use, to minimise the potential risk of dyspnoea, labored breathing, hypercapnia and hypoxia.

### Contraindications

- The sanabelle HME-A7 must not be used outside the recommended tidal volume range, in order to prevent an increased concentration of CO<sub>2</sub> in case of a low tidal volume, or insufficient humidification, if the tidal volume is too high. Please observe the doctor's recommendation. Please consult a specialist in case of questions.
- The device is not to be used on patients with strong exsiccosis or excessive secretions in the lungs and airways or in case of leaking gas flow (expired gas does not flow through the humidifier).
- Do not connect the HME to an artificial ventilation system.

### Precautions

- Always ensure the patency of the HME.
- Replace the sanabelle HME-A7 after a maximum of 24 hours to counteract any possible bacterial or fungicidal growth. The HME must be replaced more frequently depending on the needs, for example, if the breathing resistance is in danger of increasing due to accumulations of secretion.
- Monitor patients for potential fatigue! Upon indication of difficulty breathing, remove the HME immediately! Reason: the HME's resistance requires increased breathing effort.
- The sanabelle HME-A7 is for single use only! Cleaning, disinfecting or re-sterilising it for reuse may impair the function and increase the risk of transmitting infections.
- Dispose of used HMEs immediately in a hygienically proper manner!

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