


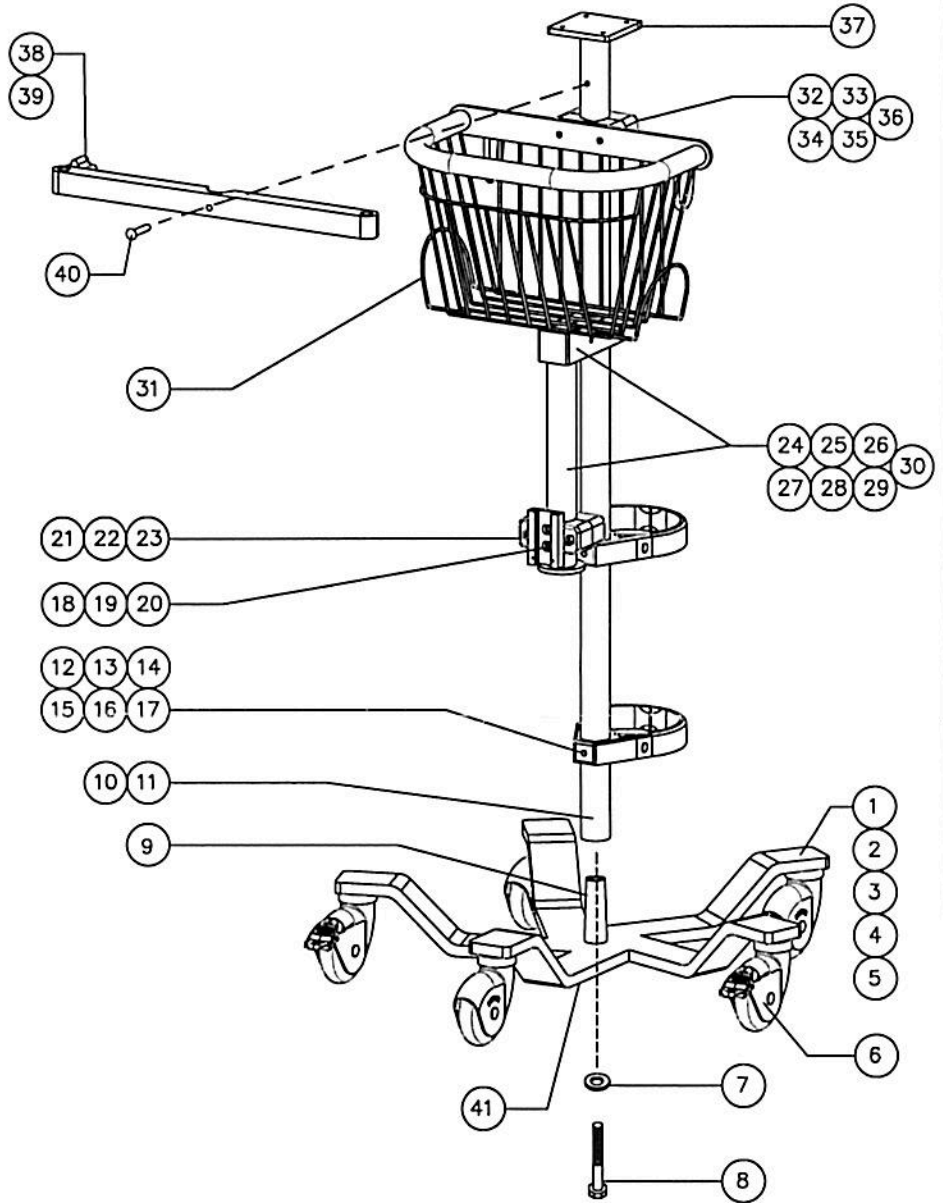

# ASSEMBLY INSTRUCTIONS

## CAT.NO. RD-0725

PARTS LIST			
ITEM	DESCRIPTION	QTY	P/N
1	STEEL BASE	1	STL-1005
2	STUD	5	SCR-1258
3	CASTER	3	CAS-1151
4	ESNA NUT	5	SCR-1257
5	WASHER	5	SCR-1341
6	LOCKING CASTER	2	CAS-1163
7	WASHER	1	SCR-1274
8	HEX BOLT	1	SCR-1349
9	TAPERED PIN	1	STL-1032
10	BASE POLE	1	TUB-1276
11	BUSHING	1	PLS-1434
12	SCREW	2	SCR-1265
13	TRUNCATED OXYGEN RING	2	PLS-1417
14	SCREW	4	SCR-1357
15	TRI GLIDE	2	MIS-1509
16	BUCKLE	2	MIS-1512
17	STRAPS	2 SET	MIS-1495
18	HUMIDIFIER BRACKET	1	MIS-1620
19	ROLL PIN	2	SCR-1585
20	SCREW	2	SCR-1609
21	MOUNT	1 SET	PLS-1496M
22	SCREW	2	SCR-1610
23	SQ NUT	2	SCR-1256
24	POST MOUNT	1	PLS-1495
25	SPACER	2	SCR-1318
26	SCREW	2	SCR-1271
27	TUBE END CAP	1	MIS-1591
28	TUBE	1	TUB-1274
29	STOP WASHER	1	SCR-1509
30	SCREW	1	SCR-1253
31	LG BASKET	1	MIS-1574
32	MOUNT	1 SET	PLS-1496
33	SQ NUT	2	SCR-1256
34	CAP NUT	2	SCR-1372
35	WASHER	2	SCR-1402
36	SCREW	2	SCR-1590
37	MOUNTING PLATE	1	STL-1524
38	AIR CIRCUIT ARM MOUNT	1	ALM-1359
39	KNOB	1	KNB-1331M
40	SCREW	1	SCR-1352
41	LABEL	1	LBL-1526
NOT SHOWN PACKAGING	HEX KEY	1	SCR-1260
	HEX KEY	1	SCR-1320
	WRENCH	1	SCR-1613
	CARTON	1	PAC-1530
	INSERT	1	PAC-1535
	BASE BOX	1	PAC-1536
	BASKET BOX	1	PAC-1537
STYROFOAM INSERTS	2	PAC-1538	
WEINAMAN DFU	1	EV89100320	
INSTRUCTION SHEET	1	LBL-3113	



**ASSEMBLY INSTRUCTIONS:**

TOOLS REQUIRED: • HEX KEY (SUPPLIED)  
WRENCH

CLEANING: SOLUTIONS USED IN CONJUNCTION WITH THIS PRODUCT MAY BE CORROSIVE. IT IS RECOMMENDED TO WIPE DOWN STAND ONCE A WEEK MORE OFTEN IF REQUIRED WITH A DAMP TOWEL AND A MILD CLEANING SOLUTION. NOTE: IF SOLUTION SPILLS ON BASE, CLEAN IMMEDIATELY.

EC REP


EMERGO EUROPE Molenstraat 15 2513 BH, The Hague The Netherlands

Pryor Products 1819 Peacock Blvd., \* Oceanside, California 92056 USA \* Telephone:(760) 724-8244 \* Toll Free:(800) 854-2280 \* Fax:(760) 724-0944



**ASSEMBLY INSTRUCTIONS**  
**CAT.NO. RD-0725**

Platte/plate

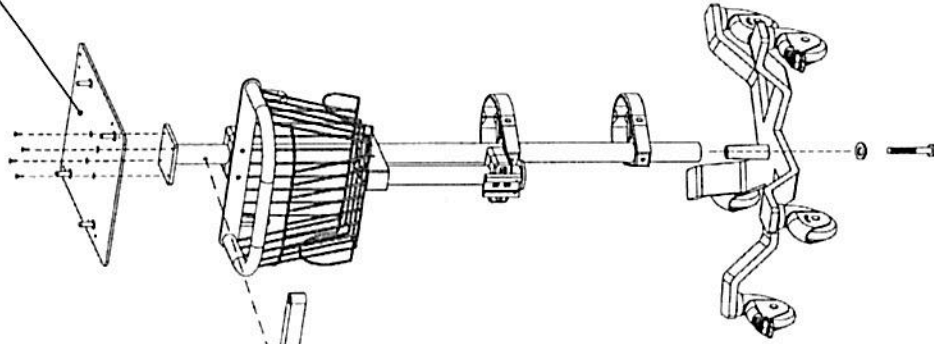

passend zu  
WM 27858/  
matches  
WM 27858

Erledigungsvorschriften:

- RoHS konform
- frei von Verunreinigungen, sowie herstellbedingten Hilfsstoffen
- einzeln im Karton verpackt
- Montageanleitung und Pos.10 beigelegt
- erfüllt IEC 60601-1

Manufacturing instructions:

- RoHS compliant
- free of contamination, impurities and additives used for manufacturing
- individually packed in a carton
- assembly instruction and item 10 enclosed
- complied IEC 60601-1


Index/Änderung	Prüfung nach DIN ISO 2859-1	Freimaßtoleranz nach	Werkstoff	Name	Datum
	Prüfniveau	ISO 2768-mK		Erstellt auf	
Werkstücknamen	ADL		Benennung	Fahrgestell/Irrolley	Maßstab
	Datum	Name			
	gez. 01.11.14	JK			
	gepr. 20.11.14	mei			
	Norm.				
WEINMANN			Zeichnungsnummer	27859	Blatt
			Ersatz f.	Ersetzt d.	1/2

**EMERGO EUROPE** Malenstraat 15 2517 BH, The Hague The Netherlands Tel +31 (0)70 345 8570 Fax +31 (0)70 346 2999  
**Pryor Products** 1819 Peacock Blvd. • Geenside, California 92056 USA • Telephone (760) 724-8244 • Toll Free (800) 854-2780 • Fax (760) 724-0344

Alle Maße in mm / All dimensions in mm  
 Für diese techn. Unterlage behalten wir uns alle Rechte vor. / All rights reserved.

ITEM	DESCRIPTION	QTY
1	Plate	1
2	Knurled screw (1)	4
3	Screw *	4
4	Knurled screw (2)	2
5	Stud	4
6	Washer	4


\* Tools required : Pozidrive screwdriver

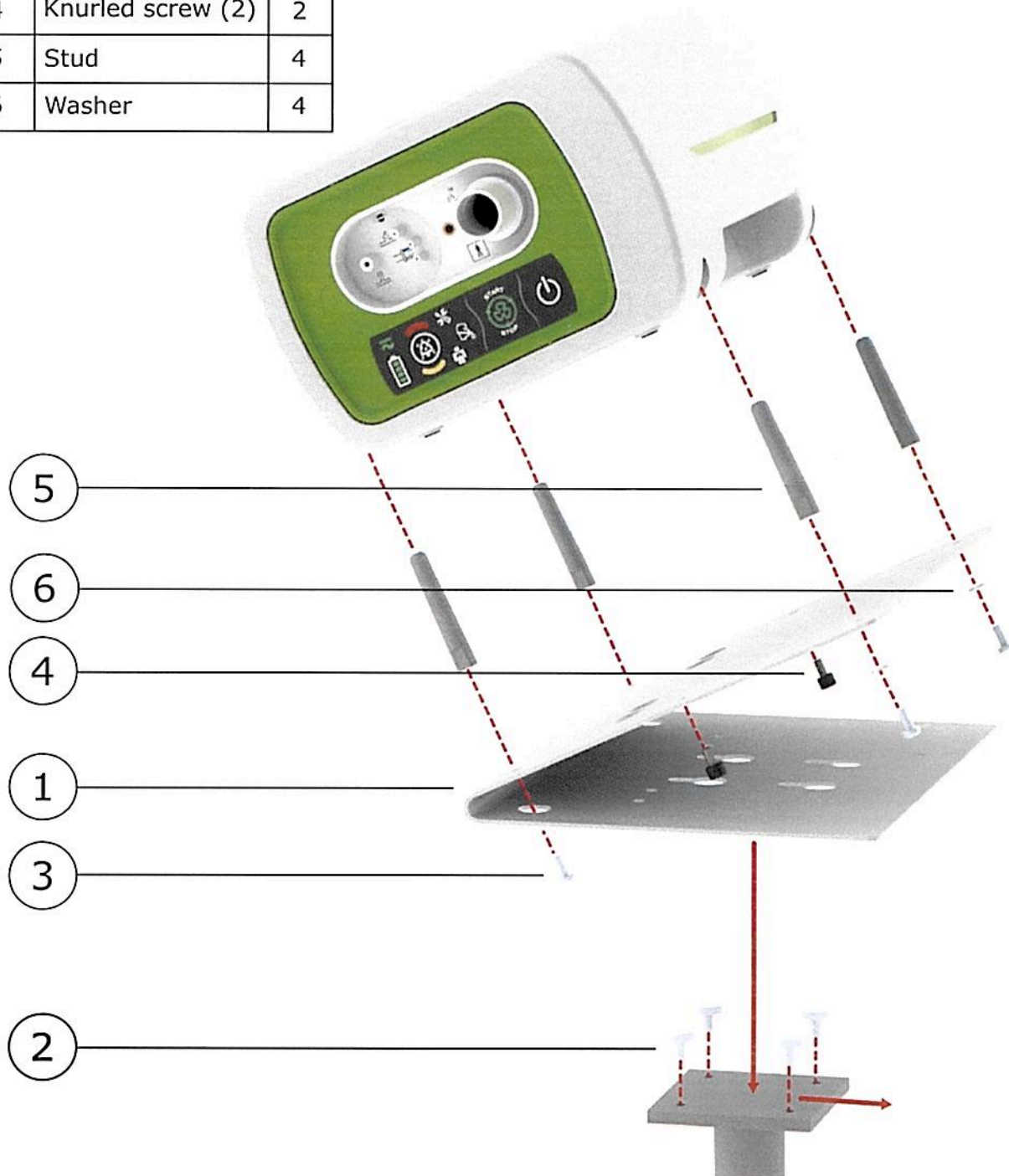



### ASSEMBLY INSTRUCTIONS

PG. 1 OF 2

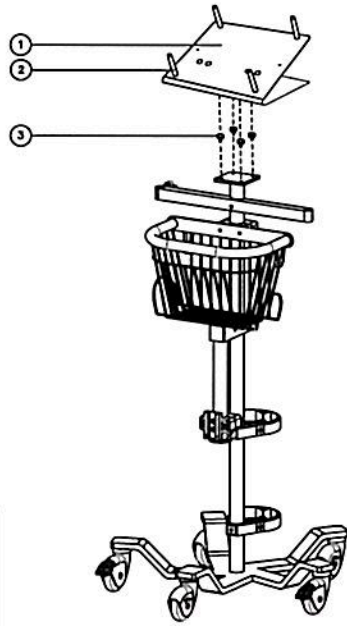

ITEM	DESCRIPTION	QTY
1	Plate	1
2	Screw * + washer + Stud	4
3	Knurled screw (1)	4
4	Knurled screw (2) **	2

\* Tools required : Pozidrive screwdriver

\*\* Not shown - Used to lock the EO-150 on the plate

**WARNING:** SOLUTIONS USED IN CONJUNCTION WITH THIS PRODUCT MAY BE CORROSIVE. IT IS RECOMMENDED TO Wipe down STAND ONCE A WEEK. MORE OFTEN IF REQUIRED WITH A DAMP TOWEL AND A MILD CLEANING SOLUTION. IN CASE OF SOLUTION SPILLS ON BASE, CLEAN IMMEDIATELY.

PRYOR PRODUCTS B.V. Heterenstraat 25 8253 BZ, The Hague The Netherlands


PRYOR PRODUCTS 1818 Peacock Blvd., \* Oxnard, California 93208 USA \* Telephone: (760) 724-8244 \* Toll Free: (800) 854-2280 \* Fax: (760) 724-0944

UL-314 REV B CE



### ASSEMBLY INSTRUCTIONS

PG. 2 OF 2

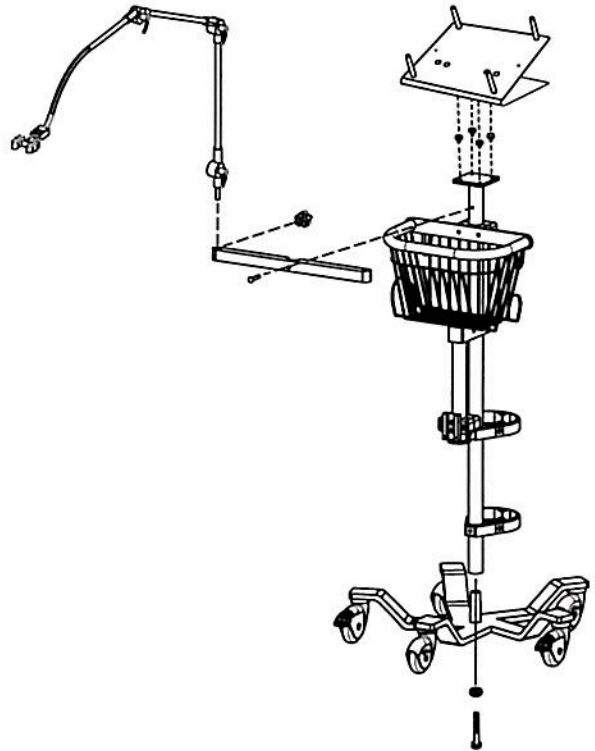

PRYOR PRODUCTS B.V. Heterenstraat 25 8253 BZ, The Hague The Netherlands

PRYOR PRODUCTS 1818 Peacock Blvd., \* Oxnard, California 93208 USA \* Telephone: (760) 724-8244 \* Toll Free: (800) 854-2280 \* Fax: (760) 724-0944