Summit Oxygen 'Everest' Silicone Nasal Cannula

Product Information:

- Product Reference Code –1200/SP/S/SL ; 1800/SP/S/SL;
1200/SP/C/SL; 1800/SP/C/SL
- Description - 3.5mm silicone nasal cannula
- Tubing length - 1200mm / 1800mm
- Material - translucent silicone
- Non-sterile
- Autoclavable
- For use with medical grade oxygen only

Latex Free

Caution:

- Oxygen in the presence of an ignitable material and an ignition source can cause a fire resulting in severe personal injury or death.

DO NOT SMOKE WHILST USING OXYGEN!

- DO NOT USE near open flames or very hot ignition sources
- Do not store in direct sunlight.
- Do not store in high temperature locations.
- Store in dry conditions.
- Do not use if this packaging has been opened or damaged.

Directions:

- Attach oxygen tubing to the oxygen flow meter.
- Set the oxygen flow as prescribed and place the nasal prongs **briefly underwater** to see that oxygen is flowing properly.
- **If there are no bubbles appearing in the water, do not use the cannula.**
- Insert nasal tips into the nostrils.
- Pass the 2 silicone tubes over the ears and under the chin. Use the toggle to adjust the cannula so that it fits snugly.

Hygiene: Product suitable for autoclaving. In absence of this facility follow the instructions below

- To disinfect, place for 20 minutes in boiling distilled water at least every 2 weeks or more often.
- Dry by flowing oxygen through the cannula.
- Do not use the cannula if water is still in the tubing.
- To remove surface dirt, wipe the cannula with a damp cloth containing a very weak solution of detergent.

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Website: www.summitoxygen.com
e-mail: enquiries@summitoxygen.com

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Manufacturer:

**fortune
medical**

Manufacturer FORTUNE MEDICAL INSTRUMENT CORP.
ADD 6Fl., No. 29, Sec. 2, Jhongjheng E. Rd., 251
Danshuei Dist., New Taipei City, Taiwan
TEL 886-2-2624-2233 **FAX** 886-2-2624-2266
Homepage <http://www.fortunemed.com>
E-Mail info@fortunemed.com
EU rep. ADD PRIM, S.A. C/F 15, POL IND. NO. 1, 28938
MOSTOLES, MADRID, SPAIN

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Hergestellt für Summit Oxygen International Ltd durch Fortune MIC.