
HUDSON RCI®

REF 41755

ISO-NEB®

# Filtered Nebulizer System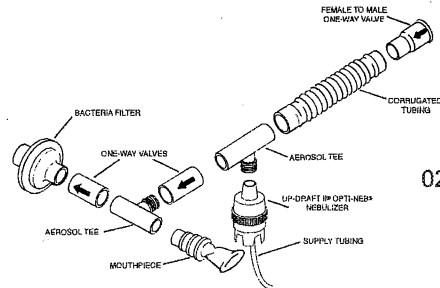

02F1002511

**Teleflex**  
MEDICAL

Research Triangle Park, NC 27709 USA  
919-544-8000 866-246-6990

Made in Mexico

© 2006 Teleflex Incorporated  
83483-18 08/06

EC REP

CE 0120

TFX Medical Ltd.  
High Wycombe HP12 3ST U.K.

## ENGLISH ISO-NEB® Filtered Nebulizer System

### Warning Statements:

- Do not reuse, wash, soak or sterilize. Residuals from medication or cleaning agents retained within the device may result in contamination, dangerously increased breathing resistance and impaired valve function.
- Do not connect directly to an endotracheal or tracheostomy tube.
- Discard after single application.

### Instructions for Use:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

- Remove the ISO-NEB from its package and be sure all components are intact.
- Rotate the nebulizer into upright position for right- or left-handed use.
- Verify that the arrows on one-way valves are oriented correctly: two on the inspiratory limb toward the mouthpiece and one on the expiratory limb toward the particle filter.
- Remove the nebulizer from the tee.
- Add prescribed medication to the nebulizer jar through the opening at the top of the cap (8 cc maximum).
- Reattach the nebulizer to the tee.
- Connect one end of the supply tubing to the fitting under nebulizer and the other end to the source gas outlet.
- Adjust the gas source to 6 to 8 LPM. When using an air compressor, adjust pressure for 23 to 25 psi.
- Proceed with treatment as prescribed. Exhale through the mouthpiece.
- Cover mouthpiece to divert flow toward filter when taking breaks during treatment.

ISO-NEB, UP-DRAFT II, OPTI-NEB, and Hudson RCI are registered trademarks of Hudson Respiratory Care, Inc. in the United States of America and in selected other countries.

Manufacturer

EC REP

Authorized Representative in  
the European Community.

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