**Sunrise Medical**

Respiratory Products Division
100 DeVilbiss Drive
Somerset, Pennsylvania 15501
USA
814-443-4881

Sunrise Medical Pty. Limited

15 Carrington Road, Unit 7
Castle Hill NSW 2154
AUSTRALIA
61-2-9899-3144

Sunrise Medical Benelux

Pascalbaan 3
Nieuwegein 3439 MP
NETHERLANDS
31-3060-82-100

Sunrise Medical Canada, Inc.

237 Romina Drive, Unit 3
Concord, Ontario L4K 4V3
CANADA
905-660-2459

Sunrise Medical

13 Rue de la Painguetterie
37390 Chanceaux / Choissille
FRANCE
33-247-55-44-00

Sunrise Medical

Rehabsenteret
1450 Nesoddtangen
NORWAY
47-66-96-38-00

Sunrise Medical Ltd.

Sunrise Business Park
High Street
Wollaston, West Midlands DY8 4PS
ENGLAND
44-138-444-6688

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Printed in the USA
on recycled paper.

©08.05 Sunrise Medical
A-808 Rev. C

DEVILBISS®

**DeVilbiss® Reusable Jet Nebulizer Model 800D/800G/800C**

CAUTION—Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
Made in USA

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INTENDED USE

The DeVilbiss Model 800 is intended for use in conjunction with a compressed gas source such as the DeVilbiss aerosol compressors to aerosolize physician prescribed liquid medication. The system is used on the order of a physician for the treatment of asthma, bronchitis and other respiratory diseases. The nebulizer converts the liquid medication into a fine aerosol mist which can be inhaled for home respiratory therapy. This allows the medication to be deposited at the affected areas of the patient's airway.

WARNING

This product contains small parts that may present a choking hazard to small children. The tubing may also present a strangulation hazard. Always use close adult supervision when administering a treatment to a child. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENTS.

NOTE– Clean all parts before first use by following the cleaning instructions.

NOTE– Each reusable pneumatic jet nebulizer for single-patient use can be expected to last up to 12 months depending on how often it is used and cleaned.

DIRECTIONS FOR USE

1. Verify that nebulizer is assembled correctly by inserting the baffle (3) into the nebulizer medicine cup (4).
2. Holding the medicine cup stationary, attach the nebulizer cap (2) to the medicine cup (4) by inserting and turning clockwise.
3. Add prescribed medication into the nebulizer through the large opening in the top of the nebulizer cap (2).
4. Attach mouthpiece (1) to the nebulizer cap (2) by aligning and pushing gently straight down.

NOTE– If you are using a standard mask (with a 22mm fitting) remove the nebulizer mouthpiece (1) from the nebulizer cap (2); then place the mask adapter into the cap opening and proceed with treatment.

5. If you are using an interrupter adapter (5) (as in the 800G model), then connect adapter to nebulizer by sliding coordinating tabs into corresponding position.

6. Attach one end of the supplied tubing to the air inlet connector on the base of the nebulizer. Attach the other end of the tubing to a suitable compressed air source.
7. Please note that medication may collect on the sides of the nebulizer during the treatment. If this occurs, gently tap the nebulizer and the medication will drop back into the medicine cup to be nebulized.

DIRECTIONS FOR CLEANING

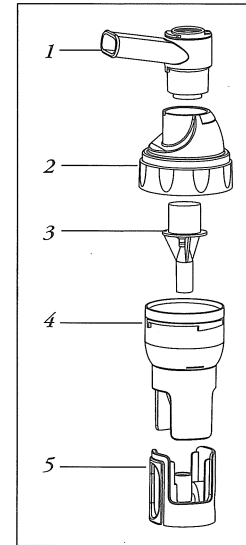
AFTER EVERY USE:

1. Remove nebulizer mouthpiece (1) from nebulizer cap (2) by grasping firmly and pulling straight up. Do not twist to remove.
2. Remove nebulizer cap (2) by turning counterclockwise.
3. Remove baffle (3) by pulling straight up from nebulizer medicine cup (4).
4. Remove interrupter adapter (5) if applicable by pulling straight out from bottom of medicine cup (4).
5. Wash all parts in a solution of hot water and dishwashing liquid and rinse under tap water. Allow to air dry.

DAILY:

Soak parts in a solution of one part distilled white vinegar to three parts hot water and rinse OR clean daily in dishwasher using top shelf only.

NOTE— Before each use, check parts for cracks or discoloration. If parts show any damage, please discard and contact your provider for a replacement.

SPECIFICATIONS

Minimum recommended fill volume	2cc
Maximum recommended fill volume	6cc
Minimum Driving Gas Flow	5 LPM @ 14 PSIG
Recommended Driving Gas Flow	5.5 LPM @ 16 PSIG
Maximum Driving Gas Flow	8 LPM @ 28 PSIG
MMAD* Range	3.6 – 4.0 Microns
Aerosol Output By Weight	0.3 ml/minute
Aerosol Output*	0.20 g/minute
Respirable Output*	0.12 g/minute
Warranty	1 year limited

*Measured according to British Standards Institute (BSI) standard # BS 7711 Part 3.

ACCESSORIES FOR USE WITH THE DEVILBISS MODEL 800

PAM10	Pediatric Mask/package of 50
DL1050	Dragon Mask (pediatric)/package of 50
AAM12	Adult Mask/package of 12

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DECLARATION OF CONFORMITY

Manufacturer:	Sunrise Medical
Address:	Respiratory Products Division 100 DeVilbiss Drive Somerset, PA 15501 U.S.A.
Product Name:	DeVilbiss® Reusable Nebulizer
Model Number:	800 Series

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

Class:	IIa, Rule 5
Quality System Standards Applied:	ISO9001/ISO13485
Notified Body RWTÜV MDD Annex II Applied	
Safety:	BS 7711.Part3.1994
Authorized Representative:	Sunrise Medical Ltd. Sunrise Business Park High Street Wollaston, West Midlands DY8 4PS ENGLAND 44-138-444-6688

® Registered U.S. Patent and Trademark
Office and other countries.

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