FitLife | 

**PHILIPS**  
  
**RESPIRONICS**

# FitLife

EN	001	INSTRUCTIONS FOR USE	HU	061	HASZNÁLATI UTASÍTÁS
FR	005	MODE D'EMPLOI	SK	065	NÁVOD NA POUŽITIE
IT	009	ISTRUZIONI PER L'USO	SL	069	NAVODILA ZA UPORABO
DE	013	GEBRAUCHSANWEISUNG	LV	073	LIETOŠANAS INSTRUKCIJA
ES	017	INSTRUCCIONES DE USO	LT	077	NAUDOJIMO INSTRUKCIJA
PT	021	INSTRUÇÕES DE UTILIZAÇÃO	ET	081	KASUTUSJUHISED
PT-BR	025	INSTRUÇÕES DE USO	RO	085	INSTRUCȚIUNI DE UTILIZARE
EL	029	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	BG	089	ИНСТРУКЦИИ ЗА УПОТРЕБА
DA	033	BRUGSANVISNING	HR	093	UPUTE ZA UPORABU
NL	037	GEBRUIKSAANWIJZING	ZH-CN	097	使用说明
SV	041	BRUKSANVISNING	ZH-TW	101	使用說明
FI	045	KÄYTTÖOHJEET	RU	105	ИНСТРУКЦИИ ПО ЭКСПЛУАТАЦИИ
NO	049	BRUKSANVISNING	SR	109	UPUTSTVO ZA UPOTREBU
PL	053	INSTRUKCJA UŻYTKOWANIA	TR	113	KULLANIM TALİMATLARI
CS	057	NÁVOD K POUŽITÍ	KO	117	사용 지침

# Instructions for Use

## FitLife Total Face Mask

ENGLISH

### Intended Use

**Small Size Intended Use:** The small size of the FitLife Total Face Mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients 7 years or older (>40 lbs/20 kg) for whom CPAP or bi-level therapy has been prescribed.

**Large Size Intended Use:** The large size of the FitLife Total Face Mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. This mask is for single patient use in the home or multi-patient use in the hospital/institutional environment. The mask is to be used on patients (>66 lbs/30 kg) for whom CPAP or bi-level therapy has been prescribed.

**Important Note:** An exhalation port is built into the FitLife Total Face Mask elbow so a separate exhalation port is not required.

### Symbols

Warning or  
Caution

Note

Tip

Consult instructions  
for use

X1 System One Resistance  
Control Setting

Does not contain  
natural rubber latex

Does not contain DEHP  
(bis (2-ethylhexyl)phthalate)

Contains PHT (phthalates)

### ⚠️ Warnings:

- This mask is not suitable for providing life support ventilation.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port. Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. Explanation of the Warning: When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- For children and those who are pregnant or breast feeding: This product may contain chemicals (phthalates) which may cause adverse health effects. Healthcare professionals should discuss the implications of using this product.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.
- The patient's physician should be contacted if the patient experiences the following symptoms while using the mask or after removing it: Unusual chest discomfort, shortness of breath, stomach distension, belching, or sever headache; drying of the eyes, eye pain, or eye infections; blurred vision. (Consult an ophthalmologist if symptoms persist.)
- At low CPAP or EPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- A minimum of 4cm H<sub>2</sub>O must be maintained when using this mask.
- This mask should not be used for patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask by themselves.
- This mask is not recommended for patients who are taking a prescription drug that may cause vomiting.

- Appropriate patient monitoring should be used as medically necessary.
- The mask contains small parts which could result in a choking hazard.
- If an additional exhalation device is added to the patient circuit, you may need to adjust the pressure level to compensate for the additional leak of the exhalation device.
- Hand wash prior to first use. Inspect the mask for damage or wear (cracking, crazing, tears, etc.). Discard and replace any components as necessary.
- Use of a nasal or full face mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. Consult your physician or dentist if symptoms occur.

- This mask may not be suitable for persons with the following conditions: glaucoma, recent eye surgery or dry eyes, impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if the patient is unable to remove the mask by themselves.
- Hand wash the mask.
- Wash your face.
- Inspect the mask components for damage or wear; replace if necessary.
- Verify that the entrainment valve functions correctly.

#### Contraindications

This mask may not be suitable for persons with the following conditions: glaucoma, recent eye surgery or dry eyes, impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if the patient is unable to remove the mask by themselves.

#### Before Use

- Read and understand the instructions completely.
- Hand wash the mask.
- Wash your face.
- Inspect the mask components for damage or wear; replace if necessary.
- Verify that the entrainment valve functions correctly.

#### Entrainment Valve

The entrainment valve consists of an air inlet and a flapper. With the airflow turned off, verify that the flapper is lying flat so that room air can flow in and out through the air inlet in the valve (Figure 2A). Next, with the airflow on, the flapper should now cover the air inlet and air from the CPAP or bi-level device should flow into the mask (Figure 2B). If the flapper does not close or does not function properly, replace the mask.

**⚠ Warning:** Do not block or try to seal the entrainment valve or exhalation ports.

#### Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask.

1. Hand wash the mask and headgear in warm water with a mild liquid dish washing detergent.

**⚠ Caution:** Do not use bleach, alcohol, or cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

2. Rinse thoroughly with drinking quality water and air dry out of direct sunlight. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.
3. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc.). Replace the mask if the cushion becomes hardened.

For **multi-patient use** in the hospital/institutional environment, use the Mask Cleaning and Disinfection Instructions to reprocess the mask between patients. These instructions can be obtained by visiting [www.sleepapnea.com](http://www.sleepapnea.com) or contacting Resironics Customer Service at 1-724-387-4000 or Resironics Germany at +49 8152 93060.

#### Dishwasher Instructions

**⚠ Caution:** Use a mild liquid dishwashing detergent only

1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.
4. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc.). Replace the mask if the cushion becomes hardened.

#### System One Resistance Control

This mask uses the System One Resistance Control setting  X 1. This symbol may appear on your therapy device. This symbol represents the level of mask resistance compensation. Match the setting to the mask if appropriate.

#### Important Notes:

- Verify the System One setting if the mask or mask cushion changes.
- The System One setting is not compatible with masks requiring a separate/additional exhalation device.

Figure 1

Figure 1  
A = Top Strap  
B = Side Strap  
C = Bottom Strap  
D = Headgear Clip  
E = Elbow with Entrainment Vents and Entrainment Valve  
F = Faceplate  
G = Cushion  
H = Accessory Swivel

#### Putting on the Mask

1. Loosen the side headgear straps by peeling back the tabs and open the straps to a larger size.
2. Disconnect one or both of the bottom headgear clips. Grasp the clip with the thumb and index finger and in a turning motion, lift the clip upwards with the thumb while pulling the clip away from the mask faceplate.
3. Hold the mask against your face. The top of the cushion will contact your face just above the eyebrows. The bottom of the cushion will contact your face just above the chin. Be sure your mouth remains slightly open.
4. Pull the headgear over your head.
5. Attach one or both bottom headgear clips by finding the tab on the lower part of the mask faceplate with your thumb (Figure 3). Guide and gently push the headgear clip onto the tab with your middle and index fingers until it clicks into place.

#### Achieving the Right Fit

1. Tighten the side headgear straps. The top strap should lie at the crown of the head. The cross-strap should sit low on the back of the head.
2. Tighten the bottom headgear straps.
3. Adjust the top headgear strap, if necessary. If the side headgear straps are resting on your ears, adjust the top headgear strap to lift the straps slightly off of your ears. DO NOT over tighten as it may cause the headgear to slip.
4. Connect the accessory swivel to the mask elbow.
5. Connect the tubing that is included with your CPAP or bi-level device to the accessory swivel.
6. With the airflow on, lie down and breathe normally with your mouth slightly open. Re-seat the mask by pulling it directly away from your face and gently setting it back into place.
7. Assume different sleeping positions and move your head around.
8. If there are any air leaks, make any final adjustments. Some air leakage is normal.

Figure 2

- Comfort Tips**
- Re-seat the mask. Pull the mask by the sides of the faceplate directly away from the face then gently set it back into place. DO NOT pull the mask by the exhalation elbow. This allows the cushion to create a new seal on the face.
  - Adjust the side headgear straps to eliminate leaks around the forehead and temples.
  - Adjust the bottom headgear straps to eliminate leaks around the cheeks and chin.
  - The most common mistake is over tightening the headgear. The headgear should fit loose and comfortable. If your skin bulges around the mask or if you see red marks on your face, loosen the headgear.

## Removing the Mask

Undo one or both bottom headgear clips and pull over the top of your head (Figure 4). By undoing the headgear clips, instead of the tabs, you will preserve your adjustments.

## Replacing the Mask Elbow

**To remove:** Hold the faceplate swivel and gently pull the elbow from the faceplate swivel (Figure 5).

**To replace:** Press the mask elbow into the faceplate swivel. Do not use excessive force or any tools.

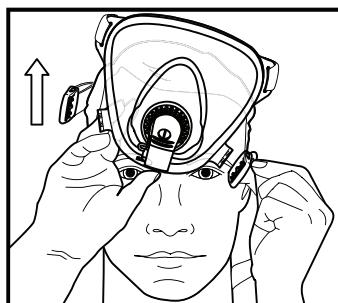
Figure 4


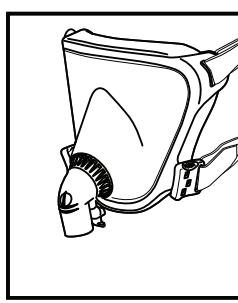
Figure 5

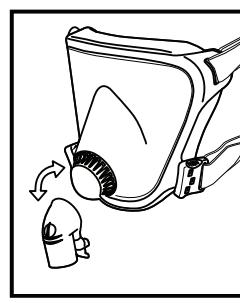
## Disposal

Dispose of in accordance with local regulations.

## Specifications

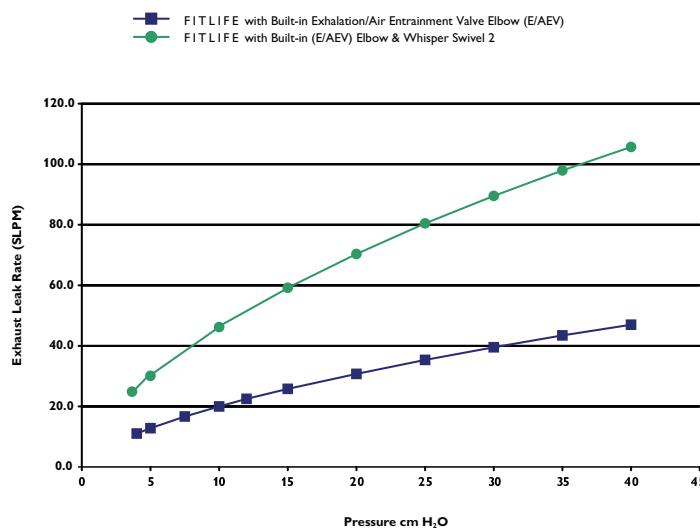**⚠ Warning:** The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

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© 2010 Koninklijke Philips Electronics N.V. All rights reserved.
Respironics Inc.