Permox® SilentCare

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This "Instructions for Use" manual is published for public use and is only meant for your information.

We do not guarantee the accuracy of these instructions with respect to your specific device type or status.

Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

PermoX[®] SilentCare

Oxygen Concentrator Instructions for Use

Contents

For Your Safety	17	What's what	26
Safety instructions.....	17	Front view.....	26
Intended Use	17	Rear view.....	26
Before Use	18	Control panel.....	27
Preparation	19	Portable humidifier unit.....	27
Positioning the unit.....	19	Connection of hoses and cables.....	27
Connecting cables and hoses.....	19	Technical Data	28
Preparing the humidifier unit.....	20	Functional Description	29
Positioning the humidifier unit.....	20	Order List	29
Operation	21		
Points to note before switching on the unit.....	21		
Switching on the unit.....	21		
Setting oxygen flow.....	21		
Fitting the nasal cannula.....	22		
Operation.....	22		
Shut-down.....	22		
Care	23		
At home.....	23		
In the hospital.....	23		
Maintenance	24		
Maintenance intervals.....	24		
Disposal of the unit.....	24		
Fault – Cause – Remedy	25		

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

For Your Safety

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions.

The apparatus is only to be used for purposes specified in the chapter "Intended Use".

Maintenance

The unit may only be serviced by service personnel from Dräger Medical or by service personnel authorized by Dräger Medical.

See chapter "Maintenance", page 24

Accessories

Do not use accessory parts other than those in the order list.

Liability for proper function or damage

The liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by Dräger Medical or if the apparatus is used in a manner not conforming to its intended use.

Dräger Medical cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Dräger Medical are likewise not modified by the recommendations given above.

Dräger Medical b.v.

Safety instructions

Oxygen promotes combustion!

When using the unit and for 15 minutes after use:

No smoking, no naked lights, such as burning matches, candles, open fires. Fire hazard.

The unit and its hoses must be kept free from oil and grease. Fire hazard.

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

The apparatus must not be allowed to come into contact with explosive gas mixtures. Explosion hazard.

Do not use any humidifiers or other accessories not listed in this manual, otherwise the correct functioning of the unit and the therapy may be jeopardized.

Intended Use

PermoX SilentCare – for oxygen enrichment of the breathing air for long-term oxygen therapy at home and in hospitals.

Only for patients whose vital functions are not directly and constantly dependent on a higher oxygen concentration in the breathing air.

In the case of patients who constantly require an elevated oxygen concentration, PermoX SilentCare may only be used in combination with an emergency supply, e.g. oxygen cylinders. Constant oxygen monitoring with the aid of a suitable measuring instrument is essential for such patients.

Only for use as directed by a physician

Oxygen for medical purposes is a highly efficient medicament. Incorrect use may produce major side effects. The doctor's instructions must therefore be followed strictly; the doctor's settings on the unit must not be changed. All changes in the patient's well-being must be reported to the attending physician without delay.

Before Use

Handling the unit

PermoX SilentCare has a telescopic handle and large wheels permitting easy transport without having to carry the unit.

The telescopic handle can be set to two heights.

To set the required height:

- 1 Press the button and set the telescopic handle to one of the two heights.
- Tilt the unit backwards by pressing against the handle: PermoX SilentCare can now be transported without difficulty.

Pull the unit slowly over door thresholds in order to avoid damage.

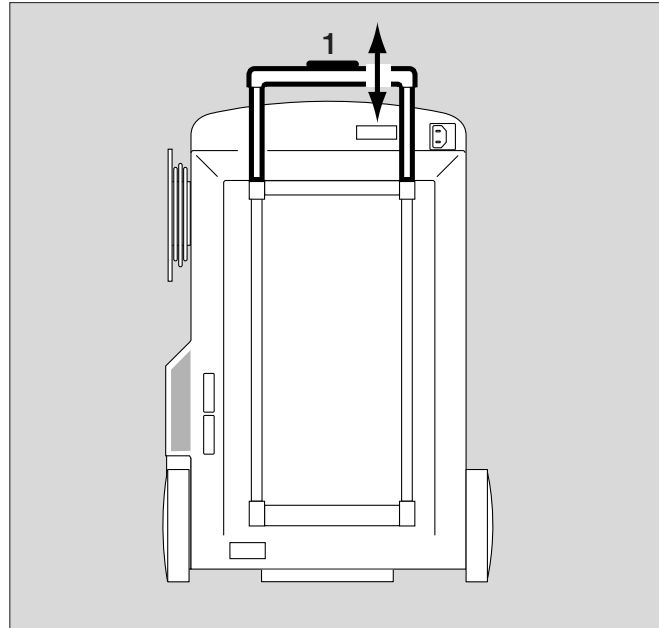
Do not pull the unit up steps and stairs, otherwise it may be damaged.

Lifting the unit

PermoX SilentCare weighs approx. 25 kg. It should not be lifted at all if possible.

If the unit must be lifted:

- 1 Press the button and push the telescopic handle completely down.


Use both hands to lift the unit:

- 1 grasp the telescopic handle **and**
- 2 the additional handle underneath the unit
– keep an upright posture to avoid back injuries.

If the unit must be operated in an elevated position, it must be placed on a platform and secured to prevent it rolling off.

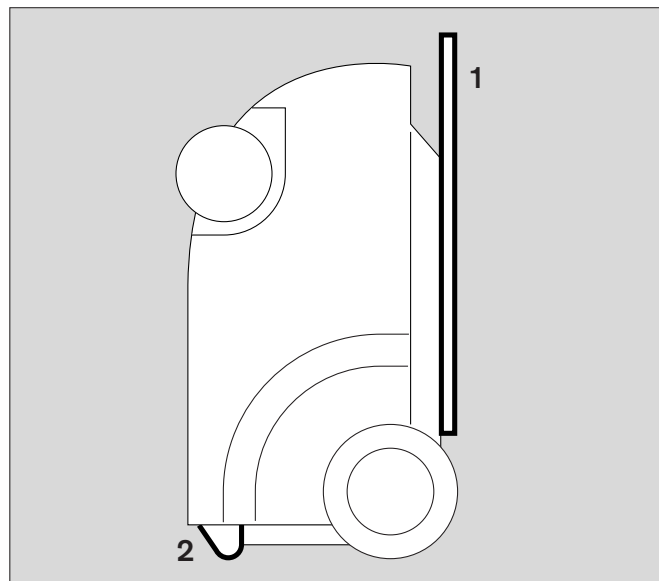The unit must always be transported and positioned upright, otherwise faults or defects may arise.

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Preparation

Step 1

Positioning the unit

The portable humidifier unit enables a large range of action without having to move the concentrator to another position. The following points must be noted to ensure an optimum supply of oxygen:

Place the unit in a well ventilated, dry, cool and dust-free room.

Avoid smoke and pollutants. This may reduce the oxygen concentration.

A distance of at least 25 cm must be maintained between the unit and any walls or furniture, otherwise it may overheat and become defective.

Do not place the unit close to a radiator or in direct sunlight. It may overheat and become defective.

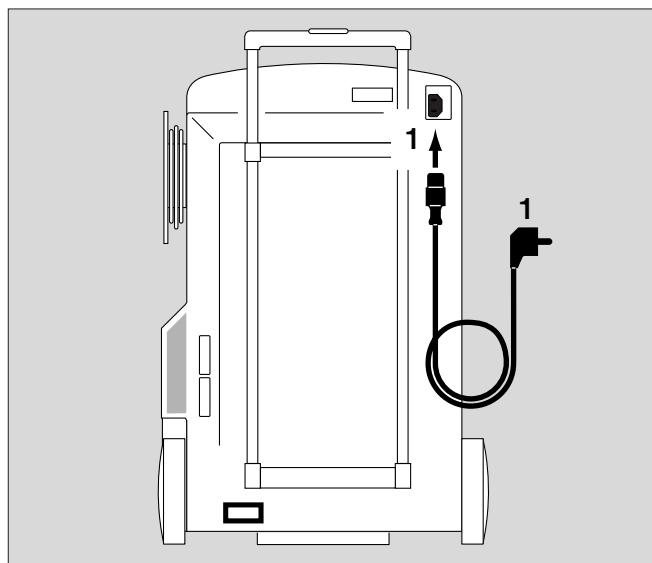
Do not place the unit in front of heat-sensitive objects. The air emerging from the vent on the rear of the unit has a temperature of 50 to 60 °C. Heat-sensitive objects may be damaged.

Step 2

Connecting cables and hoses

Check that the mains voltage corresponds to the voltage specified on the rating plate on the rear of the unit. Faults may arise if the unit is operated on a different voltage.


- 1 Plug the mains cable into the appliance socket on the rear of the unit and into the wall socket.

Step 3

Preparing the humidifier unit

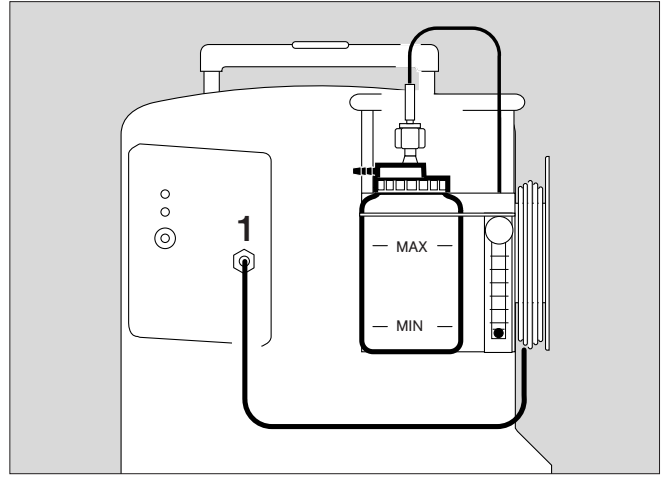- 1 Plug the extension hose into the oxygen outlet.
- Remove the humidifier bottle from the humidifier unit.


- 2 Unscrew the cap on the humidifier bottle.
- 3 Fill the humidifier bottle with distilled, sterilized or demineralized water – up to the mark.

Do not use tap water – risk of infection.

- 2 Screw the cap back onto the humidifier bottle.
- Place the humidifier bottle in the humidifier unit.

The humidifier bottle must always be placed in the humidifier unit. Ensure that the humidifier unit cannot topple over otherwise water may enter the hose and cause the unit to malfunction.

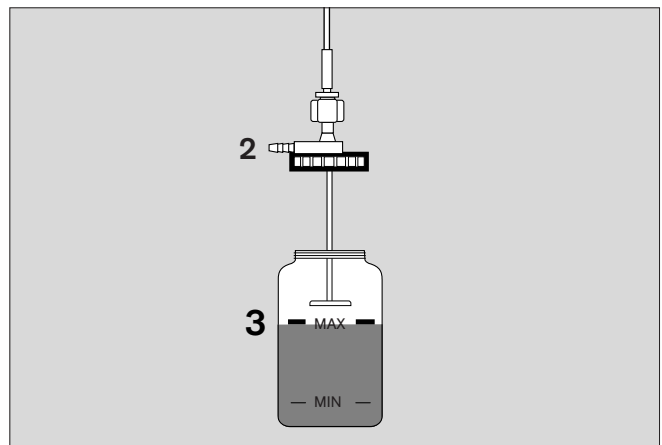The water level must not be allowed to drop below the minimum mark – the bottle must be refilled in good time.


Step 4

Positioning the humidifier unit

The humidifier unit can be detached and positioned wherever it is needed.

- Remove the humidifier unit from the Permax SilentCare,
- 4 unroll the extension hose from the reel and position the humidifier unit where it is needed.

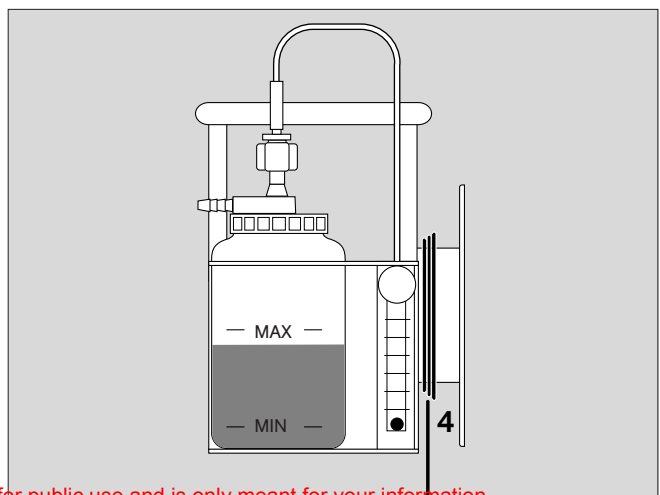Ensure that the hose is neither kinked nor crushed, otherwise the flow of oxygen will be interrupted.

This "Instructions for Use" manual is published for public use and is only meant for your information. We do not guarantee the accuracy of these instructions with respect to your specific device type or status. Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Operation

Points to note before switching on the unit

The unit must be positioned so that the user can always hear or see the alarms generated by the unit and react accordingly if a fault develops.

Extinguish all naked lights, e.g. cigarettes, candles, etc. – major fire hazard.

The unit must be allowed to warm up to room temperature if it has been stored at temperatures of less than 10 °C, otherwise it may malfunction.

The unit must not be operated at ambient temperatures of more than 40 °C, otherwise it may overheat and become defective.

Step 5

Switching on the unit

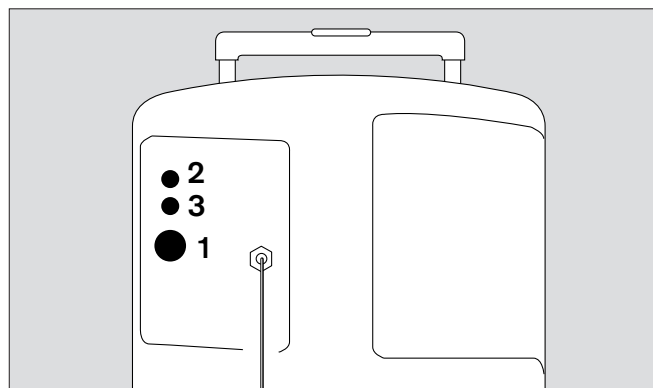
- 1 Press the ON/OFF button on the front panel until it engages. The unit is now switched on.
- 2 The red alarm lamp lights up briefly and a short alarm tone sounds.
- 3 The green lamp lights up.
The unit starts after approx. 10 seconds.

A continuous alarm tone will sound if the PermoX SilentCare has been switched on without plugging in the mains cable.

Then:

- Plug in the mains cable, the alarm tone ceases.

A steady oxygen concentration is output by the unit approx. 10 minutes after switching on.


Step 6

Setting oxygen flow

- 4 Set the oxygen flow prescribed by the doctor via the rotary knob on the humidifier unit.
- 5 Read off the flow on the flowmeter, in the middle of the ball.

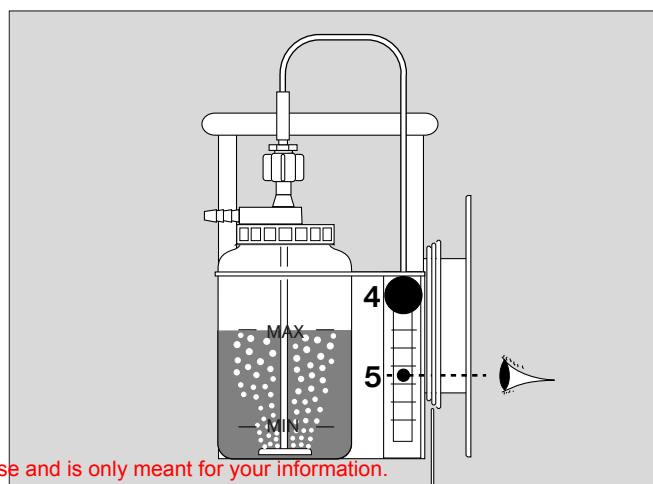The flow reading must be checked regularly during operation – so that incorrect settings can be corrected.

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Step 7

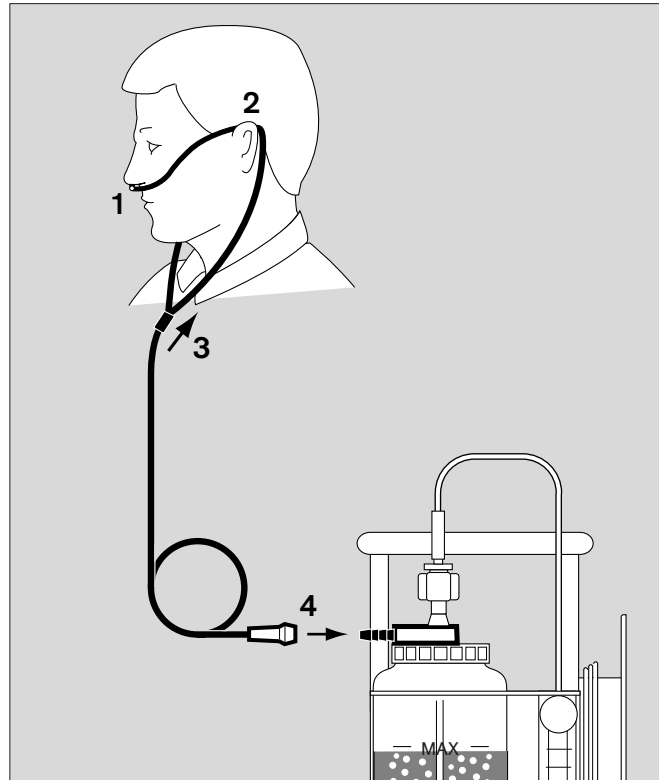
Fitting the nasal cannula

- 1 Insert the two short nozzles into the nasal passages.
- 2 Lead the hoses over the ears.
- 3 Adjust the sleeve until the nasal cannula fits securely.
- 4 Connect the end of the hose to the humidifier.

Check that all hoses have been connected correctly.

Ensure that the hoses are not kinked, otherwise oxygen cannot flow through.

Ensure that the nozzles are correctly positioned in the nasal passages during therapy. The required medical effect cannot be achieved if the nozzles slip out of place.

Step 8

Operation

The flow of oxygen is indicated on the flowmeter. Oxygen tangibly flows from the nozzles and bubbles can be seen in the water in the humidifier bottle.

Duration of therapy must be maintained as directed by the doctor, otherwise the required medical effect cannot be achieved.

The control knob for setting the oxygen flow should not be turned frequently from one end to the other, otherwise it may be damaged.

Let the unit run for at least 15 minutes. The unit may be damaged if operated for shorter periods of time.

Step 9

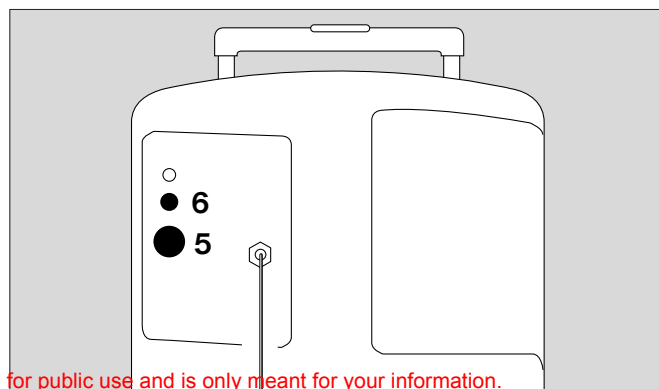
Shut-down

- 5 Press the ON/OFF button completely and then release it.
- 6 The green lamp goes out.

No smoking, no naked lights, e.g. burning matches, candles, open fires. Wait at least 15 minutes after using the unit to avoid all risk of fire.

If the humidifier unit is not needed again on the same day:

Drain and dry the humidifier bottle to avoid infection, see also "Care", page 23.

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Care

At home

Daily:

- Change the water in the humidifier bottle.
- Rinse the humidifier bottle and cap in clear, lukewarm water.
- Rinse the outside of the nasal cannula with clear, lukewarm water, holding the open ends downwards so that water cannot enter the hose.

Always use a clean brush and a clean cloth to avoid all risk of infection.

Do not use washing-up water.

Weekly:

Humidifier:

- Clean the humidifier bottle and cap with a mild detergent and water.
- Rinse the humidifier bottle and cap under running water to remove all soap residues.
- Dry the parts with a clean cloth or leave them to dry in air.
- Wipe the surface of the unit with a clean, damp disposable cloth. Use a mild detergent.

Do not allow any liquid to seep inside the unit or its connections – this may damage the unit.

Monthly:

Dust filter:

- Replace the dust filter every month or when clogged.

In the hospital

Note the hospital hygiene regulations.

Use a new nasal cannula for each patient in order to avoid infection.

Weekly:

Surface of the unit, cables and hoses:

- wipe-disinfect.

Humidifier:

- Use a disinfectant bath for the humidifier bottle and cap.

To ensure material compatibility, use disinfectants based on:

- aldehydes
- quaternary ammonium compounds

Preparations based on the following should not be used, as they may damage the materials.

- compounds releasing halogens
- strong organic acids
- compounds releasing oxygen.

Note the instructions for use of the disinfectant manufacturer in order to avoid damage to the unit and incorrect disinfection.

Do not allow any liquid to seep inside the unit or its connections – this may damage the unit.

Monthly:

Dust filter:

- Replace the dust filter every month or when clogged.

Maintenance

The unit may only be serviced and repaired by service personnel employed or authorized by Dräger Medical.

The unit must be inspected and serviced regularly at yearly intervals or after 5000 hours of operation (and a record kept). Correct functioning must be verified by service experts every 6 months.

A service manual is available to each authorized service expert for service, maintenance and repair purposes.

The unit may only be opened by authorized service experts in order to avoid damage due to incorrect maintenance.

Always unplug the mains cable before opening the unit in order to avoid the risk of electric shock.

Only use original Dräger Medical parts when servicing the unit, otherwise faults or defects may arise.

Maintenance intervals

Required action	Intervals
Inspection and maintenance	By service experts once per year or after 5000 hours of operation.
Correct functioning	Must be checked by service experts every six months.
Dust filter	Must be replaced every month or when clogged. Old filters can be disposed of as household refuse.
Internal intake filter	Must be replaced by service experts every six months.
Internal bacterial filter	Must be replaced by service experts once per year or after 5000 hours of operation.
Compressor	Must be serviced by service experts every 3 years or after 12000 hours of operation.
All inhalation tubes and filters	Must be replaced for each new user or as directed by the doctor.

Disposal of the unit

At the end of its service life:

PermoX SilentCare must be disposed of as required in consultation with the relevant waste management companies.

The applicable regulations must be observed.

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Fault – Cause – Remedy


If a fault arises, follow the recommended step-by-step procedure.

Fault	Cause	Remedy
Unit does not run, continuous acoustic alarm	Unit has been switched on but the mains cable is not plugged in.	1. Check that the mains cable has been connected correctly.
	Fuse has blown.	2. Call a service expert.
Unit runs, red alarm lamp flashes, intermittent acoustic alarm	Oxygen concentration is too low because a hose has been kinked or is not connected correctly.	1. Ensure that all hoses have been connected correctly and are not kinked.
	Oxygen concentration is too low because a hose is fouled.	2. Ensure that all hoses are clean.
	Too little air is taken in because the dust filter is fouled.	3. Clean the dust filter, page 23.
	Oxygen flow has been changed several times over the whole flow range within a very short space of time.	4. Switch off the unit, leave it to cool for approx. 30 minutes and switch on again. Do not change the set flow.
	Device error	5. Call a service expert. The unit can still be used without reservation, but the oxygen concentration may be too low.
Unit does not run, red lamp lights up constantly, continuous acoustic alarm	No oxygen supply because a hose has not been connected.	1. Ensure that all hoses have been connected correctly.
	The unit has overheated because the dust filter is fouled.	2. Clean the dust filter, page 23.
	The unit may have overheated or an internal fault may have occurred because the oxygen flow has repeatedly been changed over the entire range within a very short space of time.	3. Switch off unit, leave it to cool for approx. 30 minutes and switch on again. Do not change the set flow.
	Device error	4. Call a service expert.
Unit runs, no acoustic alarm, no alarm lamp lights up, no oxygen supply	Oxygen is not supplied because a hose has not been connected.	1. Ensure that all hoses and the humidifier unit have been connected correctly.

What's what

Front view

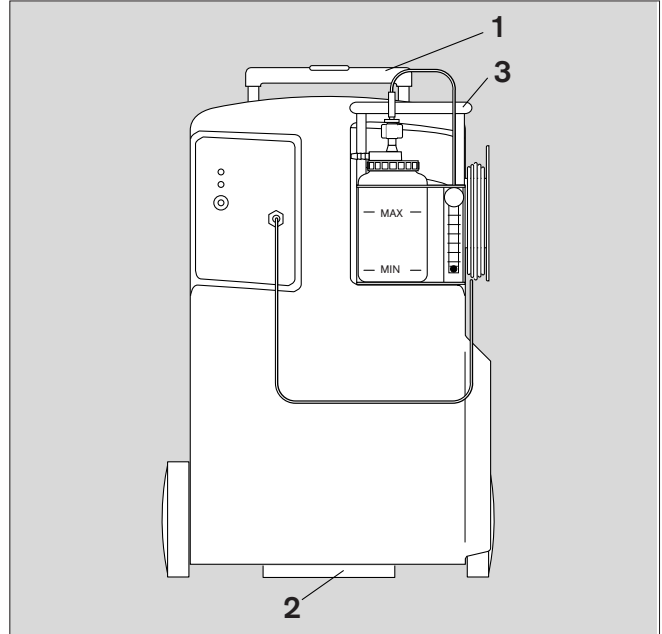
- 1 Telescopic handle
- 2 Second handle for lifting the unit
- 3 Portable humidifier unit with flowmeter

000372805

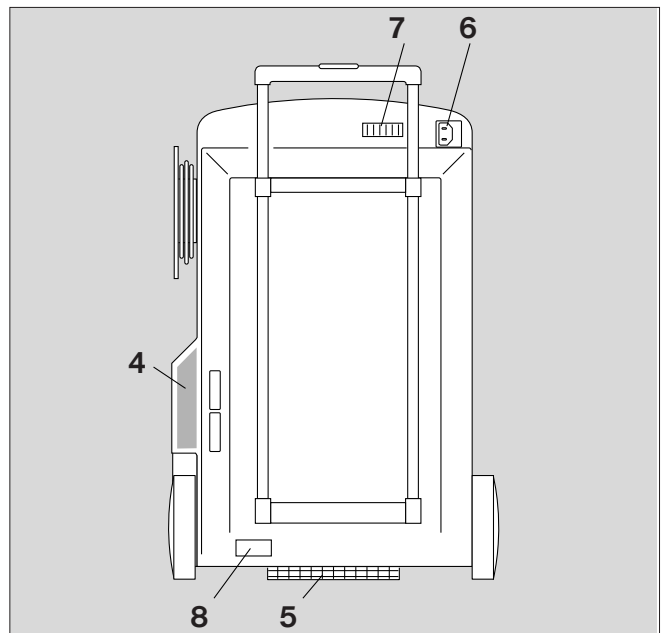Rear view

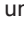
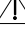
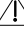
- 4 Dust filter for the air intake
- 5 Outlet for exhaust air
- 6 Appliance socket for mains cable
- 7 Operating hours counter
- 8 Rating plate


01132205

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.


Control panel

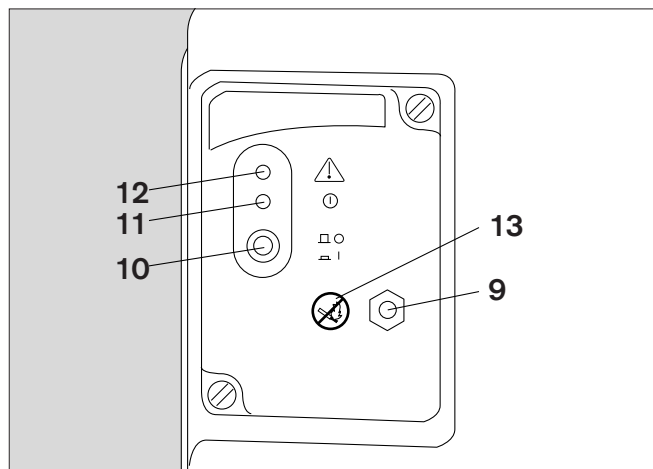- 9 Oxygen outlet
- 10 ON/OFF switch »  «
- 11 Green lamp »  « shows unit is switched on
- 12 Red lamp »  « for alarms
- Symbol  : Read Instructions for Use
- 13 Symbol  : No smoking, no naked lights


Portable humidifier unit

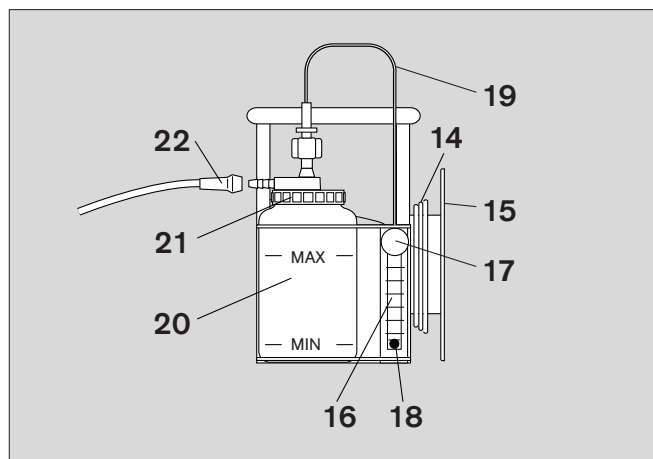- 14 Extension hose
- 15 Reel for extension hose
- 16 Flowmeter
- 17 Rotary control knob for setting the oxygen flow rate
- 18 Ball in flowmeter to indicate the oxygen flow rate in L/min
- 19 Connection hose bottle
- 20 Humidifier bottle
- 21 Cap of humidifier bottle
- 22 Connection of nasal cannula

Connection of hoses and cables

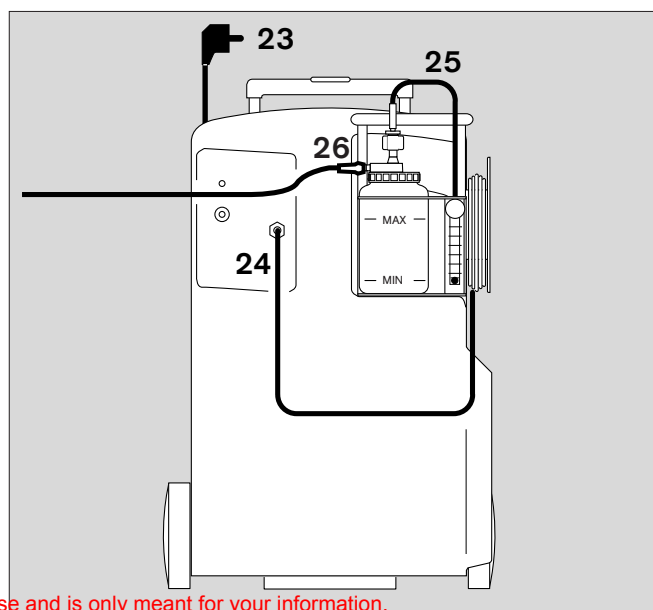- 23 Mains cable plugged into rear of unit
- 24 Extension hose plugged into oxygen outlet
- 25 Connection hose bottle to union nut of humidifier unit
- 26 Hose of nasal cannula connected to humidifier bottle

This "Instructions for Use" manual is published for public use and is only meant for your information.
 We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
 Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

Technical Data

Ambient conditions

Operation

Temperature	10 to 40 °C
Air pressure	600 to 1100 hPa (600 to 1100 mbar)
Humidity	10 to 90 % rel. humidity

Transport and Storage

Temperature	-20 to 60 °C
Air pressure	600 to 1100 hPa (600 to 1100 mbar)
Humidity	10 to 95 % rel. humidity

Performance data

Flow rate	0.5 to 6 L/min
Oxygen concentration ¹⁾	
at flow rates of 0.5 to 4 L/min	93 ±3 vol.%
at a flow rate of 5 L/min	90 ±3 vol.%
at a flow rate of 6 L/min	83 ±3 vol.%
Maximum altitudes ²⁾	
at flow rates of 0.5 to 3 L/min	up to 4000 m
at a flow rate of 4 L/min	up to 3000 m
at a flow rate of 5 L/min	up to 2000 m
Outlet pressure	approx. 70 kPa (700 mbar)
Back pressure effect at flow 6 L/min	
at 0 kPa outlet pressure	6.0 L/min
at 7 kPa outlet pressure	5.9 L/min

Operating data

Operating voltage	230 VAC, 50 Hz
Mean current	2.0 A
Mean power consumption	390 W
Sound pressure level ³⁾	45 dB (A)
Alarms ⁴⁾	visual and acoustic
Weight	25 kg
Dimensions	
Height x width x depth	640 x 400 x 415 mm
Electromagnetic compatibility EMC	according EN 60601-1-2
Protection class, according EN 60601-1	Class II; Type B 
Classification in accordance with EC Directive 93/42/EEC Annex IX	Class IIa
UMDNS Code ⁵⁾	12-873

1) at 20 °C, 1013 mbar, 50 % rel. humidity, 10 minutes after switching on or 10 minutes after setting a new flow rate

2) at 20 °C and 50 % rel. humidity

3) measured at a distance of 1 m

4) PermoX SilentCare will generate an alarm if it does not function properly. In this case oxygen concentration may fall below 82 vol.%

5) Universal Medical Device Nomenclature System

Functional Description

The unit enriches the oxygen concentration of the ambient air from approx. 21 vol.% to max. 96 vol.%. This enrichment is based on the adsorption effect of molecular sieves.

The ambient air is drawn in and compressed by a compressor and then forced through a molecular sieve which adsorbs (retains) the nitrogen contained in the ambient air (approx. 79 vol.%) and delivers the remaining oxygen. The unit switches over to a second sieve before the first molecular sieve is saturated with nitrogen. The first sieve is regenerated while the second sieve takes over the job of oxygen enrichment. This process is repeated in cycles. A slight hissing sound is heard when the unit switches over from one sieve to the other.

Alternate regeneration of the two molecular sieves permits continuous oxygen enrichment.

Order List

Part name / description	Part Number
Permox SilentCare basic unit	57 30 289
Consisting of: Concentrator, mains cable, portable humidifier unit 6 L, extension tube 7.5 m, nasal cannula Country specific units are available.	
Optional accessories	
Insufflation:	
Nasal cannula O ₂ , 2.1 m	57 30 354
Humidification:	
Portable humidifier unit 6 L with flowmeter and humidifier bottle	57 30 290
Humidifier bottle	57 30 291
Connection tube bottle	57 30 292
Extension tube, 7.5 m	57 30 293
Extension tube, 15 m	57 30 294
Replacement parts	
Dust filter Permox	57 30 298
Mains cable EURO	57 30 042
Mains cable CH	57 30 043
Mains cable IT	57 30 255
Mains cable GB	57 30 256
Instructions for Use	57 30 442

This "Instructions for Use" manual is published for public use and is only meant for your information.
We do not guarantee the accuracy of these instructions with respect to your specific device type or status.
Only the "Instructions for Use" manual delivered together with a device is officially published for the use of that specific device.

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© Dräger Medical b.v.