PT

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UK

IT

**Explanation of symbols:** Packaging Unit**Purpose:**

**PRIMA-SECURE II** secures the inner cannula of the Care Cannula Series against this slipping out of the outer cannula if, for example, a respiratory tube has been connected to the inner cannula.

**Description:**

See illustrations 1 to 3

**Cleaning:**

The **PRIMA-SECURE II** is a re-usable item. Cleaning is to be carried out without chemical additives in saline solution (0,9%) at a maximum of 65 °C. We recommend subsequent disinfection with a suitable disinfectant (i.e. the **PRIMA-STOM** Care and Disinfectant Solution from HEIMOMED) following the instructions for use. Non-compliance with the cleaning instructions invalidates the guarantee.

**Safety Information:**

The **PRIMA-SECURE II** is not to be used with cannulae from other manufacturers, as in a serious instance quick and safe disconnection cannot be guaranteed. If damage is evident or functionality impaired it must be disposed of.

**Product Disposal**

Disposal of products in germ-free condition may be carried out only in accordance with current national regulations for waste materials.

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