PT

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Finalidade:

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Failure to follow this cleaning procedure invalidates the guarantee.

Safety Information:

If the packaging is damaged the product must not be used. Failure to follow the cleaning instructions can result in the product becoming affected with candida formation and the inner cannula is no longer able to be securely positioned in the outer tube. If the cuff is too tightly positioned, pressure necrosis in the trachea can result.

Always unlock maintenance cannula completely prior to insertion into the trachea. The maintenance cannula should not be withdrawn before blocking, so that the inner lumen cannot be obstructed by the block.

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Packaging unit

Purpose:

The tubes are designed for use by patients with swallowing difficulties. They consist of one outer tube and two inner cannulae, Universal Connector (in part) and cuff for secure positioning and/or suction.

Description:

Prior to insertion of the tube, the cuff must be inflated using a pressure measuring device (e.g. CUFF-CONTROL from HEIMOMED). If the cuff fits tightly the pressure is released and the tube can be inserted by a doctor or nurse. In order for the tube to be easily inserted in the tracheostomy it should be lubricated with a suitable solution (e.g. STOMA-GLIDE from HEIMOMED). With the cuff firmly positioned, care must be taken to ensure that in long-term use the pressure does not exceed 25 mmHg (equivalent to 34 cmH₂O). For this reason the cuff pressure should be checked regularly using a pressure measuring device. If the tube becomes clogged due to high build-up of secretions, the inner cannula must be removed and cleaned. Suctioning through the outer tube using a tracheal suction catheter follows. The inner cannula can then be re-inserted. When removing the tube, any fluid which has accumulated above the cuff must be suctioned before the cuff is released. The upper part of the inner cannulas are colour-coded so that they can be correspondingly matched to a compatible outer tube.

Inner and outer tubes must be matched as follows:

Size	Inner diameter outer tube	Outer diameter inner cannula	Colour-code inner cannula	Length
7	7 mm	6,8 mm	grün	80 mm
8	8 mm	7,8 mm	weiß	80 mm
9	9 mm	8,8 mm	gelb	85 mm
10	10 mm	9,8 mm	blau	95 mm
11	11 mm	10,8 mm	rosa	97 mm

The tubes are made from top-grade plastic and are for use by one patient only. Their high quality and material characteristics ensure optimal conformity to the anatomy of the trachea. Maximum usage time is 29 days. After a maximum usage time of 6 days in position the tube must be removed in order for the trachea to be inspected for, e.g. stenosis to be avoided or recognised in its early stages. Prior to re-use the tube must be cleaned as described in the section „Cleaning“.

Caution: This product is not for use with lasers.

Cleaning:

Cleaning is performed in a saline solution at a maximum temperature of 65 °C without chemical additives. Following this we recommend disinfection of the inner cannulae and outer tubes using a suitable disinfectant/cleanse (e.g. PRIMA-STOM

desinfektionsmittel from HEIMOMED) and according to its instructions. Finally the inner cannula is treated with a lubricant (e.g. PRIMA-STOM II oil from HEIMOMED) using a soft cloth as described in the instructions. After successful cleaning and disinfection the cuff must be checked for tightness.

Failure to follow this cleaning procedure invalidates the guarantee.

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Sterility information:

If the packaging is damaged the product must not be used. Failure to follow the cleaning instructions can result in the product becoming affected with candida formation and the inner cannula is no longer able to be securely positioned in the outer tube.

If the cuff is too tightly positioned, pressure necrosis in the trachea can result.

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Product Disposal:

Disposal of maintenance cannulas and their accessories in a germ-free state must be done in accordance with the applicable national regulations for waste products.

The manufacturer's products are subject to change at all times.

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