Dräger HomeCare

Dräger

RespiCare S RespiCare CV

Atemtherapie-Gerät
Ventilation Therapy Unit

Gebrauchsanweisung
Instructions for Use
Software 3.n

Inhalt

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Contents

For Your Safety and that of Your Patients	4
Intended Use	6
Preparing for Use	8
Positioning	8
Insert coarse dust filter	9
Connect ventilation hose	10
Connect power supply	11
Preparing for Use – By Doctors Only!.....	11
Connect control panel	11
Switch on unit	13
Set ventilation modes / parameters	14
When oxygen is applied.....	26
Operation	28
Start operation	28
Using the soft start function	29
Nasal mask	30
Display airway pressure	30
In the event of a power failure	30
In the event of an alarm	31
Service indicator	33
End of operation	33
Fault – Cause – Remedy	34
Care	36
Stripping down	36
Cleaning / disinfection in the hospital	38
Cleaning / disinfection at home	39

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For Your Safety and that of Your Patients

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

Maintenance

The apparatus must be inspected and serviced regularly by trained service personnel at twelve monthly intervals (and a record kept).
Repair and general overhaul of the apparatus may only be carried out by trained service personnel.
We recommend that a service contract be obtained with Dräger HomeCare and that all repairs also be carried out by them.
Only authentic Dräger HomeCare spare parts may be used for maintenance.
Observe chapter "Maintenance Intervals".

Accessories

Do not use accessory parts other than those in the order list.

Not for use in areas of explosion hazard

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

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Dräger HomeCare
a division of Dräger Nederland B. V.

Safe connection with other electrical equipment

Electrical connections to equipment which is not listed in these Instructions for Use should only be made following consultations with the respective manufacturers or an expert.

Liability for proper function or damage

The liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by Dräger HomeCare or if the apparatus is used in a manner not conforming to its intended use.

Dräger HomeCare cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Dräger HomeCare are likewise not modified by the recommendations given above.

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Intended Use

RespiCare® – respiratory support unit for patients who need time-cycled, pressure-controlled ventilation support for therapy reasons. For use only with patients who can maintain spontaneous breathing.

RespiCare should only be used with a non-invasive ventilation connector such as nasal mask, mouth mask or mouth-and-nose mask.

For sleep apnoea therapy,
for weaning long-term ventilation patients and
for disorders of the respiratory muscles.

For use
in hospitals,
in sleep laboratories,
at home,
for stationary or mobile use in vehicles and wheelchairs.

RespiCare is available in two models with specific ventilation modes:

RespiCare S

CPAP – Spontaneous breathing with continuous positive pressure in the airways.

CPAP/ASB – Spontaneous breathing with synchronized pressure support and optional apnoea ventilation.

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RespiCare CV

CPAP – Breathing with continuous positive pressure in the airways.

CPAP/ASB – Spontaneous breathing with synchronized pressure support and optional apnoea ventilation.

PCV(S) – Pressure controlled, time cycled ventilation with synchronized inspiration phase.

PCV – Pressure controlled ventilation.

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The unit may only be used as prescribed by a doctor and with the settings specified by the doctor!

The unit must not be used for life-saving functions.

Mobile telephones must not be used within a distance of 10 metres from the unit!

Mobile telephones can impair the correct functioning of electro-mechanical equipment and can therefore put patients at risk.

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The unit must not be used in explosion-hazard areas!
Risk of fire and explosion.

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Preparing for Use

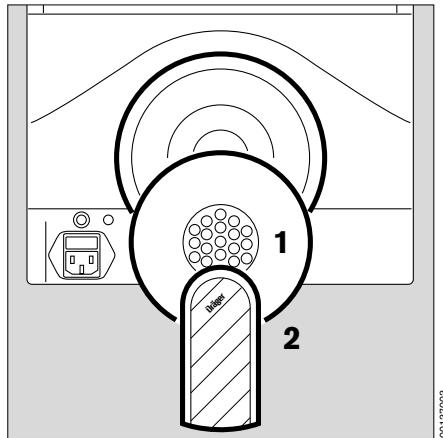
Positioning

- The RespiCare must be set up on a level, horizontal surface, such as a bedside table, wall shelf or trolley. Ventilation slits in the base of the housing must remain clear! Ensure that the unit cannot fall down!
 - The unit must not be set up near a heater and must not be exposed to direct sunlight, otherwise it may overheat! Ensure that air can circulate freely around the unit.
 - Filter and silencer on the rear of the unit must not be blocked by curtains, blankets, etc. otherwise air cannot be drawn into the unit!
 - The vent in the carrying handle must not be clogged!
 - Ensure that the unit cannot fall into full bathtubs, basins, etc. as this can cause electric shocks!
- When used out of doors, e.g. in a wheelchair:
- Ensure that the unit is protected from wetness!

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Insert coarse dust filter

- 1 Plug new coarse dust filter into insert on rear of unit and turn clockwise as far as possible = lock into place.
- 2 Plug silencer into filter housing until it engages.

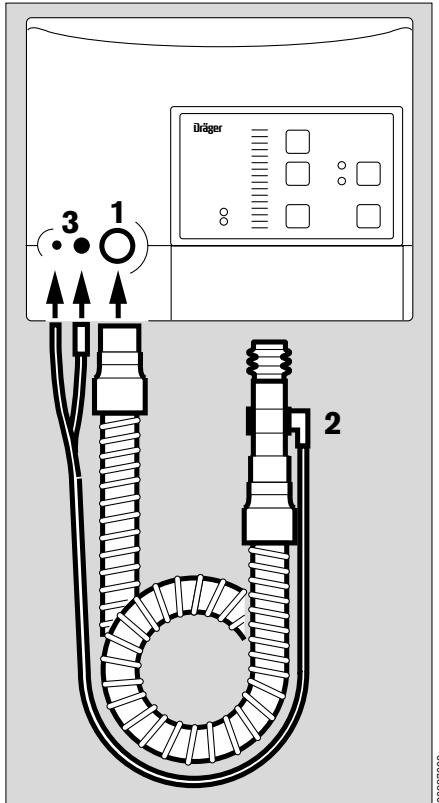
The coarse dust filter traps coarse particles, such as fluff.

Allergic patients can use a fine dust filter to trap household dust, pollen, etc.
See order list, page 60.

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Connect ventilation hose

Only use original ventilation hoses!

- 1 Firmly plug the ventilation hose into the socket on the unit.
 - 2 Plug the other end of the ventilation hose into the E-Vent test outlet – test hoses pointing towards ventilation hose.
 - 3 Plug test hoses into the sockets on the unit – note different diameters.
- Connect the E-Vent test outlet with the port on the nasal mask.
A nasal mask without outflow port must be used!
CO₂ is purged from the system via the E-Vent test outlet.

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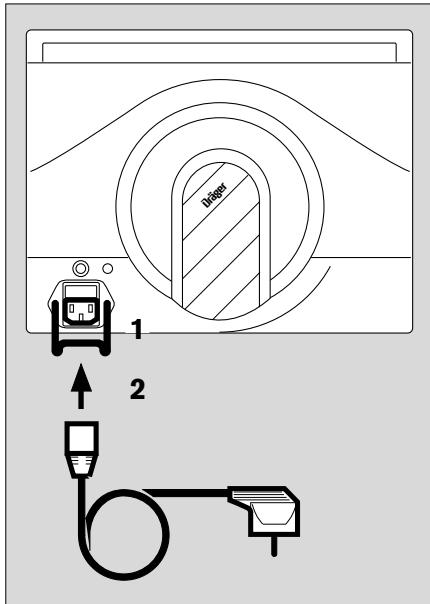
Connect power supply

The RespiCare units are designed for operation on voltages between 100 and 240 V. They are set automatically.

- 1 Press the safety clip down to open.
- 2 Plug the power cord into the socket on the rear of the unit and into the mains socket.
- 1 Push the safety clip up until the power cord is clamped in place.

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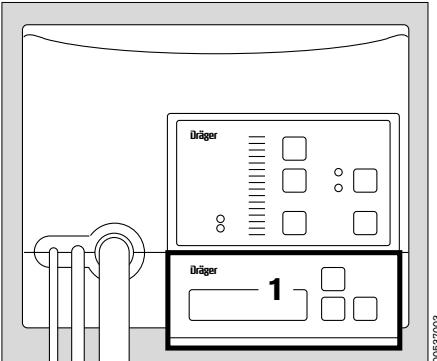
Every basic RespiCare unit can be set as required with the aid of the control panel. **The control panel may only be used by a doctor or by someone who has been duly authorized by the doctor!**

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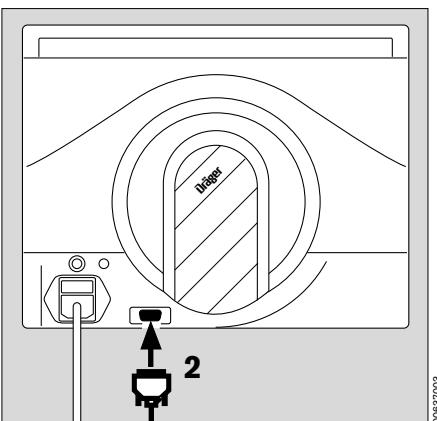
- 1 Insert the control panel in the magnetic holder on the front of the unit. Lay the unit on its side, press the lead into the duct in the base of the unit and

- 2 plug the control panel lead into the socket on the rear.

- Return the unit to an upright position.

Remote operation of the control panel:

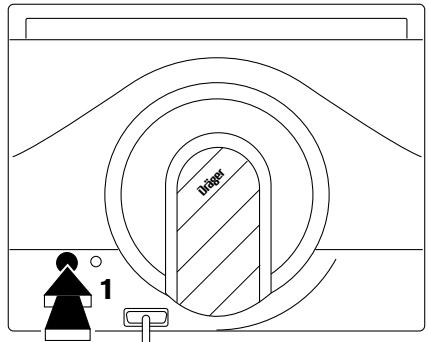
- Connect the extension lead to the control panel plug and plug the extension lead into the socket on the rear of the unit.
Only use the original extension lead.
Other components must not be connected.

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

Switch on unit

- 1 Press on/off switch on rear of unit – until it engages = ON.
- The RespiCare unit starts a self-test for approx. 1 second to test the internal program memories. All LEDs briefly light up during this time.
- 2 The green LED in the  button remains alight.

On the control panel:

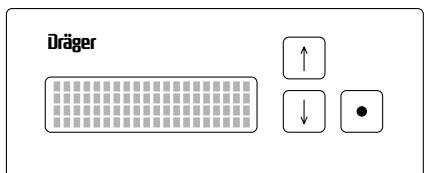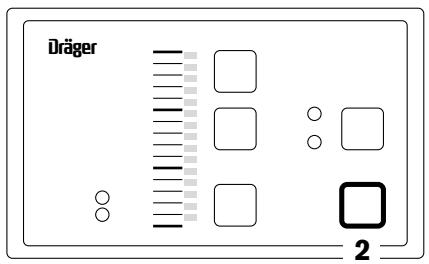- All pixels appear on the display and the display is illuminated for approx. 1 second. The model name then appears in the first line, together with the last ventilation mode selected. This line remains on display as the status line.
Example:
CPAP/ASB

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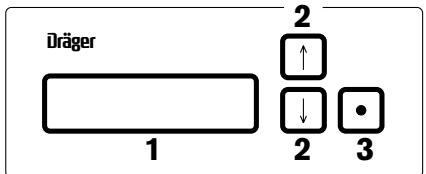
Set ventilation modes / parameters

This is done with the aid of the control panel.

The control panel may only be used by a doctor or by someone who has been duly authorized by the doctor!

1 Display - example:

↓ CPAP ↓

Vertical arrows identify the selected line and the direction in which additional lines can be displayed.

2 Press button ↓ or ↑ to select a line.

3 Press button • to activate the line.

The two arrows flash and point towards the text. Example:

→ Paw = 20 mbar ←

The mode or parameter can now be set:

2 Use the buttons ↓ or ↑ to set the required values. The settings are immediately adopted by the unit.

3 Press button • to confirm.

The display is automatically illuminated for two minutes after pressing one of the three buttons.

The settings are saved even when the unit is switched off.

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Set CPAP

CPAP – Continuous Positive Airway Pressure
Spontaneous breathing with continuous positive pressure in the airways.

- Press to select the first line and activate by pressing
- Use to set CPAP mode and press to confirm.
- Press to select Paw = _ _ mbar and activate by pressing
- Press or to set Paw = _ _ mbar and press to confirm.

Adjustment range: 3 to 20 mbar
in increments of 0.2 mbar

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CPAP – Continuous Positive Airway Pressure

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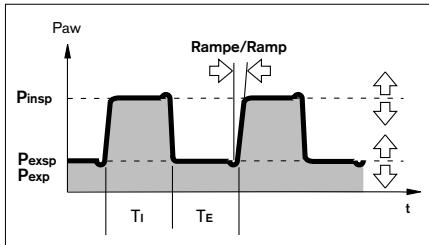
Set CPAP/ASB

CPAP – Continuous Positive Airway Pressure

Spontaneous breathing with continuous positive pressure in the airways.

ASB – Assisted Spontaneous Breathing

Spontaneous breathing with synchronized assistance.

With every spontaneous inspiration, the unit provides assistance until the pressure level P_{insp} is reached and then switches over to pressure level P_{exp} when expiration commences or after not more than 4 seconds.

- Press to select the first line and activate by pressing
- Use or to set **CPAP/ASB** mode and press to confirm.
- Press to select $P_{insp} = \underline{\underline{\text{mbar}}}$ and activate by pressing
 P_{insp} : inspiratory pressure.
- Use or to set $P_{insp} = \underline{\underline{\text{mbar}}}$ and press to confirm.

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- Press to select **Pexp = _ _mbar** and activate by pressing
Pexp: expiratory pressure.
- Use or to set **Pexp = _ _mbar** and press to confirm.
- After switching on or disconnecting the unit, wait for a few breaths until the system has stabilized before setting the trigger.
- Press to select **Trigg.I = _ of 5** and activate by pressing
Trigg.I: Sensitivity for synchronizing spontaneous breathing and inspiration.
1 = Maximum sensitivity
- Use or to set **Trigg.I = _ of 5** and press to confirm.
- Press to select **Trigg.E = _ of 5** and activate by pressing
Trigg.E: Sensitivity for synchronizing spontaneous breathing and expiration.
1 = Maximum sensitivity
- Use or to set **Trigg.E = _ of 5** and press to confirm.

Each inspiration triggered by spontaneous breathing will be indicated by a star in the first line of the control panel display.

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- Press to select **Ramp = 0._ s** and activate by pressing Ramp: Time for pressure increase from Pexp to Pinsp.
0.0 s = Steepest pressure increase
- Use or to set **Ramp = 0._ s** and press to confirm.

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1) At extreme settings (Pinsp = 30 mbar, Pexp = 2 mbar) Pexp. = 2 mbar may not be attained in some cases.

2) Trigger sensitivity: see Technical Data, page 53

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Apnoea ventilation

CPAP/ASB mode can optionally be combined with apnoea ventilation. In this case, the unit switches over to time controlled ventilation if it does not detect any spontaneous breathing within a definable time. When apnoea ventilation is selected, ApnoeVent appears in the first line. The unit stops time controlled ventilation when spontaneous breathing is restored, and at the latest after one minute.

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- Press  to select **ApneaVent** = __, and activate by pressing 
- Press  to set **ApneaVent** = **On**, and press 
- Press  to select **AV-Delay** = __ s, and activate by pressing 

AV = Apnea Ventilation
AV-Delay = Time after end of inspiration phase.

- Press  or  to set **AV-Delay** = __ s, and press 
- Press  to select **AV-Freq** = __ /min, and activate by pressing 

AV-Freq = Ventilation frequency during apnoea ventilation

- Press  or  to set **AV-Freq** = __ /min, and press 
- Press  to select **AV-Insp** = __ %, and activate by pressing 

AV-Insp = Inspiration as a proportion of the overall breathing phase in %

- Press  or  to set **AV-Insp** = __ %, and press 

Adjustment ranges for apnoea ventilation

Parameter	Range	Increments
AV-Delay	5 to 30 s	1 s
AV Freq	6 to 40 /min	1 /min
AV Insp	20 to 50 %	2 %

PCV, PCV(S) einstellen

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PCV – Pressure Controlled Ventilation

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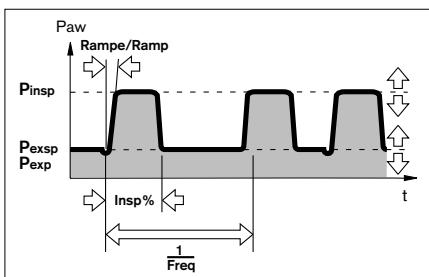
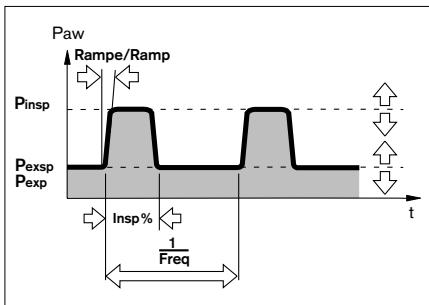
Set PCV / PCV(S)

Only with RespiCare CV.

PCV – Pressure Controlled Ventilation

Pressure-controlled and time-cycled ventilation.

PCV(S) – Pressure Controlled Ventilation Synchronized

Pressure-controlled and time-cycled ventilation (like PCV) in which the patient can trigger an early ventilation stroke in synchrony with spontaneous inspiration, thereby increasing the ventilation frequency.

The inspiration time is predetermined by the setting and limited to not more than 2 seconds.

PCV(S) differs from PCV in that ventilation is synchronized to the patient's inspiration.

To switch between PCV and PCV(S), activate or deactivate the trigger **Trigg. I**, see page 22.

- Press to select the first line and activate by pressing
- Use to set **PCV** or **PCV(S)** and press to confirm.

PCV initially appears in the display for both modes.

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- Press to select **Pinsp = _ _ mbar** and activate by pressing .
P_{insp}: inspiratory pressure
- Use or to set **Pinsp = _ _ mbar** and press to confirm.
- Press to select **Pexp = _ _ mbar** and activate by pressing .
P_{exp}: expiratory pressure.
- Use or to set **Pexp = _ _ mbar** and press to confirm.
- Press to select **Freq = _ _ /min** and activate by pressing .
- Use or to set **Freq = _ _ /min** and press to confirm.
- Press to select **%Insp = _ _ %** and activate by pressing .
% Insp: Inspiration share of the respiratory phase in %.
- Use or to set **%Insp = _ _ %** and press to confirm.

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for PCV(S):

After switching on or disconnecting the unit, wait for a few breaths until the system has stabilized before setting the trigger.

- Press  to select **Trigg.I = _ of 5** and activate by pressing 

Trigg.I: Sensitivity for synchronizing spontaneous breathing and inspiration.

1 = Maximum sensitivity

- Use  or  to set **Trigg.I = _ of 5** and press  to confirm.

- The mode appears in the first line **PCV(S)**

Each inspiration triggered by spontaneous breathing will be indicated by a star in the first line of the control panel display.

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for PCV:

- Press  to select **Trigg.I = _of 5** and press  to activate

- Press  to select until **Trigg.I = inactive** and press  to confirm

- The mode appears in the first line: **PCV**

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- 1) At extreme settings (Pinsp = 30 mbar, Pexp = 2 mbar) Pexp. = 2 mbar may not be attained in some cases.
- 2) Trigger sensitivity: see Technical Data, page 53

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Activate / deactivate

disconnection / apnoea alarm

- Press  or  to select **Alarm** and activate by pressing .
- Select required status
Alarm = active by pressing 
or
Alarm = inactive by pressing  and press  to confirm.

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- Press  or  to select **Patient = ___h**.

Reset application time to 0:

- Press  or  to select **Patient = ___h**, press  to confirm and
- press  to reset display to 0 and press  to confirm.

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Monitor functions

RespiCare CV only

A status is displayed in the first line of the control panel behind
the ventilation mode selected, e.g. PCV(S):

e.g. DISCONN or APNOEVENT

The parameters calculated are displayed in the 2nd, 3rd and
4th lines:

Tidal volume VT,

Frequency f

Minute volume MV

Example:

VT 1000 mL

f 20 1/min

MV 20.0 L/min

The parameters calculated are determined from several
breaths. They are for information only and are not used to
monitor ventilation.

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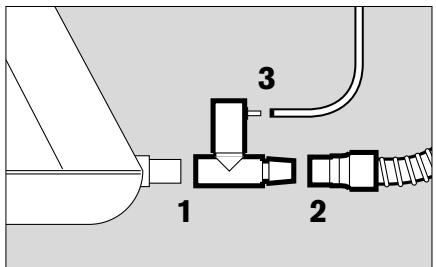
When oxygen is applied

Oxygen (O_2) actively supports combustion.

No smoking, no naked lights!

Use the oxygen adapter, see order list, page 60.
Other adapters must not be used!

The oxygen adapter prevents oxygen leaking into the unit when the fan is defective (no airway pressure).

Oxygen must not be directed into the unit or intake port, due to the risk of fire!

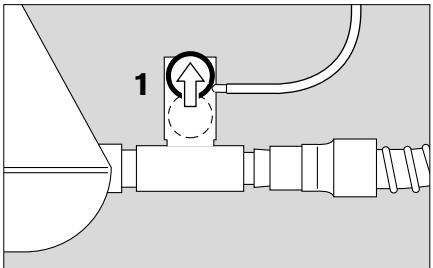
- 1 Plug the oxygen adapter onto the socket of the RespiCare. **Ensure that the valve housing points upwards.**
- 2 Plug the ventilation hose onto the socket of the oxygen adapter.
- 3 Plug the oxygen metering hose onto the socket of the adapter.

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Switch on the RespiCare:

- Press on/off switch,
 - press button , the green LED goes out.
- 1 The ball in the oxygen adapter **must** move upwards when the fan is switched on.
 - Turn on the oxygen supply.

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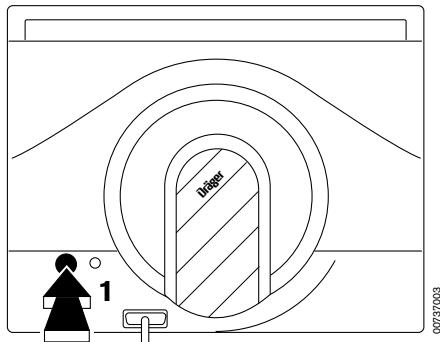
After use:

- **Turn off the oxygen supply before switching off the RespiCare.**

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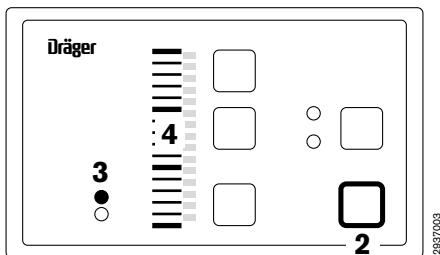
Operation

Start operation

- 1 Press on/off switch on rear of unit – until it engages = ON.
The RespiCare performs a self-test.

- 2 The green LED in the button lights up and the unit is now on standby.
- 3 The "min." (minutes) LED lights up while the unit is on standby.
- 4 The bar graph indicates the set time for the "soft start" function: 1 bar represents 2 minutes.
To set the soft start function, see page 29.
- 2 Press the button. The green LED goes out.
The soft start time display goes out, and the blower starts delivering air.

After starting up the unit suppresses any alarms for 1 minute.

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
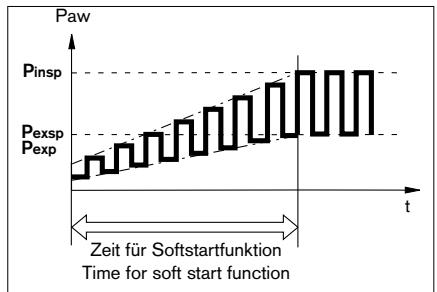
Using the soft start function

Can only be set in standby mode.

For slowly and "softly" increasing the set airway pressure over a variable time of 0 to 30 minutes.

- 1 Press  to switch to standby. The green LED in the button lights up.
- 2 The **min.** LED lights up.
- 3 The time (minutes) elapsed before reaching the airway pressure set on the control panel is indicated on the bar graph. Each gradation corresponds to 2 minutes.
- 4 Press  or  to set the required time. The time changes by 2 minutes whenever one of the two buttons is pressed.
 - 1 Press  to switch to operating mode. The green LED goes out and the set time becomes effective.
 - 3 The current airway pressure is now displayed on the bar graph.

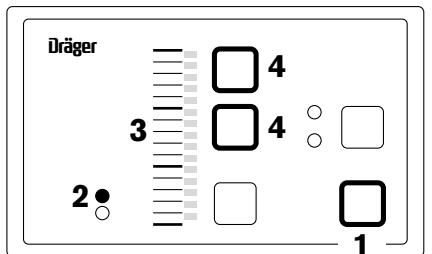The soft start function is effective whenever changing over from standby to operation and is also retained after switching off the unit.

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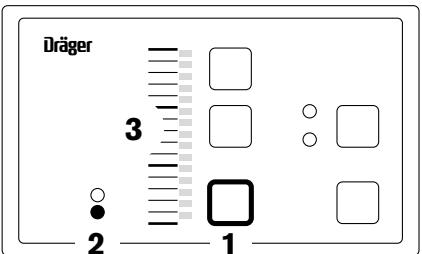
Nasal mask

- For RespiCare, use only nasal masks without outflow port. CO₂ is purged via the E-Vent test outlet.
- Put on nasal mask in accordance with separate Instructions for Use.

Display airway pressure

during operation.

- 1 Press 
- 2 The **mbar** LED lights up.
- 3 The current airway pressure is shown on the bar graph. Each gradation corresponds to 2 mbar.

In the event of a power failure

a continuous tone sounds for at least 2 minutes.

This acoustic alarm cannot be switched off.

- Remove the nasal mask to avoid any accumulation of CO₂ and a shortage of oxygen!

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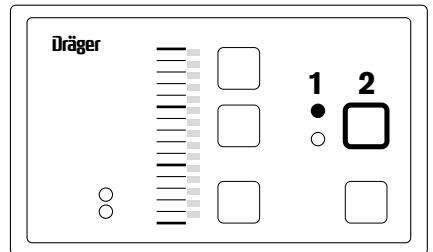
alarm LED Δ :

Disconnection or apnoea.

The audio alarm can be suppressed for 2 minutes:

2 Press A .

- Eliminate the cause of the disconnection.

Further information for troubleshooting can be found in the table "Fault – cause – remedy" on page 34. The flashing red alarm LED Δ goes out and the audio alarm stops when the cause of the fault has been remedied.

The following message flashes in the first line when the control panel is connected:

Disconnect

or

ApnoeVent

- Remedy the cause of the alarm. The display will clear automatically.

Parameters can be set even while an alarm is displayed.

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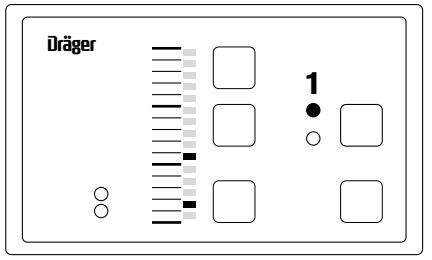
LED ⚠ lights up steadily:

Equipment fault.

Individual segments of the bar graph light up constantly.

The alarm tone cannot be switched off.

The fan switches off.

- Remove the nasal mask to avoid any accumulation of CO₂ and a shortage of oxygen!
- Read off the illuminated gradations on the bar graph (example: 4/12) and report these to Dräger HomeCare. The numbers refer to the equipment fault.

The following message appears on the display of the control panel when the latter is connected:

Equipment fault, technical defect

with the same numbers as on the bar graph.

Continuous tone without visual indication

draws attention to a power failure.

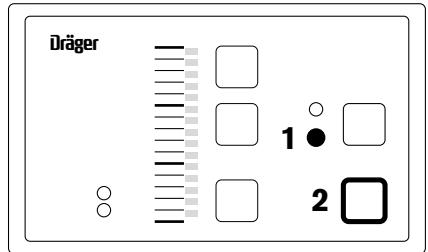
This alarm tone cannot be switched off.

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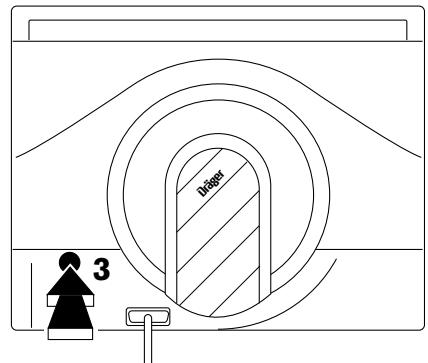
Service indicator

The required inspection and maintenance intervals are monitored and signalled automatically by the RespiCare unit:

- 1 Yellow LED lights up :
The unit must be inspected and serviced by experts within the next two months.
The LED does not go out until the unit has been inspected and serviced.

End of operation

When oxygen has been applied:

- **Turn off the oxygen supply first** and then switch the unit to standby or off.

In the event of brief interruptions up to approx. 15 minutes:

- 2 Press and the green LED lights up. The unit remains on standby.

For longer breaks:

- 2 Press and the green LED lights up. The unit is on standby.

- 3 Depress switch on rear of unit completely = OFF.
The unit is now switched off.

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Fault – Cause – Remedy

If a fault occurs, the following table can help to identify the underlying cause and remedy it.

Fault	Cause	Remedy
Intermittent tone, red alarm LED  flashes.	Disconnection	Ensure that ventilation hose, test hoses, nasal mask are fitted without leaks or kinks.
Airway pressure is not reached during inspiration.	Coarse dust filter/ fine dust filter clogged.	Clean coarse dust filter, page 44, or replace it. Replace fine dust filter.
Airway pressure has increased significantly.	Leak in pressure measuring hose or hose has slipped off socket.	Ensure that there are no leaks or kinks in the test hoses.
Unit does not switch over to pressure assistance in CPAP/ASB mode.	Leak in test hoses or hoses buckled.	Ensure that there are no leaks or kinks in the test hoses.
No pressure change in PCV / PCV(S) modes.	Pressure measurement defective.	Ensure that there are no leaks or kinks in the test hoses.

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Fault	Cause	Remedy
Trigger function or pressure control defective	Water residue in test hoses	Fully dry out the test hoses
Set pressure parameters too low after switching on.	Soft start function effective	Check setting of soft start function, page 29.
Continuous tone, red alarm LED  lights up continuously.	Equipment fault.	Call Dräger HomeCare or an authorized expert.
Continuous tone, all lamps off.	Fuse has blown, no mains power.	Replace fuse, page 45. See fuse ratings in Technical Data, page 53.
Airway pressure not displayed on bar graph.	Bar graph illumination deactivated.	Press  to activate bar graph illumination.

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If the faults cannot be remedied:

- Use a spare unit and call Dräger HomeCare or an authorized expert.

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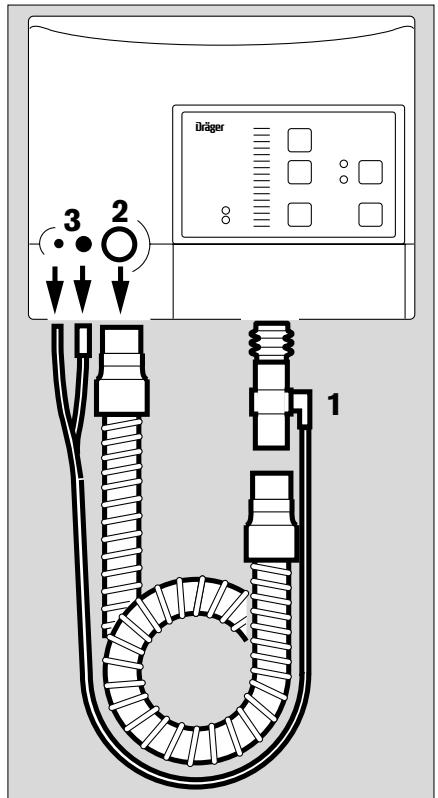
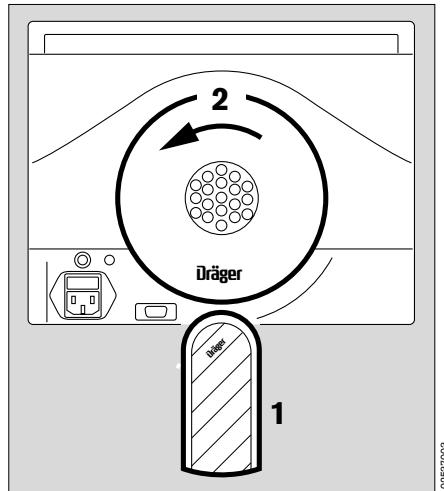
Otherwise, risk of electric shock.

Stripping down

- 1 Remove E-Vent test outlet from ventilation hose.
- 2 Pull ventilation hose off its socket and unplug the connector to the oxygen adapter if applicable.
- 3 Remove test hoses from sockets.

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On the rear:

- 1 Remove silencer.
 - 2 Turn coarse dust filter or fine dust filter anticlockwise = unlock.
Remove filter.
- Clean coarse dust filter, see page 44.
Fine dust filter must be replaced after approx. 500 hours of application.

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Cleaning / disinfection in the hospital

Use a surface disinfectant, such as a product based on:

- Aldehydes
- Alcohol
- Quaternary ammonium compounds.

The following are not suitable:

- Compounds containing phenol
- Compounds which give off halogens
- Strong organic acids
- Compounds which give off oxygen.

Users in the Federal Republic of Germany are advised to use the disinfectants specified in the current DGHM* list.

This list also specifies the active agent in each disinfectant. Products based on the active agents specified above are recommended for users in countries in which the DGHM list is not available.

Control panel, basic unit, cable

- Wipe-disinfecting:
The following disinfectants can e.g. be recommended:
Incidur®, Frekanol®, Gigasept FF®
Note the manufacturer's directions.
Liquid must not be allowed to leak into the connections.

Ventilation hose, test hoses with E-Vent test outlet, oxygen adapter

- Disinfect in hot liquid (93 °C /10 minutes) in an automatic washing and disinfection machine – **with detergent only.**

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If a washing machine is not available:

- Disinfect in a bath, e.g. with Helipur (made by Braun, Melsungen, Germany). Note the manufacturer's directions. The parts must subsequently be rinsed in clear water, preferably from the soft water tap. Thoroughly shake off all remaining water. Allow the parts to dry completely. Water in the test hoses may impair the unit's correct operation.
- Hot steam sterilization:
After hot liquid disinfecting or immersion disinfecting, the ventilation hose, test hoses and E-Vent can be sterilized in hot steam at 134 °C.

Nasal mask

- Must be prepared in accordance with the separate Instructions for Use.

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Otherwise, risk of electric shock.

Control panel, basic unit, cable

- Remove any dirt with a disposable cloth.
Liquid must not be allowed to leak into the connections

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Ventilation hose, test hoses, E-Vent test outlet and oxygen adapter

- Wash thoroughly in warm soapy water and rinse thoroughly under running water.
Then
- place in a solution of one part wine vinegar and two parts water for approx. 30 minutes.
- Rinse thoroughly under running water. Shake hoses to drain off all water and dry thoroughly..
- Dry parts thoroughly.
Water in the test hoses or in the oxygen adapter may impair the operation of the unit.
Water in the ventilation hose or in the E-Vent test outlet can reduce the air flow through the test outlet and allow the CO₂ content of the inspiratory air to rise.

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
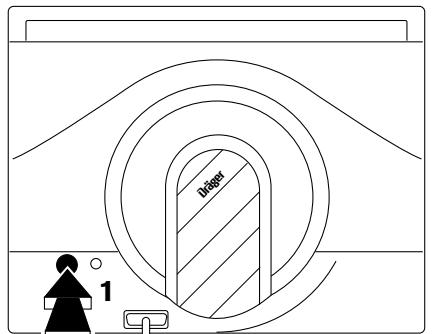
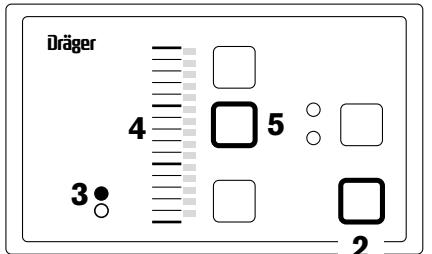
Check Readiness for Operation

The unit's readiness must be checked whenever it has been cleaned and disinfected.

After assembling the unit ready for operation:

- 1 Switch on unit:
Press button on rear of unit – until it engages = ON.
- 2 The green LED in the button lights up.
- 3 LED **min.** lights up.
- 4 The time for the soft start is shown on the bar graph in minutes.
- 5 Press and set the time for the soft start function to zero.
- Put on a suitable nasal mask and
- 2 press to switch on the fan.

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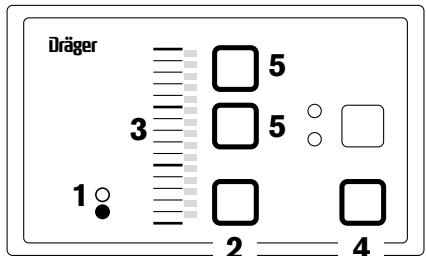- 1 LED **mbar** lights up.
- 2 Press :
- 3 The set airway pressure must appear on the bar graph.
- 4 Press , the green LED lights up and the fan is switched off.
- Remove nasal mask.
- 5 Use the buttons and to set the time for the soft start.

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Maintenance Intervals

Coarse dust filter

Must be cleaned every three weeks.
The filter must be replaced if damaged,
at the latest every six months.
It can be disposed of as household refuse.

Fine dust filter

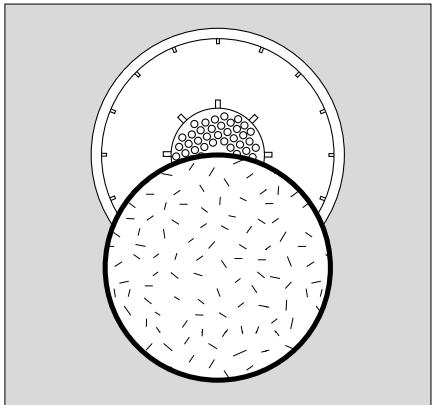
Must be replaced after approx. 500 hours of application.
It must be replaced at the latest after six months.
The filter can be disposed of as house-hold refuse.

Inspection and maintenance

As indicated by the service indicator  see page 33.
At least once per year.

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
Clean coarse dust filter

- Every three weeks.
- Draw foam insert out of filter housing.
- Wash foam insert in soapy water, rinse thoroughly in clear water and dry completely.
- Press foam insert back into filter housing.
- Damaged foam inserts must be replaced.

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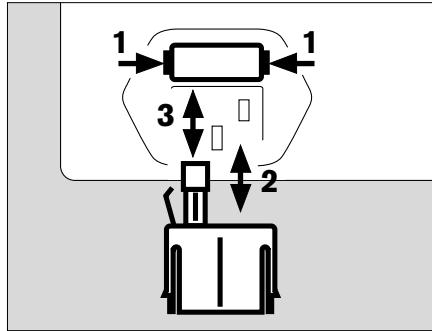Replace fuse

- Disconnect mains plug!
- 1 Press the two tabs on the fuse insert together and simultaneously
- 2 pull out the insert. There is a spare fuse inside the insert.
- 3 Remove the blown fuse and insert a new fuse.
Use a slow-blow fuse T 2 H 250 V
IEC 127-2/V.
Order a new spare fuse!
- Press the fuse insert in – until it engages.

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Disposal of the unit

- when it reaches the end of its useful life.
- RespiCare must be appropriately disposed of in agreement with the competent specialized waste disposal company.
All applicable legal regulations must be obeyed.

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Customer Service Mode

For selecting the language in which messages are displayed on the control panel.

For testing the alarm tone and lamps.

For indicating the hours of operation.

For indicating the software version.

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← - 001 + →

Select customer service mode

- Disconnect RespiCare from the patient and connect the control panel.
- Depress on/off switch completely = OFF.
- Press on/off switch – until it engages = ON simultaneously
- press  for at least 3 seconds.
- The RespiCare unit runs a self-test before the following message appears:

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Set language

To activate:

- Briefly press and the set language appears on the second line.

Display (example):

Adjust language
deutsch

- The following languages can be set as required using the and buttons:

English	
German	(deutsch)
French	(français)
Spanish	(español)
Italian	(italiano)
Portuguese	(português)
Swedish	(svenska)
Dutch	(nederlands)
Danish	(dansk)

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Pat: Period of use by patient

or

Service: Hours of operation since unit
was last serviced

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← - 004 + →

Press  to select:

Main device: Software version of the basic unit

Remote device: Software version of the control panel

To end the display:

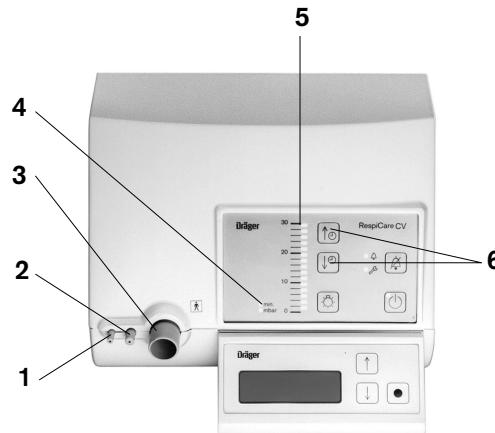
- Press  again.

To leave customer service mode:

- Depress on/off switch completely = OFF.

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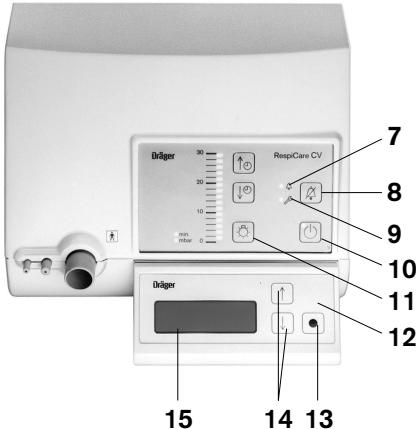
- 1 Socket for "airway pressure" test hose
- 2 Socket for "differential pressure" test hose
- 3 Socket for ventilation hose
- 4 Yellow LEDs for bar graph
 - **min.** LED lights up: time for soft start (minutes)
 - **mbar** LED lights up: airway pressure
- 5 Bar graph for airway pressure (mbar) or time for soft start (min.)
- 6 Two buttons for setting the time for the soft start
↓ = decrease, ↑ = increase

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
- 7 Red alarm LED ⚡
- 8 Button ⚡ to suppress the alarm tone for approx. 2 minutes
- 9 Yellow LED for service required ⚡
- 10 Button ⏹ to switch the fan on and off (standby)
- 11 Button ⚡ to switch the bar graph illumination on and off
- 12 Control panel for settings and data
- 13 Button ⚡ to activate and confirm settings
- 14 Two buttons ⬇ and ⬈ for making selections and settings
- 15 Display for settings, data and messages

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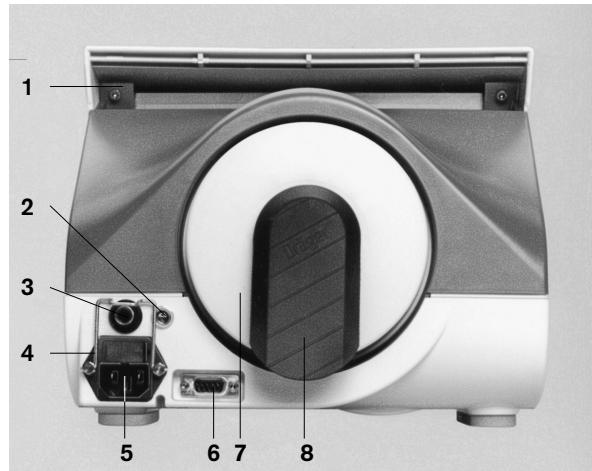Rating plate on underside
(not illustrated)

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Ambient conditions

In operation

Temperature	5 to 40 °C
Air pressure	600 to 1100 hPa
Rel. humidity	10 to 95 %, no condensation

In storage

Temperature	-20 to 70 °C
Air pressure	600 to 1200 hPa
Rel. humidity	10 to 95 %, no condensation

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Settings, RespiCare CV

Ventilation mode CPAP	
Airway pressure	3 to 20 mbar
Ventilation mode CPAP/ASB	
Inspiration pressure Pinsp	5 to 30 mbar
Expiration pressure Pexp	2 to 20 mbar
Trigger sensitivity Trigg.I	5 settings (approx. 10 to 50 L/min)
Trigger sensitivity Trigg.E	5 settings (approx. 15 to 75 L/min below peak flow)
Ramp	11 settings (0 to 1.0 s)
Apnoea Ventilation	ON/OFF
Delay time	5 to 30 s
AV-Frequency	6 to 40 /min (bpm)
AV-Insp.	20 to 50 %

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Ventilation mode PCV/ PCV(S)

Inspiration pressure Pinsp	5 to 30 mbar
Expiration pressure Pexp	2 to 20 mbar
Ramp	11 settings (0 to 1.0 s)
Ventilation frequency Freq	6 to 60/min (bpm)
Insp. as % of resp. phase % Insp	20 to 66 %
Trigger sensitivity Trigg.I	5 settings (approx. 10 to 50 L/min)
Time for soft start function	0 to 30 min

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Monitors

Disconnection/apnoea

Triggered when alarm limit is exceeded for at least 5 seconds.

Alarm limit for airway pressures >10 mbar:
actual value at least 5 mbar below set value.

Alarm limit for airway pressures ≤10 mbar:
actual value less than 50 % of set value.

The alarm is suppressed for approx. 1 minute after switching on the fan and when changing the ventilation mode.

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Airway pressure output Paw (t)

Output voltage	40 mV/mbar, 0 mbar = 0.9 V
Load impedance	>100 kΩ
Wiring	Brown = +, white = – shield connected to plug housing
Electromagnetic compatibility EMC	Tested to EN 60 601-1-2

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Abbreviations and Symbols

ApneaVent	Pressure controlled, time controlled ventilation in the event of apnoea
AV-Delay	Selectable delay time between the end of the inspiration phase and the start of apnoea ventilation
AV-Freq	Ventilation frequency during apnoea ventilation
AV-Insp.	Inspiration phase as a proportion of the overall respiration phase during apnoea ventilation
ASB	A sisted S pontaneous B reathing Spontaneous breathing with synchronized assistance
Op. time	Operating time counter for the total operating time of the unit in hours
CPAP	C ontinuous P ositive A irway P ressure B reathing Spontaneous breathing with continuous positive pressure in the airways
Freq	Ventilation frequency
Insp %	Percentage inspiration in the respiratory phase
mbar	Bar graph indication of the airway pressure (mbar)
min.	Bar graph indication of the time (minutes) for the soft start function
Patient	Resettable display of the duration of application in hours
Paw	Airway pressure
Pexp	Expiration pressure
Pinsp	Inspiration pressure

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Designation/Description	Part No.
RespiCare S , consisting of: Basic unit, coarse dust filter, power cord, ventilation hose, E-Vent test outlet, test hose system, Instructions for Use and bag.	57 30 325 57 30 308
RespiCare CV , consisting of: Basic unit, coarse dust filter, power cord, ventilation hose, E-Vent test outlet, test hose system, Instructions for Use and bag.	57 30 326 57 30 309
Control panel	57 30 310
Hose system:	
Ventilation hose, 2 m	57 30 311
Test hose system	57 30 312
E-Vent test outlet	57 30 313
Hose clamp (pack of 10) for fixing the test hoses to the ventilation hose	57 30 314
Mask connection, set of 5, standard	57 30 315
Mask connection, set of 5, ISO 15	57 30 316
Coarse dust filter	57 30 317
Fine dust filter	57 30 318
Bag	57 30 319
Silencer	57 30 320
Mains cable euro	57 30 042
Mains cable swiss	57 30 043
Mains cable Italy	57 30 255
Mains cable UK	57 30 256

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Designation/Description	Part No.
Accessories for use with the basic unit	
Extension cable for control panel, 30 m	57 30 321
DC/AC converter (12 V DC to 230 V / 50 Hz)	57 30 322
Cable for analog output Paw (t), 10 m	57 30 323
Oxygen adapter	57 30 324
Trolley	57 30 355
Table for trolley	57 30 356
Instructions for Use de/en	57 30 327
Instructions for Use fr/it	57 30 328
Instructions for Use es/pt	57 30 329

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*) streichen

Date Weight kgBlood pressure **Setting on unit:**

CPAP/ ASB	PCV (S)	PCV
Pinsp: mbar	Pinsp: mbar	Pinsp: mbar
Pexsp: mbar	Pexsp: mbar	Pexsp: mbar
Trigg.I:	Freq.: min.	Freq.: min.
Trigg.E:	%insp: %	%insp: %
Ramp: 0, sec.	Trigg.I:	Ramp: 0, sec.
Alarm active/inactive*)	Ramp: 0, sec.	Alarm active/inactive*)
ApnoeVent OFF/ON*)	Alarm active/inactive*)	
AV latency: sec.		
AV freq.: min.		
AV insp.: %		

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These Instructions for Use apply only to
RespiCare 3.n
with Serial No.:

If no Serial No. has been filled in by
Dräger HomeCare these Instructions for
Use are provided for general information
only and are not intended for use with any
specific machine or device.

Directive 93/42/EEC
concerning Medical Devices

Dräger HomeCare
a division of Dräger Nederland B. V.
Kanaaldijk 29
5683 CR Best, The Netherlands
Phone +31 (0) 499 331 332
Fax +31 (0) 499 331 333
e-mail homecare@draeger.com

Dräger HomeCare
a division of Dräger Nederland B. V.
Kanaaldijk 29
5683 CR Best, The Netherlands
Phone +31 (0) 499 331 332
Fax +31 (0) 499 331 333
e-mail homecare@draeger.com

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57 30 327 de/en
90 37 003
© Dräger HomeCare
2nd edition - March 2000
Subject to alteration