RespiCare SC

Atemtherapie-Gerät
Ventilation Therapy Unit

Gebrauchsanweisung
Instructions for Use
Software 2.n

Inhalt

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Contents

For Your Safety and that of Your Patients	4
Intended Use	6
Preparing for Use	8
Positioning	8
Insert coarse dust filter	9
Connect ventilation hose	10
Connect power supply	11
Preparing for Use – By Doctors Only!	11
Connect control panel	11
Switch on unit	13
Set ventilation modes / parameters	14
When oxygen is applied.....	29
When a humidifier is used.....	30
Operation	31
Start operation	31
Using the soft start function	32
Nasal mask	33
Display airway pressure	33
In the event of a power failure	33
In the event of an alarm	34
Service indicator	36
End of operation	36
Fault – Cause – Remedy	37
Care	39
Stripping down	39
Cleaning / disinfection in the hospital	41
Cleaning / disinfection at home	42

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For Your Safety and that of Your Patients

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

Maintenance

The apparatus must be inspected and serviced regularly by trained service personnel at twelve monthly intervals .
Repair and general overhaul of the apparatus may only be carried out by trained service personnel.
We recommend that a service contract be obtained with Dräger Medical and that all repairs also be carried out by them.
Only authentic Dräger Medical spare parts may be used for maintenance.
Observe chapter "Maintenance Intervals".

Accessories

Do not use accessory parts other than those in the order list.

Not for use in areas of explosion hazard

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

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Dräger Medical b.v.

Safe connection with other electrical equipment

Electrical connections to equipment which is not listed in these Instructions for Use should only be made following consultations with the respective manufacturers or an expert.

Liability for proper function or damage

The liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by Dräger Medical or if the apparatus is used in a manner not conforming to its intended use.

Dräger Medical cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Dräger Medical are likewise not modified by the recommendations given above.

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Intended Use

RespiCare® SC – respiratory support unit for time-cycled, pressure-controlled ventilation support for therapy reasons. For use only with patients who can maintain spontaneous breathing, and not for unconscious or non-spontaneously breathing patients.

RespiCare SC should only be used with a non-invasive ventilation connector such as nasal mask or full face mask.

For sleep apnoea therapy, for weaning long-term ventilation patients and for disorders of the respiratory muscles.

For use
in hospitals,
in sleep laboratories,
at home,
for stationary or mobile use in vehicles and wheelchairs.

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RespiCare SC includes the following ventilation modes

CPAP – Breathing with continuous positive pressure in the airways.

CPAP/ASB – Spontaneous breathing with synchronized assistance.

PCV(S) – Pressure controlled, time cycled ventilation with synchronized inspiration phase.

PCV – Pressure controlled ventilation.

PCV(+) – Spontaneous breathing with time cycled changeover between two pressure levels and automatic synchronization during pressure changeover.

With disconnection/stenosis alarm.

The unit may only be used as prescribed by a doctor and with the settings specified by the doctor!

The unit must not be used for life-saving functions.

Mobile telephones must not be used within a distance of 10 metres from the unit!

Mobile telephones can impair the correct functioning of electro-mechanical equipment and can therefore put patients at risk.

The unit must not be used in explosion-hazard areas!
Risk of fire and explosion.

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Preparing for Use

Positioning

- The RespiCare SC must be set up on a level, horizontal surface, such as a bedside table, wall shelf or trolley. Ventilation slits in the base of the housing must remain clear!
Ensure that the unit cannot fall down!
- The unit must not be set up near a heater and must not be exposed to direct sunlight, otherwise it may overheat!
Ensure that air can circulate freely around the unit.
- Filter and silencer on the rear of the unit must not be blocked by curtains, blankets, etc. otherwise air cannot be drawn into the unit!
- The vent in the carrying handle must not be clogged!
- Ensure that the unit cannot fall into full bathtubs, basins, etc. as this can cause electric shocks!

When used out of doors, e.g. in a wheelchair:

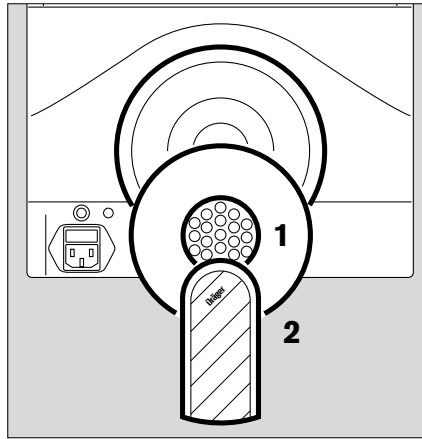
- Ensure that the unit is protected from wetness!

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Insert coarse dust filter

- 1 Plug new coarse dust filter into insert on rear of unit and turn clockwise as far as possible = lock into place.
- 2 Plug silencer into filter housing until it engages.

The coarse dust filter traps coarse particles, such as fluff.

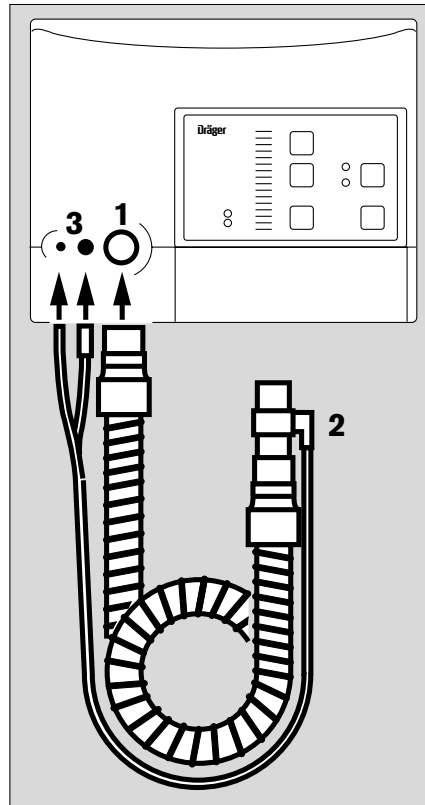
Allergic patients can use a fine dust filter to trap household dust, pollen, etc.

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Connect ventilation hose

Only use original ventilation hoses!

- 1 Firmly plug the ventilation hose into the socket on the unit.
 - 2 Plug the other end of the ventilation hose into the E-Vent test outlet – test hoses pointing towards ventilation hose.
 - 3 Plug test hoses into the sockets on the unit – note different diameters.
- Connect the E-Vent test outlet with the port on the nasal mask.
A nasal mask without outflow port must be used!
CO₂ is purged from the system via the E-vent test outlet.

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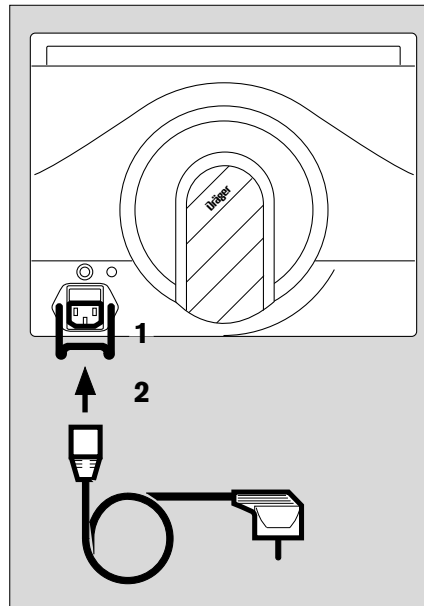
Connect power supply

The RespiCare SC units are designed for operation on voltages between 100 and 240 V. They are set automatically.

- 1 Press the safety clip down to open.
- 2 Plug the power cord into the socket on the rear of the unit and into the mains socket.
- 1 Push the safety clip up until the power cord is clamped in place.

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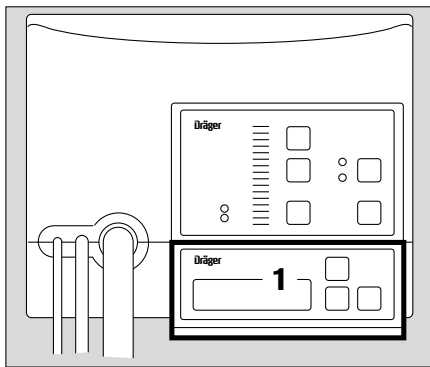
Every basic RespiCare unit can be set as required with the aid of the control panel.

The control panel may only be used by a doctor or by someone who has been duly authorized by the doctor!

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
- 1 Insert the control panel in the magnetic holder on the front of the unit. Lay the unit on its side, press the lead into the duct in the base of the unit and

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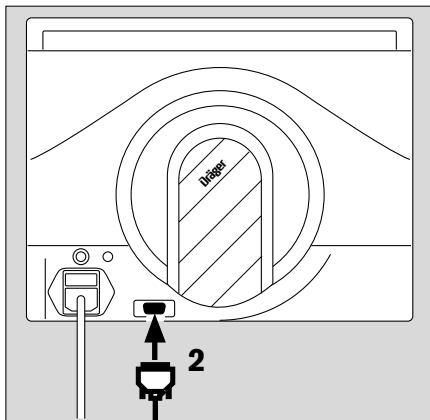- 2 plug the control panel lead into the socket on the rear.

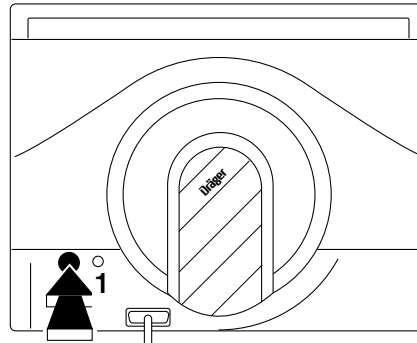
- Return the unit to an upright position.

Remote operation of the control panel:

- Connect the extension lead to the control panel plug and plug the extension lead into the socket on the rear of the unit.
Only use the original extension lead.
Other components must not be connected.

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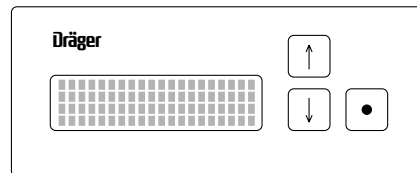
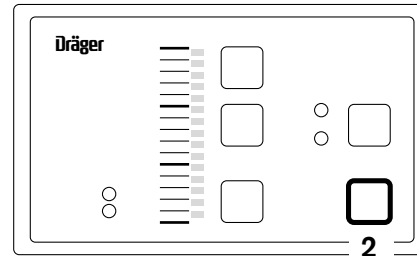
Switch on unit

- 1 Press on /off switch on rear of unit – until it engages = ON.
- The RespiCare SC unit starts a self-test for approx. 1 second to test the internal program memories. All LEDs briefly light up during this time.
- 2 The green LED in the  button remains alight.

On the control panel:

- All pixels appear on the display and the display is illuminated for approx. 1 second. The model name then appears in the first line, together with the last ventilation mode selected. This line remains on display as the status line.
Example:
Type SC CPAP/ASB

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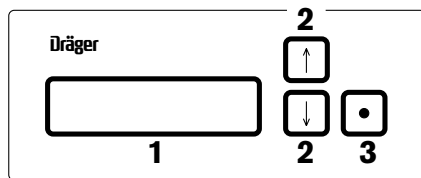
Set ventilation modes / parameters

This is done with the aid of the control panel.

The control panel may only be used by a doctor or by someone who has been duly authorized by the doctor!

1 Display - example:

↓ **Type SC CPAP** ↓

Vertical arrows identify the selected line and the direction in which additional lines can be displayed.

2 Press button  or  to select a line.

3 Press button  to activate the line. The two arrows flash and point towards the text. Example:

→ **Paw = 20 mbar** ←

The mode or parameter can now be set:

2 Use the buttons  or  to set the required values. **The settings are immediately adopted by the unit.**

3 Press button  to confirm.

The display is automatically illuminated for two minutes after pressing one of the three buttons.

The settings are saved even when the unit is switched off.

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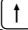
Set CPAP

CPAP – Continuous Positive Airway Pressure
Spontaneous breathing with continuous positive pressure in the
airways.

- Press  to select the first line
and activate by pressing 
- Use  to set **CPAP** mode
and press  to confirm.
- Press  to select **Paw = __ mbar**
and activate by pressing 
- Press  or  to set **Paw = __ mbar**
and press  to confirm.

Adjustment range: 3 to 20 mbar
in increments of 0.2 mbar

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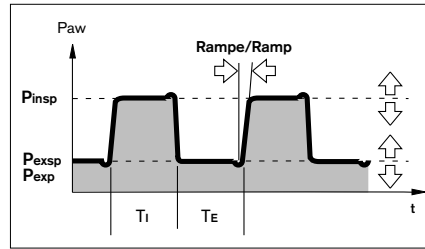
CPAP – Continuous Positive Airway Pressure

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Set CPAP/ASB

CPAP – Continuous Positive Airway Pressure

Spontaneous breathing with continuous positive pressure in the airways.

ASB – Assisted Spontaneous Breathing
Spontaneous breathing with synchronized assistance.

With every spontaneous inspiration, the unit provides assistance until the pressure level P_{insp} is reached and then switches over to pressure level P_{exp} when expiration commences or after not more than 4 seconds.

- Press to select the first line and activate by pressing
- Use or to set **CPAP/ASB** mode and press to confirm.
- Press to select **$P_{\text{insp}} = _ _ \text{mbar}$** and activate by pressing
 P_{insp} : inspiratory pressure.
- Use or to set **$P_{\text{insp}} = _ _ \text{mbar}$** and press to confirm.

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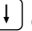
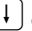
- Press  to select **Pexp** = **__mbar**
and activate by pressing 
P_{exp}: expiratory pressure.
- Use  or  to set
Pexp = **__mbar** and press  to confirm.

After switching on or disconnecting the unit, wait for a few breaths until the system has stabilized before setting the trigger.

- Press  to select **Trigg.I** = **_ of 5**
and activate by pressing 
Trigg.I: Sensitivity for synchronizing spontaneous breathing and inspiration.
1 = maximum sensitivity
- Use  or  to set **Trigg.I** = **_ of 5**
and press  to confirm.
- Press  to select **Trigg.E** = **_ of 5**
and activate by pressing 
Trigg.E: Sensitivity for synchronizing spontaneous breathing and expiration.
1 = maximum sensitivity
- Use  or  to set **Trigg.E** = **_ of 5**
and press  to confirm.

Each inspiration triggered by spontaneous breathing will be indicated by a star in the first line of the control panel display.

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- Press  to select **Ramp = 0._ s** and activate by pressing 
Ramp: Time for pressure increase from P_{exp} to P_{insp} .
0.0 s = steepest pressure increase
- Use  or  to set **Ramp = 0._ s** and press  to confirm.

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


Apnoea ventilation

CPAP/ASB mode can optionally be combined with apnoea ventilation. In this case, the unit switches over to time controlled ventilation if it does not detect any spontaneous breathing within a definable time. The unit stops time controlled ventilation when spontaneous breathing is restored, and at the latest after one minute.

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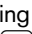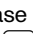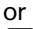- Press  to select **ApneaVent** = __,
and activate by pressing 
- Press  to set **ApneaVent = On**,
and press  to confirm.
- Press  to select **AV-Delay** = __ s,
and activate by pressing 
AV = Apnea Ventilation
AV-Delay = Time after end of inspiration phase.
- Press  or  to set **AV-Delay** = __ s,
and press 
- Press  to select **AV-Freq** = __ /min,
and activate by pressing 
AV-Freq = Ventilation frequency during apnoea ventilation
- Press  or  to set **AV-Freq** = __ /min,
and press 
- Press  to select **AV-Insp** = __ %,
and activate by pressing 
AV-Insp = Inspiration as a proportion of the overall
breathing phase in %
- Press  or  to set **AV-Insp** = __ %,
and press 

Adjustment ranges for apnoea ventilation

Parameter	Range	Increments
AV-Delay	5 to 30 s	1 s
AV Freq	6 to 40 /min	1 /min
AV Insp	20 to 50 %	2 %

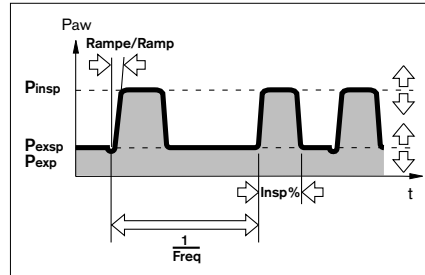
PCV(S) einstellen

Pressure Controlled Ventilation Synchronized

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01290131

Set PCV(S)

Pressure Controlled Ventilation Synchronized

Pressure controlled and time cycled ventilation, by which the patient can trigger an early ventilation stroke in synchrony with spontaneous inspiration, thereby increasing the ventilation frequency.

The inspiration time is predetermined by the setting and limited to not more than 2 seconds.

- Press to select the first line and activate by pressing
- Use or to set **PCV(S)** and press to confirm.
- Press to select **P_{insp} = __ mbar** and activate by pressing
P_{insp}: inspiratory pressure.
- Use or to set **P_{insp} = __ mbar** and press to confirm.
- Press to select **P_{exp} = __ mbar** and activate by pressing
P_{exp}: expiratory pressure.
- Use or to set **P_{exp} = __ mbar** and press to confirm.

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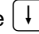
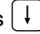
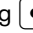
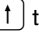
- Press  to select **Freq = __/min**
and activate by pressing 
Freq: Controlled ventilation frequency

- Use  or  to set **Freq = __/min**
and press  to confirm.

- Press  to select **%Insp = __%**
and activate by pressing 
% Insp: Inspiration share of the respiratory phase in %.

- Use  or  to set **%Insp = __%**
and press  to confirm.

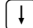
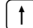
After switching on or disconnecting the unit, wait for a few breaths until the system has stabilized before setting the trigger.

- Press  to select **Trigg.I = _ of 5**
and activate by pressing 
Trigg.I: Sensitivity for synchronizing spontaneous breathing and inspiration.
1 = maximum sensitivity

- Use  or  to set **Trigg.I = _ of 5**
and press  to confirm.

Each inspiration triggered by spontaneous breathing will be indicated by a star in the first line of the control panel display.

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- Press  to select **Ramp = 0._ s**
and activate by pressing 
Ramp: Time for pressure increase from P_{exp} to P_{insp} .
0.0 s = steepest pressure increase
- Use  or  to set **Ramp = 0._ s**
and press  to confirm.

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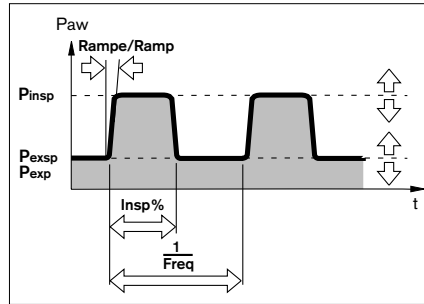
Set PCV

PCV – Pressure Controlled Ventilation
Pressure controlled, time cycled
ventilation.

- Press \uparrow to select the first line and
activate by pressing \bullet
- Use \uparrow to set **PCV** and press \bullet to
confirm.
- Press \downarrow to select **P_{insp}** = __ **mbar**
and activate by pressing \bullet
P_{insp}: inspiratory pressure.
- Use \downarrow or \uparrow to set **P_{insp}** = __ **mbar**
and press \bullet to confirm.
- Press \downarrow to select **P_{exp}** = __ **mbar**
and activate by pressing \bullet
P_{exp}: expiratory pressure.
- Use \downarrow or \uparrow to set **P_{exp}** = __ **mbar**
and press \bullet to confirm.
- Press \downarrow to select **Freq** = __ **/min**
and activate by pressing \bullet
- Use \downarrow or \uparrow to set **Freq** = __ **/min**
and press \bullet to confirm.

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- Press to select **%Insp = __%**
and activate by pressing
% Insp: Inspiration share of the respiratory phase in %.
- Use or to set **%Insp = __%**
and press to confirm.
- Press to select **Ramp = 0._ s**
and activate by pressing
Ramp: Time for pressure increase from P_{exp} to P_{insp} .
0.0 s = steepest pressure increase
- Use or to set **Ramp = 0._ s**
and press to confirm.

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
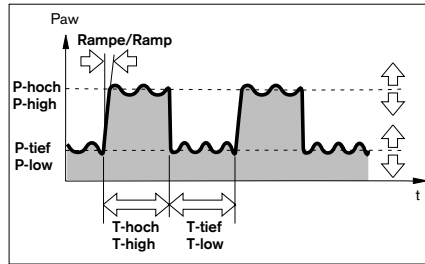
Set PCV(+)

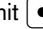Pressure Controlled Ventilation

Time controlled changeover between two pressure levels, with the possibility of spontaneous breathing at both pressure levels. The pressure changeovers are synchronized with spontaneous breathing.


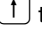
- Press to select the first line and activate by pressing
- Use or to set **PCV(+)** and press to confirm.
- Press to select **P-high = __ mbar** and activate by pressing
P-high: upper pressure level
- Use or to set **P-high = __ mbar** and press to confirm.
- Press to select **P-low = __ mbar** and activate by pressing
P-low: lower pressure level
- Use or to set **P-low = __ mbar** and press to confirm.

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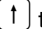- Press  to select **T-high** = __s
and activate by pressing 
T-high: time for upper pressure level
- Use  or  to set **T-high** = __s
and press  to confirm.
- Press  to select **T-low** = __s
and activate by pressing 
T-low: time for lower pressure level
- Use  or  to set **T-low** = __s
and press  to confirm.

After switching on or disconnecting the unit, wait for a few breaths until the system has stabilized before setting the trigger.

- Press  to select **Trigg.I** = _ of 5
and activate by pressing 
Trigg.I: Sensitivity for synchronizing spontaneous breathing and inspiration.
1 = maximum sensitivity
- Use  or  to set **Trigg.I** = _ of 5
and press  to confirm.

Each inspiration triggered by spontaneous breathing will be indicated by a star in the first line of the control panel display.

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- Press to select **Trigg.E** = **_ of 5**
and activate by pressing
Trigg.E: Sensitivity for synchronizing spontaneous breathing and expiration.
1 = maximum sensitivity
- Use or to set **Trigg.E** = **_ of 5**
and press to confirm.
- Press to select **Ramp** = **__%**
and activate by pressing
Ramp: Time for pressure increase from P-low to P-high as a percentage of T-high.
- Use or to set **Ramp** = **__%**
and press to confirm.

Setting ranges for PCV(+)

Parameter	Range	Increments
P-high	5 to 30 mbar	1 mbar
P-low	2 to 20 mbar	1 mbar
T-high	1 to 60 s	1 s
T-low	1 to 60 s	1 s
Trigg.l	1 to 5 *	1 step
Trigg.E	1 to 5 *	1 step
Ramp	0 to 50 %	10 %

* Trigger sensitivity: see Technical Data, page 55

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Activate / deactivate disconnection / stenosis alarm

- Press  or  to select **Alarm** and activate by pressing 
- Select required status
Alarm = active by pressing 
or
Alarm = inactive by pressing  and press  to confirm.

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- Press  or  to select **Patient = ___h**.

Reset application time to 0:

- Press  or  to select **Patient = ___h**, press  to confirm and
- press  to reset display to 0.

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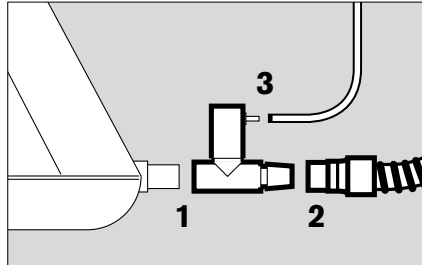
When oxygen is applied

Oxygen actively supports combustion. No smoking, no naked lights!

Use the oxygen adapter, see order list, page 62.

Other adapters must not be used!

The oxygen adapter prevents oxygen leaking into the unit when the fan is defective (no airway pressure).

Oxygen must not be directed into the unit or intake port, due to the risk of fire!

- 1 Plug the oxygen adapter onto the socket of the RespiCare SC.
Ensure that the valve housing points upwards.
- 2 Plug the ventilation hose onto the socket of the oxygen adapter.
- 3 Plug the oxygen metering hose onto the socket of the adapter.

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
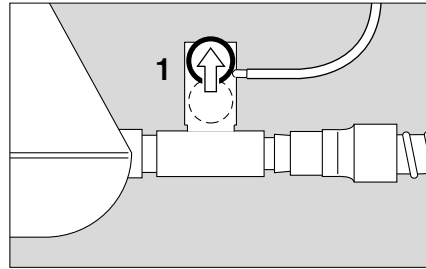
- Max. permissible oxygen flow: 10 L/min.

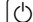Switch on the RespiCare SC:

- Press on/off switch,
 - press button , the green LED goes out.
- 1 The ball in the oxygen adapter **must** move upwards when the fan is switched on.
- Turn on the oxygen supply.

After use:

- **Turn off the oxygen supply before switching off the RespiCare SC.**

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When a humidifier is used

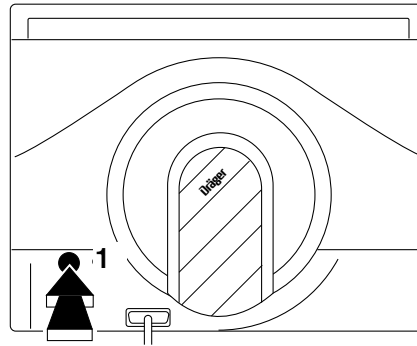
In case of irritation or dryness of the upper airway a humidifier can be used in order to prevent this. For installation of a humidifier and its breathing tube system carefully follow the instructions in the user manual of the breathing systems (57 30 161 / 57 02 17)

When using a humidifier it is not allowed to use the E-Vent (2M 86 522)

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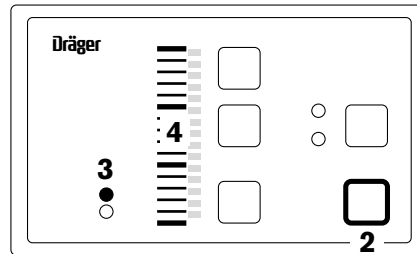
Operation

Start operation

- 1 Press on/off switch on rear of unit – until it engages = ON.
The RespiCare SC performs a self-test.
- 2 The green LED in the  button lights up and the unit is now on standby.
- 3 The »min.« (minutes) LED lights up while the unit is on standby.
- 4 The bar display indicates the set time for the "soft start" function: 1 graduation represents 2 minutes.
To set the soft start function, see page 32.
- 2 Press the  button. The green LED goes out.
The soft start time display goes out, and the blower starts delivering air.

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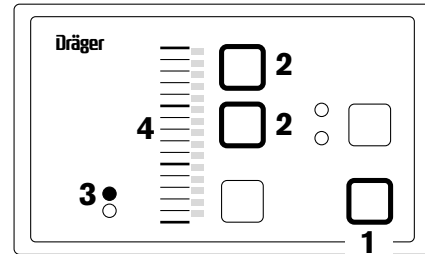
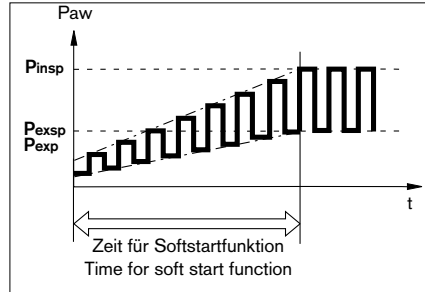
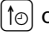
Using the soft start function

Can only be set in standby mode.

For slowly and "softly" increasing the set airway pressure over a variable time of 0 to 30 minutes.

- 1 Press  to switch to standby.
The green LED in the button lights up.
- 2 Press  or 
- 3 The »min.« LED lights up.
- 4 The time (minutes) elapsed before reaching the airway pressure set on the control panel is indicated on the bar graph.
Each gradation corresponds to 2 minutes.
- 2 Press  or  to set the required time.
The time changes by 2 minutes whenever one of the two buttons is pressed.
- 1 Press  to switch to operating mode. The green LED goes out and the set time becomes effective.

The soft start function is effective whenever changing over from standby to operation and is also retained after switching off the unit.

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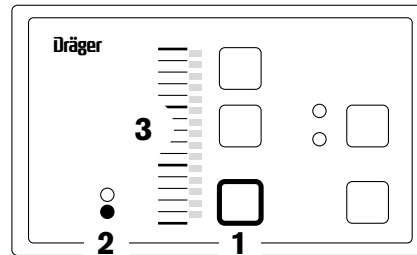
Nasal mask

- For RespiCare SC, use only nasal masks without outflow port. CO₂ is purged via the E-Vent outflow valve.
- Put on nasal mask in accordance with separate Instructions for Use.

Display airway pressure

during operation.

- 1 Press :
- 2 The »mbar« LED lights up.
- 3 The current airway pressure is shown on the bar graph. Each gradation corresponds to 2 mbar.

In the event of a power failure

a continuous tone sounds for at least 2 minutes.
This acoustic alarm cannot be switched off.

- Remove the nasal mask to avoid any accumulation of CO₂ and a shortage of oxygen!

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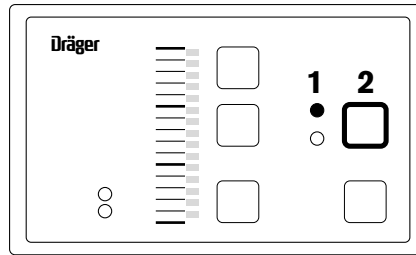
In the event of an alarm

- 1 **Intermittent tone and flashing red alarm LED » Δ «:**
Disconnection or stenosis.

The intermittent tone can be suppressed for 2 minutes:

- 2 Press 
- Eliminate the cause of the disconnection/stenosis.
Further information for troubleshooting can be found in the table "Fault – Cause – Remedy" on page 37.
The flashing red alarm LED » Δ « goes out and the intermittent tone stops when the cause of the fault has been remedied.

The following message flashes in the first line when the control panel is connected:

Disconnect/Stenosis!

The message remains on the display even when the cause has been remedied.

To clear the display:

- Press  to select the appropriate line and then press  to confirm.

The status display moves to the second line. Parameters can also be set even while an alarm is being displayed.

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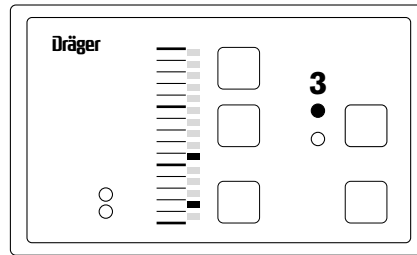
3 Continuous tone and red alarm LED » ⚠ « lights up steadily:

Equipment fault.

Individual segments of the bar graph light up constantly.

The alarm tone cannot be switched off.

The fan switches off.

- Remove the nasal mask to avoid any accumulation of CO₂ and a shortage of oxygen!
- Read off the illuminated gradations on the bar graph (example: 4/12) and report these to the service technician. The numbers refer to the equipment fault.

The following message appears on the display of the control panel when the latter is connected:

Equipment fault, technical defect

with the same numbers as on the bar graph.

Continuous tone without visual indication

draws attention to a power failure.

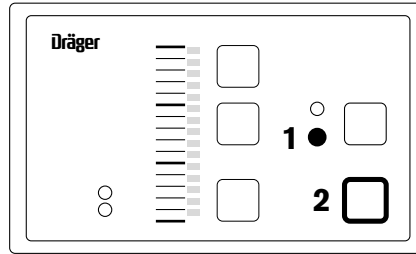
This alarm tone cannot be switched off.

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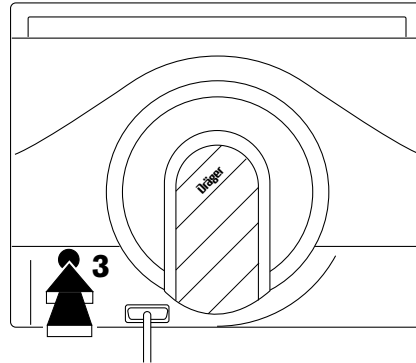
Service indicator

The required inspection and maintenance intervals are monitored and signalled automatically by the RespiCare SC unit:

- 1 Yellow LED lights up »  «: The unit must be inspected and serviced by experts within the next two months. The LED does not go out until the unit has been inspected and serviced.

End of operation

When oxygen has been applied:

- **Turn off the oxygen supply first** and then switch the unit to standby or off.

In the event of brief interruptions up to approx. 15 minutes:

- 2 Press  and the green LED lights up. The unit remains on standby.

For longer breaks:

- 2 Press  and the green LED lights up. The unit is on standby.
- 3 Depress switch on rear of unit completely = OFF. The unit is now switched off.

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Fault – Cause – Remedy

If a fault occurs, the following table can help to identify the underlying cause and remedy it.

Fault	Cause	Remedy
Intermittent tone, red alarm LED »  « flashes.	Disconnection or stenosis.	Ensure that ventilation hose, test hoses, nasal mask are fitted without leaks or kinks.
Airway pressure is not reached during inspiration.	Coarse dust filter/ fine dust filter clogged.	Clean coarse dust filter, page 46, or replace it. Replace fine dust filter.
Airway pressure has increased significantly.	Leak in pressure measuring hose or hose has slipped off socket.	Ensure that there are no leaks or kinks in the test hoses.
Unit does not switch over to pressure assistance in CPAP/ASB mode.	Leak in test hoses or hoses buckled.	Ensure that there are no leaks or kinks in the test hoses.
No pressure change in PCV(+) and PCV modes.	Pressure measurement defective.	Ensure that there are no leaks or kinks in the test hoses.

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Fault	Cause	Remedy
Trigger function or pressure control defective.	Water residue in test hoses.	Fully dry out the test hoses.
Set pressure parameters too low after switching on.	Soft start function effective.	Check setting of soft start function, page 32.
Continuous tone, red alarm LED »▲« lights up continuously.	Equipment fault.	Call Service technician.
Continuous tone, all lamps off.	Fuse has blown, no mains power.	Replace fuse, page 47. See fuse ratings in Technical Data, page 55.

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If the faults cannot be remedied:

- Use a spare unit and call a Service technician.

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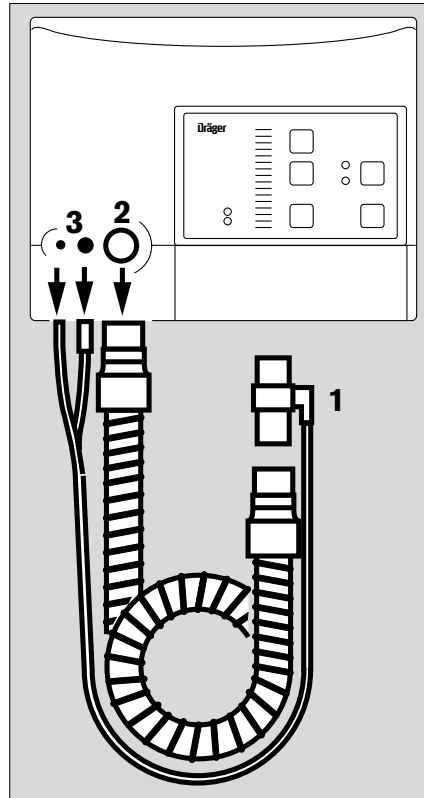
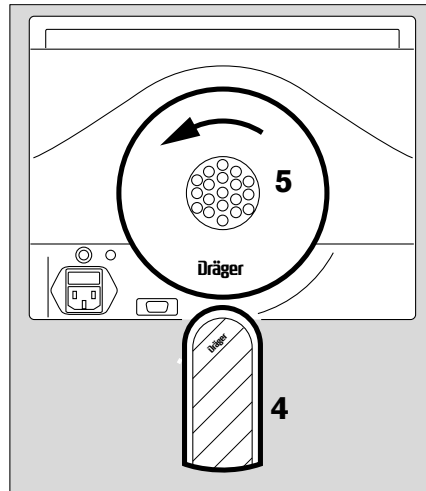
Otherwise, risk of electric shock!

Stripping down

- 1 Remove E-Vent from ventilation hose.
- 2 Pull ventilation hose off its socket and unplug the connector to the oxygen adapter if applicable.
- 3 Remove test hoses from sockets.

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On the rear:

- 4 Remove silencer.
- 5 Turn coarse dust filter or fine dust filter anticlockwise = unlock. Remove filter.
- Clean coarse dust filter, see page 46. Fine dust filter must be replaced after approx. 500 hours of application.

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Cleaning / disinfection in the hospital

Use a surface disinfectant, such as a product based on:

- Aldehydes
- Alcohol
- Quaternary ammonium compounds.

The following are not suitable:

- Compounds containing alkylamine
- Compounds containing phenol
- Compounds which give off halogens
- Strong organic acids
- Compounds which give off oxygen

Users in the Federal Republic of Germany are advised to use the disinfectants specified in the current DGHM* list. This list also specifies the active agent in each disinfectant. Products based on the active agents specified above are recommended for users in countries in which the DGHM list is not available.

Control panel, basic unit, cable

- Wipe-disinfecting:
The following disinfectants can e.g. be recommended:
Incidur[®], Frekanol[®], Gigasept FF[®]
Note the manufacturer's directions.
Liquid must not be allowed to leak into the connections.

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Breathing tube with integrated measuring tubes, SilentFlow and oxygen adapter

- Disinfect in a bath, e.g. with Gigasept FF[®]
The parts must subsequently be rinsed in clear boiled water. Thoroughly shake off all remaining water. Allow the parts to dry completely. Water in the test hoses may impair the unit's correct operation.

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Nasal mask

- Must be prepared in accordance with the separate Instructions for Use.

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Control panel, basic unit, cable

- Remove any dirt with a disposable cloth.
Liquid must not be allowed to leak into the connections.

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Breathing tube with integrated measuring tubes, SilentFlow and oxygen adapter

Specific cleaning and disinfecting procedure information is found in the user manuals of the accessories.

General cleaning instructions

- Remove all visible deposits.
- Clean the accessories with warm soapy water.
- Rinse with boiled lukewarm water.
- Dry parts thoroughly.
Water in the test hoses or in the oxygen adapter may impair the operation of the unit.
Water in the breathing tube or in the SilentFlow with measuring adapter can reduce the air flow through the measuring adapter and allow the CO₂ content of the inspiratory air to rise.

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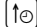
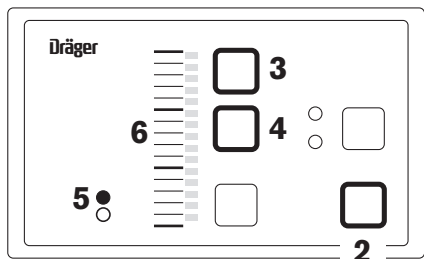
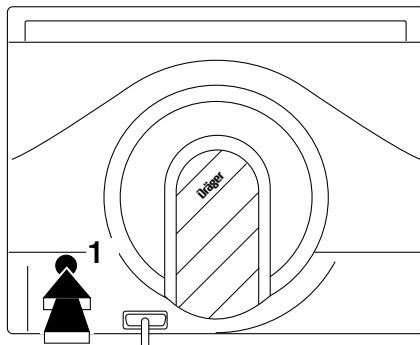
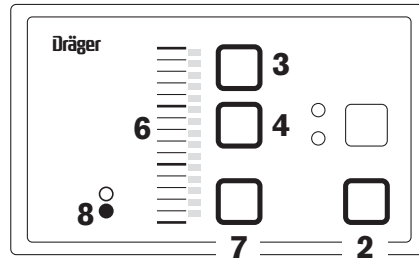
Check Readiness for Operation


The unit's readiness must be checked whenever it has been cleaned and disinfected.

After assembling the RespiCare SC is ready for operation:

- 1 Switch on unit:
Press button on rear of unit – until it engages = ON.
- 2 The green LED in the  button lights up.
- 3 Press 
- 5 LED »min.« lights up.
- 6 The time for the soft start is shown on the bar graph in minutes.
- 4 Press  and set the time for the soft start function to zero.
- Put on a suitable nasal mask and
- 2 press  to switch on the fan.

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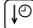- 7 Press :
- 8 LED »mbar« lights up.
- 6 The set airway pressure must appear on the bar graph.
- 2 Press , the green LED lights up and the fan is switched off.
- Remove nasal mask.
- 4 Press .
- 3/4 Use the buttons  or  to set the time for the soft start.

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The RespiCare SC is now ready for use!

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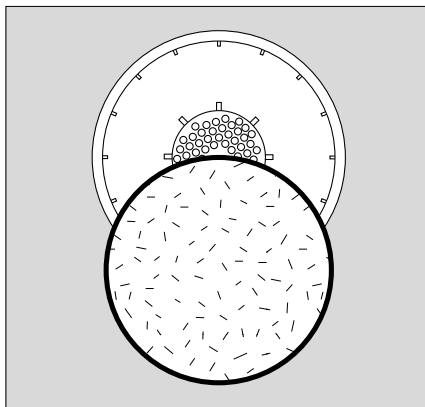
Maintenance Intervals

Coarse dust filter

Must be cleaned every three weeks.
The filter must be replaced if damaged,
at the latest every six months.
It can be disposed of as household
refuse.

Fine dust filter

Must be replaced after approx. 500 hours
of application.
It must be replaced at the latest after six
months.
The filter can be disposed of as house-
hold refuse.

Inspection and maintenance

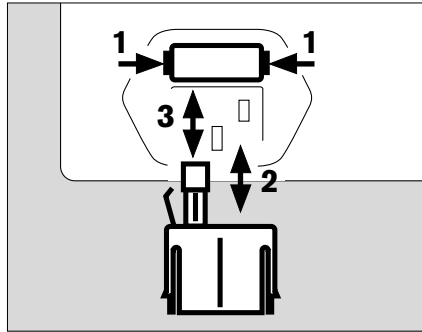
As indicated by the service indicator
» 🔧 « see page 36.
At least once per year.

Clean coarse dust filter

- Every three weeks.
- Draw foam insert out of filter housing.
- Wash foam insert in soapy water,
rinse thoroughly in clear water and
dry completely.
- Press foam insert back into filter
housing.
- Damaged foam inserts must be
replaced.

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Replace fuse

- Disconnect mains plug!
- 1 Press the two tabs on the fuse insert together and simultaneously
- 2 draw out the insert.
- 3 Remove the blown fuse and insert a new fuse.
Use a slow-blow fuse T 2 H 250 V
IEC 127-2/V.
- Press the fuse insert in – until it engages.

Disposal of the unit

- when it reaches the end of its useful life.
- RespiCare SC must be appropriately disposed of in agreement with the competent specialized waste disposal company.
All applicable legal regulations must be obeyed.

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← - 001 + →

Customer Service Mode

For selecting the language in which messages are displayed on the control panel.

For testing the alarm tone and lamps.

For indicating the hours of operation.

For indicating the software version.

Select customer service mode

- Disconnect RespiCare SC from the patient and connect the control panel.
- Depress on/off switch completely = OFF.
- Press on/off switch – until it engages = ON simultaneously:
- press  for at least 3 seconds.
- The RespiCare SC unit runs a self-test before the following message appears:

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Set language

To activate:

- Briefly press and the set language appears on the second line.

Display (example):

Adjust language
deutsch

- The following languages can be set as required using the and buttons:

English	
German	(deutsch)
French	(français)
Spanish	(español)
Italian	(italiano)
Portuguese	(português)
Swedish	(svenska)
Dutch	(nederlands)
Danish	(dansk)

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Pat: Period of use by patient

or

Service: Hours of operation since unit was last serviced

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← - 004 + →

Press to select:

Main device: Software version of the basic unit

Remote device: Software version of the control panel

To end the display:

- Press again.

To leave customer service mode:

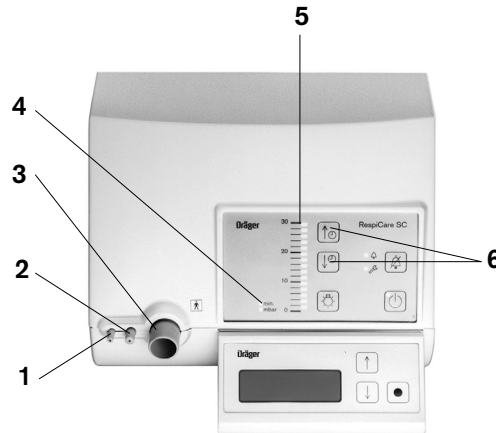
- Depress on/off switch completely = OFF.

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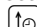
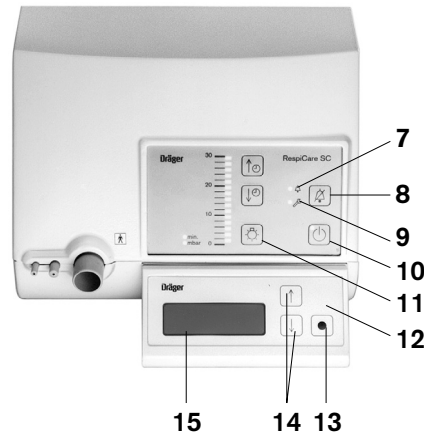
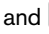
- 1 Socket for "airway pressure" test hose
- 2 Socket for "differential pressure" test hose
- 3 Socket for ventilation hose
- 4 Yellow LEDs for bar graph
– »min.« LED lights up: time for soft start (minutes)
– »mbar« LED lights up: airway pressure
- 5 Bar graph for airway pressure (mbar) or time for
soft start (minutes)
- 6 Two buttons for setting the time for the soft start
 = decrease,  = increase

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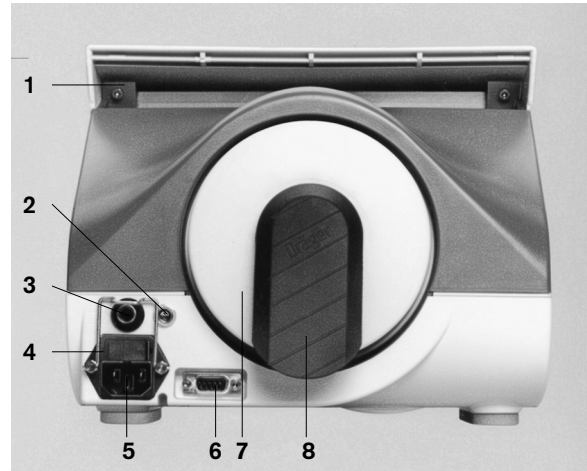
- 7 Red alarm LED »  «
- 8 Button  to suppress the alarm tone for approx. 2 minutes
- 9 Yellow LED »  « for service required
- 10 Button  to switch the fan on and off (standby)
- 11 Button  to switch the bar graph illumination on and off
- 12 Control panel for settings and data
- 13 Button  to activate and confirm settings
- 14 Two buttons  and  for making selections and settings
- 15 Display for settings, data and messages

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- 1 Handle for carrying the unit, also serves as vent
- 2 Analog output for airway pressure
- 3 On/off switch
- 4 Mains fuse
- 5 Device socket for power cord
- 6 Connection for control panel
- 7 Coarse dust filter / fine dust filter
- 8 Silencer

Rating plate on underside
(not illustrated)

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Ambient conditions

In operation

Temperature	5 to 35 °C
Air pressure	600 to 1100 hPa
Rel. humidity	10 to 95 %, no condensation

In storage

Temperature	-20 to 70 °C
Air pressure	600 to 1200 hPa
Rel. humidity	10 to 95 %, no condensation

Settings

Ventilation mode CPAP

Airway pressure	3 to 20 mbar
-----------------	--------------

Ventilation mode CPAP/ASB

Inspiration pressure P_{insp}	5 to 30 mbar
Expiration pressure P_{exp}	2 to 20 mbar
Trigger sensitivity Trigg.I	5 settings (approx. 10 to 50 L/min)
Trigger sensitivity Trigg.E	5 settings (approx. 15 to 75 L/min below peak flow)

Ramp	11 settings (0 to 1.0 s)
------	-----------------------------

Apnea Ventilation	ON/OFF
Delay time	5 to 30 s
AV-Frequency	6 to 40/min (bpm)
AV-Insp.	20 to 50 %

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Device parameters

Device compliance (with patient hose system)	≤0.6 mL/mbar
Maximum pressure limit	40 mbar (on patient)
Inspiration and expiration resistance	max. 6 mbar at 60 L/min
Airway pressure measurement	
Range	0 to 30 mbar
Resolution	2 mbar
Accuracy	±2 mbar
Monitors	
Disconnection/stenosis alarm	Triggered when alarm limit is exceeded for at least 5 seconds. Alarm limit for airway pressures >10 mbar: actual value at least 5 mbar below set value. Alarm limit for airway pressures ≤10 mbar: actual value less than 50 % of set value. The alarm is suppressed for approx. 1 minute after switching on the fan and when changing the ventilation mode.

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Operating parameters

Rated voltage	Range: 100 to 240 V AC ±10 % 50/60 Hz
Rated current	1.5 to 0.6 A
Power consumption	max. 100 W
Device fuse	Slow-blow T 2 H 250 V IEC 127-2/V
Classification as per EN 60601-1	Protection class II
Sound pressure at 10 mbar Free-field measurement over a reflecting surface, distance 1 m	max. 40 dB (A)
Dimensions (W x H x D)	
Basic unit (incl. hose sockets)	255 x 217 x 288 mm
Control panel	150 x 65 x 75 mm
Weight	
Basic unit	4.2 kg
Control panel	0.25 kg
Connection for ventilation hose	Dia. 22 not tapered

Airway pressure output Paw (t)

Output voltage	40 mV/mbar, 0 mbar = 0.9 V
Load impedance	>100 kΩ
Wiring	Brown = +, white = – shield connected to plug housing

Elektromagnetische Verträglichkeit EMV	geprüft nach EN 60 601-1-2	Electromagnetic compatibility EMC	Tested to EN 60 601-1-2
Klassifizierung gemäß Richtlinie 93/42/EWG Anhang IX	Klasse II a	Classification as per EC Directive 93/42/EEC Annex IX	Class II a
UMDNS-Code Universal Medical Device Nomenclature System – Nomenklatur für Medizingeräte	14-355	UMDNS-Code Universal Medical Device Nomenclature System – Nomenclature for medical products	14-355

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Abbreviations and Symbols

Apnea- Vent	Pressure controlled, time controlled ventilation in the event of apnoea
AV-Delay	Selectable delay time between the end of the inspiration phase and the start of apnoea ventilation
AV-Freq	Ventilation frequency during apnoea ventilation
AV-Insp	Inspiration phase as a proportion of the overall respiration phase during apnoea ventilation
ASB	A ssisted S pontaneous B reathing Spontaneous breathing with synchronized assistance
Op. time	Operating time counter for the total operating time of the unit in hours
CPAP	C ontinuous P ositive A irway P ressure B reathing Spontaneous breathing with continuous positive pressure in the airways

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Netzschalter

Typ BF = body floating

Standby button for switching the fan
on and off

Setting the time for the soft start

Switches the bar graph illumination
on and off

Making selections and settings

Activating and confirming settings

Suppresses the alarm tone for approx. 2 minutes

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Order List

Designation/Description	Part No.
RespiCare SC , consisting of: Basic unit, dust filter, mains cable, ventilation tube, E-Vent exp. valve, measuring tube system, Instructions for Use and carrying bag	57 30 374
RespiControl	57 30 310
Tube system disp.	57 02 176
Tube system reus.	57 30 161
Tube humid disp.	57 02 178
Tube humid. reus.	57 02 177
Measuring adapter	57 02 200
Mask connection ISO-15, 5 pieces	57 30 316
Dust filter	57 30 317
Micro filter	57 30 461
Carrying bag RespiCare	84 13 396
Silencer	57 30 320
Data cable, 30 m	84 13 357
Converter DC/AC 12 to 230 V	57 30 322
Cable control analog, 10 m	84 13 398
Oxygen adapter	84 13 397
Trolley	2M 86 029
Trolley table	2M 86 030
Cable mains Euro II	57 30 042
Cable mains GB CL I	57 30 256

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Index

Abbreviations	59
Accessories.....	4
Airway pressure, display.....	33
Alarms.....	34
Application time, register.....	28
Buzzer and LEDs, test	50
Care	39
Cleaning.....	41
Coarse dust filter, clean.....	46
Coarse dust filter, insert.....	9
Connection to electrical devices.....	4
Control panel.....	11
CPAP, set.....	15
CPAP/ASB, set.....	16
Customer service mode.....	48
Disconnection/stenosis alarm	28
Disinfection.....	41
Disposal of the unit.....	47
Fault – cause – remedy	37
Front view.....	52
Fuse, replace.....	47
Intended use	6
Language, set	49
Maintenance	4
Maintenance intervals.....	46
Mobile telephones.....	7
Nasal mask	33

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These Instructions for Use apply only to
RespiCare SC
with Serial No.:

If no Serial No. has been filled in by
Dräger Medical these Instructions for
Use are provided for general information
only and are not intended for use with any
specific machine or device.

Directive 93/42/EEC
concerning Medical Devices

Dräger Medical b.v.

Kanaaldijk 29
5683 CR Best, The Netherlands
Phone +31 (0) 499 331 332
Fax +31 (0) 499 331 333
e-mail medical.best@draeger.com

57 30 406

90 37 275 – GA 5665.505 de/en
© Dräger Medical b.v.
5th edition - September 2003
Subject to alteration